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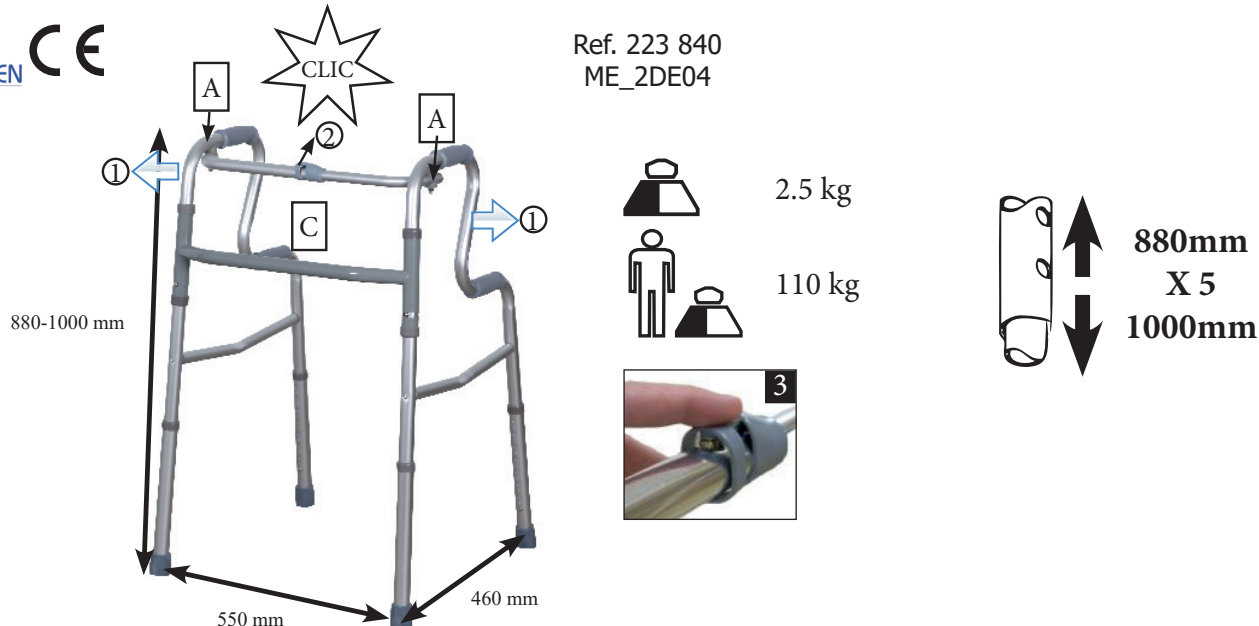
EN- Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips...).
- ♥ Check regularly if all screws are tightened.
- ♥ Hold firmly and completely the handle when using.
- ♥ In case of falling : let the product fall down on the side, so that you don't fall on it.
- ♥ Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
- ♥ Don't use on wet or icy surfaces (snow, ice, water...) !!
- ♥ Don't use in stairs, severe risk of injuries!!
- ♥ Use only as a walking aid and in proper way : don't use as a lever or to move furniture.
- ♥ ALWAYS observe the weight limit on the labelling of your walker.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After unfolding or assembling the walking aid, make sure that it is securely locked in OPEN position and level to the ground before using.
- ♥ DO NOT hang anything on the walking aid.
- ♥ ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.
- ♥ In case you didn't use your walking aid for a long time, let it checked by the technical staff, before using.

- ♥ Be careful when using: beware holes and deformations in the floor.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

2. Intended use, indications and contraindications

Intended use:

Walking aids are used for two purposes: as part of a rehabilitation programme when the user is recovering from an injury or operation and as a long-term aid to mobility when the user has a permanent difficulty with walking.

These devices help in maintaining balance, aid in the prevention of injuries (particularly falls), and allow the person to attend to his/her errands independently. Mobility aids can be used indoor or outdoor, depending on the model.

Mobility aids are used to modify the weight redistribution: some of the weight carried through the legs when walking is transferred through the arms of the frame or stick as it is leant on for support. Products equipped with a seat allow short rest to the user.

Indications

- Loss of autonomy due to walking disorders
- Balance and/or posture disorders
- Walking problems
- Rehabilitation programme after an injury or an operation

Contraindication

Don't use in case of:

- Severe cognitive disorder
- Severe loss of balance
- Severe motor or perception disorders
- Weakness in the upper limbs

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

To mount legs:

Depress the push button in the frame and insert the leg extension to the required height adjustment hole. Make sure that the push button is fully engaged through the extension leg hole.

To unfold:

Swing both side frames [A] out away (1) from the front crossbrace [C] until an audible «click» (2) is heard. The walking frame is then ready for use.

To stand up :

From a seating position, use the lower handles and stand up. Then use the upper handles.

Walking :

To walk, align the middle of your foot with the back legs of the walker. Lift the walker up and place it comfortably in front of you, making sure all four legs are touching the floor. Carefully, walk forward one step at a time.

To fold:

Depress the push button located on the crossbrace as shown on photo (3) and fold the side frames [A] in toward the front crossbrace [C].

Adjusting the height:

The legs are height-adjustable with push buttons: depress the push button in the frame and adjust the leg extension to the required height adjustment hole; the legs must all have the same height. Make sure that the push button is fully engaged through the extension leg hole. Check the stability before use.

3. Cleaning

The walking frame has to be cleaned regularly with a mild cleaning agent and a soft cloth.

NEVER use OILY substances!! Slip hazard!!

Ask your retailer or the technical staff to check the walking aid regularly. Rubber tips must be checked and changed regularly.

4. Materials

The product is made up of aluminium anodised tubing, some models are equipped with an epoxy-treated steel frame, and various plastic components, mostly handles and rubber tips.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect.

This product is biocompatible. Test: EN-ISO 111 99-1

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