DOMICILE - HOMECARE

CONTACT PLUS

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EN - Instructions for installation and use

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HERDEGEN SAS,
46bis avenue du Maine
75015 PARIS – France
qualite@hacare.com

Copyright 2025 par HERDEGEN SAS– ALL RIGHTS RESERVED

In the interests of continuous improvement, HERDEGEN SAS reserves the right to change any detail or specification in this document without prior notice.

The information contained in this document is confidential. It may only be shared with a third party with the prior written consent of HERDEGEN SAS. This document may not be reproduced or communicated, in full or in part, in any way or by any means, whether technical or electronic, by photocopying, recording, or using any information or data extraction systems.

You can obtain a new copy of this document from HERDEGEN SAS by contacting your regional sales representative or directly contacting HERDEGEN SAS.

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FOREWORD

You have just purchased a HACARE product and we are pleased to have you as one of our clients. We also thank you for placing your trust in our company and our products.

Your trust assures us that the HACARE company continues to meet your specific requirements and needs. That's why we always strive to innovate, providing you with quality solutions and products that improve the comfort and quality of patient care while making the caregiver's daily tasks easier.

HACARE products have gained the confidence of many professionals and institutions due to their functional and aesthetic qualities, and are now considered to be a solid benchmark. Simplicity, reliability and safety are the hallmarks of our electrical healthcare cradles.

You've chosen HACARE, and we're committed to satisfying your needs.

However, you are responsible for the proper use of the cradles. That's why we invite you to familiarize yourself with the technical specifications, and the instructions for use and maintenance of the equipment by reading through this manual.

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EN - Please read these safety instructions carefully before use and keep them for future question.

1. SAFETY INDICATION, ABNORMAL USES, DANGERS

- Always consult with your physician or supplier to determine proper adjustment and use of the device.
- Do not lean out of the raised seat to avoid falling out of the device.
- Do not seat or step on the lid.
- Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature).
- Don't use outside but only inside and on standard toilet bowls.
- Use only as a raised toilet seat and in proper way : don't knee or step on the product, don't seat on the lid.
- Always observe the weight limit on the labeling of your device.
- No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.

- After assembling the product, make sure that it is securely locked and fixed before using.
- In case you didn't use your raised seat for a long time, let it checked by the technical staff, before using.
- Always check the stability before using.

- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (qualite@hacare.com) and the competent authority of the member state in which the user and/or the patient is established.

- The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. INTENDED USE, INDICATIONS AND CONTRAINDICATIONS

Intended use: A raised toilet seat is used to raise the height of a toilet to a comfortable level, making it easier to sit down and get up. Raised toilet seats make it easier to use the toilet independently, maintaining dignity and privacy for anyone with impaired mobility. Raised toilet seats can be used at home or in nursing homes.

Indication

- Lack of trunk control, Weakness in lower extremities
- Loss of balance, hip replacement
- Difficulty sitting and standing - Stiffness in the hips and/or spine

Contraindication

- Severe cognitive disorder, severe loss of balance, sitting impairment.
- Patient weight is greater than the product's supported weight.
- Other contraindications to be determined by the prescribing physician based on the patient's condition

3. INSTALLATION

A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again. Do not use the product if you see any risk.

- 1-Lift the toilet seat.
- 2-Push the raiser downwards on the toilet bowl until it fits.
- 3-Check that the «lips» of the raiser are covering the toilet bowl edge.
- 4-Check stability before use.
- 5-Remove simply the raiser to clean and restart from point 1 to install.

Rental: If the product has been regularly and properly maintained and after thorough cleaning and disinfection, it can be rented again. The following actions must be carried out during maintenance before new rental:

- 1- Visual assessment: make sure that the product has no defects or damage.
- 2- Evaluation of functions: ensure that the operation of the product has not been altered and repair it if necessary.
- 3- Cleaning and disinfection: this product can be cleaned and disinfected with conventional household products that do not contain chlorine.

Caution: This user's manual belongs to that product and must be delivered to each new user.

4. CLEANING

The toilet seat raiser has to be cleaned daily with a non-abrasive cleaner. It can be cleaned with products containing no chlorine, such as Surfanios (Anios laboratories), or with soapy water. Do not use any corrosive or abrasive detergents, or pressure cleaners. Sterilisation is possible at 90°C (194°F) and 80 % humidity for 5 minutes.

Caution: Do not use any abrasive cleaners or high-pressure cleaners that could damage the product.

5. DISPOSAL

Dispose of this product in accordance with your regional waste disposal regulations.

6. TECHNICAL SPECIFICATIONS

See table page 6. This product is biocompatible. The warranty duration of the product and its various elements is specified in the commercial conditions.

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