beurer

SL 60

DE	Schnarchmaske Gebrauchsanweisung	2
EN	Snore mask Instructions for use.....	13
FR	Masque anti-ronflements Mode d'emploi	24
ES	Máscara antirronquidos Instrucciones de uso	35
IT	Maschera antirussamento Istruzioni per l'uso	46

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ENGLISH

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, air and baby.


With kind regards,
Your beurer team

Getting to know your device

A restorative night's sleep is of huge importance for our physical and psychological well-being; after all, it is during sleep that we process things such as new experiences and improve our mental and physical fitness. If you snore during your sleep, however, then the quality of your sleep is impaired considerably. The beurer SL 60 snore mask was especially developed to reduce snoring and to improve sleep quality and well-being.

How does the snore mask work?

The snore mask is worn like a regular sleep mask. While it is being used, acoustic checks for snoring noises are carried out. In addition to audible sounds, snoring causes vibrations. These travel through the cranial bones and are likewise picked up and evaluated by the snore mask. Once sounds and vibrations are recognised, the control unit begins to vibrate. The intensity of this vibrating pulse increases from a low level until the desired result is achieved. The throat muscles become firm and the airways open as a reaction to the vibration. Breathing is calmed and the snoring sound decreases. You may also change your lying position. An additional effect is the training of the throat muscles. In the long term this can result in a lasting reduction in snoring.

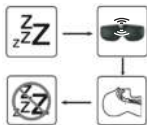The “beurer SleepQuiet” app provides the following functions:

- Detailed snoring analysis including snores per hour, recording duration and determination of the maximum snoring volume
- Calendar with snoring diary
- Evaluation view (in days/weeks/year)
- Adjustable impulse intensity and sensitivity

System requirements:

iOS ≥ 12.0, Android™ ≥ 8.0, Bluetooth® ≥ 4.0

List of supported devices:

Table of contents

1. Included in delivery	14	7. Use	18
2. Signs and symbols	15	8. Charging the battery	19
3. Intended use	15	9. Transferring data to your smart- phone	19
4. Warnings and safety notes	16	10. Cleaning and maintenance	20
5. Device description	17	11. Disposal	21
6. Initial use	17	12. Technical specifications	21
6.1 Connecting the snore mask to the “beurer SleepQuiet” app	17	13. Warranty/service	22
6.2 Putting on the snore mask	18		

1. Included in delivery

Check that the exterior of the delivery cardboard packaging is intact and make sure that all components are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1x snore mask
- 1x micro USB charging cable

2. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	WARNING Warning notice indicating a risk of injury or damage to health
	IMPORTANT Safety note indicating possible damage to the device/accessory
	Note Note on important information
	Read the instructions
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Dispose of packaging in an environmentally friendly manner
	Manufacturer
	This product satisfies the requirements of the applicable European and national directives.
	Certification symbol for products that are exported to the Russian Federation and members of the CIS
	Voltage supply

3. Intended use

WARNING

The snore mask must only be used by humans. Do not use the snore mask on animals. The snore mask is only intended for use in enclosed spaces and for domestic/private use. The snore mask is not intended for medical or commercial use. The snore mask must only be used for the purpose it was developed for and in the manner specified in these instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or incorrect use.

4. Warnings and safety notes


WARNING

- The removable control unit of the snore mask is only splash-proof.
- Keep the removable control unit of the snore mask away from water!
- Protect the snore mask from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
- Keep the snore mask and packaging material away from children. Choking hazard!
- Keep the removable control unit of the snore mask away from heat sources and do not use it near (~1 m) shortwave or microwave devices.
- Use and store the snore mask at an ambient temperature between 0°C and 60°C. Higher or lower temperatures may damage the snore mask.
- Do not use chemical cleaners or abrasive cleaning agents when cleaning the removable control unit of the snore mask. Clean the removable control unit occasionally with a 70% isopropyl alcohol pad. The snore mask itself is machine washable at 30°C (extra-delicate cycle).
- If your skin or eyes come into contact with fluid from a rechargeable battery cell, flush out the affected areas with water and seek medical assistance.
- Protect batteries from excessive heat.
- Risk of explosion! Never throw batteries into a fire.
- Do not disassemble, split or crush the batteries.
- Fully charge the battery prior to initial use.
- Fully charge the battery at least every 3 months.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.

5. Device description

- 1 Control unit
- 2 Micro USB connection
- 3 ON/OFF switch
- 4 *Bluetooth*® connection button
- 5 Operating light
 - Green = ready for use
 - Red = charge the battery
- 6 Display for *Bluetooth*®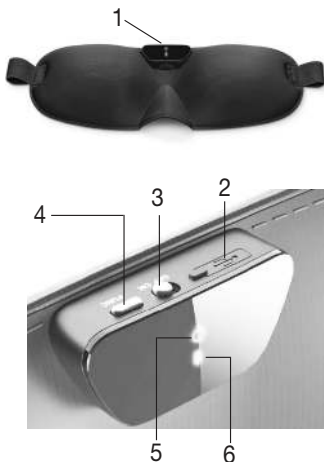

6. Initial use

6.1 Connecting the snore mask to the “beurer SleepQuiet” app

You can use the basic functions of the snore mask also without the “beurer SleepQuiet” app. The additional features such as recording, statistics, calendar and settings are only available in conjunction with the “beurer SleepQuiet” app.

To connect your snore mask with the “beurer SleepQuiet” app, please proceed as follows:

1. Ensure that your smartphone is connected to the Internet.
2. Activate *Bluetooth*® on your smartphone.
3. Download the free “beurer SleepQuiet” app from the Apple App Store (iOS) or at Google Play (Android™).

Depending on the smartphone settings, additional charges may be incurred for connecting to the Internet or transferring data.


4. Start the “beurer SleepQuiet” app and follow the instructions.

6.2 Putting on the snore mask

1. To ensure optimum wearing comfort, adjust the hook-and-loop fastener on the back of the snore mask to your desired size. Ensure that the snore mask is not too tight or too loose.
2. Put the snore mask on like a regular sleeping mask.

Ensure that the plastic panel on the back of the snore mask is touching the forehead with light pressure. This is the only way to ensure optimum snoring detection.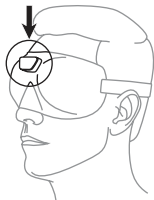

7. Use

1. Move the ON/OFF switch to “ON” to switch on the snore mask. The operating light will now illuminate green for 15 seconds. The snore mask is ready for use. Additionally, the *Bluetooth*® display will flash blue for a maximum of 15 seconds. The blue flashing indicates that the snore mask is attempting to establish a connection with your smartphone. The *Bluetooth*® display will continuously illuminate blue while data is being transferred.

If the snore mask does not switch on or the operating light illuminates red, you must charge the batteries of the snore mask. Recharge the battery as described in Chapter “6. Charging the battery”.

2. Put the snore mask on and go to bed. So as not to disturb the sleep of others, the lights on the device switch off automatically after 15 seconds.
3. As soon as you get up, remove the snore mask and switch it off. Move the ON/OFF switch to “OFF” to switch off the snore mask.

8. Charging the battery

To check the current battery capacity, press the *Bluetooth*® connection button. If the operating light is green, the battery has sufficient capacity for use for another night (more than 41%). If the operating light is red, you must first charge the battery prior to use. If the snore mask is connected to the “beurer SleepQuiet” app, you will be able to see the current battery capacity (%) in the app. If you are a heavy snorer, we recommend that you charge the battery daily.

To charge the battery, proceed as follows:

1. Open the cover of the micro USB connection on the snore mask.
2. Connect the charging cable included in delivery to the snore mask and a computer that is switched on. Alternatively, connect the charging cable to the snore mask and a USB mains adapter (not included in delivery). Insert the USB mains adapter into a suitable socket.
3. Whilst charging, the operating light flashes green. As soon as the battery is fully charged, the operating light will remain green.
4. It usually takes approx. 2 hours to fully charge the battery. As soon as the battery is completely charged, remove the charging cable from the computer or the socket and the USB mains adapter and the snore mask.
5. When the battery is fully charged, the battery life is approx. 2 to 3 days. The operating light turns red as soon as the battery capacity is below 41%.

9. Transferring data to your smartphone


To transfer the recorded data from your snore mask to the “beurer SleepQuiet” app, proceed as follows:

1. Activate *Bluetooth*® on your smartphone.
2. Open the “beurer SleepQuiet” app.
3. Slide the ON/OFF switch on the snore mask to “ON”. The display for *Bluetooth*® flashes blue. As soon as the snore mask is connected to your smartphone, the *Bluetooth*® display will continuously illuminate blue. The recorded data is now transferred to the smartphone. For a large volume of data, this may take some time. After data has been transferred, the *Bluetooth*® display will go out.

10. Cleaning and maintenance

The control unit of the snore mask is removable. To do this, use both hands to pull the control unit from the snore mask, following the direction of the arrow on the underside.

Please note that removing the control unit too frequently has a negative effect on the product. The control unit should only be removed in order to wash the snore mask.

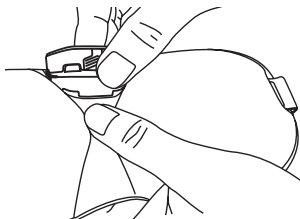IMPORTANT

- Before cleaning, always remove the control unit from the snore mask. Otherwise there is a risk of damage.

IMPORTANT

- The control unit of the snore mask must never come into contact with water or other liquids. This may cause damage.

When cleaning the control unit use a cotton pad soaked in 70% isopropyl alcohol. Do not use any chemical or abrasive cleaning agents.

IMPORTANT

- Please note that the snore mask must not be chemically cleaned, wrung out, tumble dried, put through a mangle or ironed. Otherwise the snore mask may be damaged.

The snore mask is machine washable. To do this, close the hook-and-loop fastener on the back of the snore mask and place it in a pillow case or wash bag. Set the washing machine to a particularly gentle wash cycle at 30°C (extra-delicate cycle).

For ecological reasons, always wash the snore mask together with other textiles. Use a delicate laundry detergent and measure it out according to the manufacturer's instructions.

IMPORTANT

- Please note that washing has a negative effect on the snore mask. The snore mask should therefore be washed in a washing machine a maximum of 10 times during its life. Let the snore mask dry flat on a clothes horse.

IMPORTANT

- Only reattach the control unit to the snore mask when the snore mask is fully dry. Otherwise the control unit may be damaged.

11. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life.

Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

12. Technical specifications

Dimensions (L x W x H)	Control unit: 4.3 x 1.9 x 1.2 cm Mask: 23.5 x 9.0 x 3.4 cm
Weight	48 g
Connection	Micro USB connection
Voltage supply	Input: 100-240 V~; 50/60 Hz; 0.3 A Output: 5 V  1000 mA
Battery	Capacity: 80 mAh Nominal voltage: 3.7 V Type designation: Lithium-ion
Battery life	Approx. 2–3 days (depending on use)
Memory	1 night for heavy snoring
Material	Control unit: Acrylonitrile butadiene styrene (ABS), Polycarbonate (PC), Polymethacrylate (PMMA) Mask: Textile
Data transfer	<i>Bluetooth</i> [®] low energy technology
Permissible operating conditions	0°C to 60°C, 30 to 75% RH 860 to 1060 hPa ambient pressure
Permissible transport/ storage conditions	-20°C to 60°C, <75% RH 860 to 1060 hPa ambient pressure
Frequency band	2402 to 2480 MHz
Transmission power	3.7 dBm

Subject to technical changes.

We hereby confirm that this product complies with the European RED Directive 2014/53/EU.

The CE Declaration of Conformity for this product can be found under:

<https://www.beurer.com/web/we-landingpages/de/cedclarationofconformity.php>.

13. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer. The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use. German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with
- a copy of the invoice/purchase receipt, and

- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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The *Bluetooth*® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer GmbH is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC.

Android is a trademark of Google LLC.

Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany
www.beurer.com • www.beurer-gesundheitsratgeber.com
www.beurer-healthguide.com

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