AT03003 AT3004

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage and personal injury.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

USE/INDICATIONS

A lumbar pillow can be extremely helpful in providing comfort and improving posture while sitting, especially in office work, car travel or prolonged sitting. Here are some indications for using a lumbar pillow:

- Back pain: If you suffer from back pain, especially in the lower back (lumbar), a lumbar pillow can help reduce muscle tension and relieve pain.
- Posture improvement: The lumbar pillow can support the natural lumbar curvature of the spine, which helps maintain proper posture while sitting. This improves workplace ergonomics and can reduce the risk of back pain associated with poor posture.
- support while sitting: For those who spend hours sitting at a desk or driving for long hours, a lumbar pillow can provide extra support and comfort for the lower back.
- rehabilitation after injury: For those undergoing rehabilitation after spinal injuries or surgeries, the lumbar pillow can often be recommended by therapists as a tool to help improve and maintain proper posture.
- preventing back pain: Regular use of a lumbar pillow can help prevent chronic back pain problems, especially in people with poor posture or those who spend a lot of time sitting.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

- individual needs: Not every person will need a lumbar pillow. Some people may have a different spinal structure or different support needs, so it is a good idea to consult a doctor or physical therapist before using a pillow.
- acute spinal injuries: For those suffering from acute spinal injuries, such as fractures or herniated discs, the use of a lumbar pillow may not be advisable without consulting a doctor. In some cases, other therapeutic methods may be necessary.
- excessive lumbar curvature: Some people have excessive lumbar curvature (hyperlordosis), which can be exacerbated by the use of a lumbar pillow. In such cases, an individual assessment by a physiotherapist is necessary to determine the appropriate support.
- overweight and obesity: People who are overweight or obese may need special lumbar support, but standard lumbar pillows may be inadequate or uncomfortable. In such cases, more advanced orthopedic solutions are worth considering.
- significant postural disorders: In some cases, especially when postural disorders are significant, the use of a lumbar cushion may be ineffective or even harmful. In such situations, a comprehensive evaluation by a specialist is necessary.

FEATURES

AT03003 Lumbar pillow

- Pillow, made of dense viscoelastic foam with memory effect
- a pleasant to the touch, removable cover
- Dimensions: length 42 cm Width 35 cm Height 7 - 10 cm

AT03004 Lumbar pillow

- Pillow, made of dense viscoelastic foam with memory effect
- a pleasant to the touch, removable cover
- Dimensions: length 33 cm Width 22 cm Height 7 cm

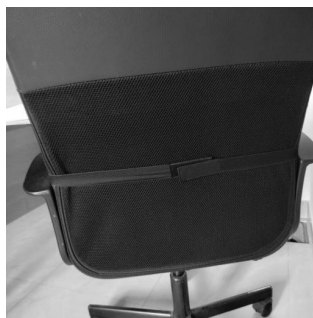
USE

- 1.Place the lumbar pillow on the chair or armchair where you want to support the sacrum of your back.
- 2.Sit on a chair or armchair in a comfortable position. Then move the lumbar pillow so that it is exactly under the lower spine area.


3. Check that the pillow provides adequate support for your spine. You can adjust the angle and position of the pillow for maximum support and comfort.

AT03003 cushions are equipped with a special elastic band, fastened with strong Velcro. With the help of this rubber, the lumbar cushion can be firmly attached to a chair or armchair.

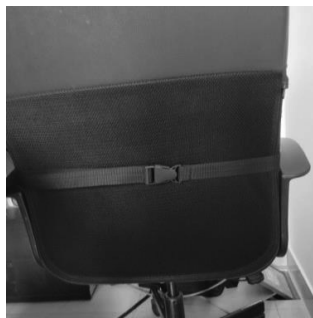AT03004 cushions are equipped with a special belt that fastens with a buckle. With the help of this belt, the lumbar pillow can be firmly attached to a chair or armchair.

NOTE: The lumbar pillow is designed for use while sitting. Remember to adjust its position regularly according to your sitting position.

NOTE: Despite the use of a lumbar cushion, it is important to change the sitting position regularly and take breaks from prolonged sitting.

COMPONENT PARTS: lumbar cushion, instruction manual.

NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person or
- (b) temporary or permanent deterioration of the health of the patient, user or other person or
- (c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTE: If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

NOTE: it is forbidden to use the product other than for its intended purpose .

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device independently or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

CLEANING AND CARE

Cleaning viscoelastic foam (also known as memory foam) requires a delicate approach to avoid damaging its structure. Here's a guide to properly clean it:

What You'll Need:

- Mild detergent or dish soap
- Warm water
- Baking soda (optional for deodorizing)
- Vacuum cleaner (preferably with a soft brush attachment)
- Spray bottle
- Soft cloth or sponge
- Enzyme cleaner (optional for stains)

Steps for Cleaning:

Vacuum the Foam:

Remove dust and debris: Use a vacuum cleaner with a soft brush attachment to gently vacuum the surface of the foam. This will remove dust, dirt, and debris before you begin any wet cleaning.

Spot Clean with Mild Detergent:

Mix detergent: Combine a few drops of mild detergent with warm water in a spray bottle.

Lightly mist the foam: Avoid soaking the foam. Lightly spray the mixture on the affected area.

Blot the stains: Use a soft cloth or sponge to gently blot the area. Do not scrub, as this could damage the foam's structure.

Rinse the area: If needed, lightly spray with clean water to rinse off any soap residue.

Dealing with Tough Stains:

For urine or biological stains: Use an enzyme cleaner. Lightly apply the enzyme cleaner to the stain, let it sit for a while (following product instructions), then blot with a clean cloth.

Oil-based stains: Use a small amount of dish soap or a specialized upholstery cleaner designed for foam materials.

Deodorizing (Optional):

Use baking soda: Sprinkle a layer of baking soda on the foam and let it sit for a few hours, or even overnight, to absorb odors.

Vacuum off the baking soda after it has sat for long enough.

Dry the Foam:

Air dry completely: Foam is very absorbent, so it can take time to dry. Leave the foam in a well-ventilated area.

Use a fan to speed up the process: Never use direct heat, such as a hair dryer, as this can damage the foam. Just a cool fan is ideal.

Ensure the foam is fully dry before using it again to prevent mold or mildew growth.

What to Avoid:

Never submerge the foam in water, as it can be difficult to dry and may cause damage.

Avoid using harsh chemicals like bleach or alcohol-based cleaners that can degrade the foam.

No direct heat: Heat can damage the structure of the memory foam, so avoid putting it in the dryer or using a hot air source to dry.

CLEANING THE COVER: unzip the cover with the zipper and remove the foam. The cover can be hand washed at a maximum temperature of 40°C. Do not iron. Do not dry clean. Do not tumble dry.

	Hand wash at up to 40°C
	Do not bleach
	Do not iron.
	Do not tumble dry
	Do not dry clean

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight. Not suitable for children under three years of age. Store away from sources of fire.

**METHOD OF DISPOSAL OF THE PRODUCT**

Once the device is decommissioned, the medical device can be disposed of as ordinary municipal waste.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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