AT03021 AT03022

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INTRODUCTION

Thank you for purchasing our product. Please read this user manual carefully before using it. Incorrect use may lead to damage to health, injury or accidents. Thus, in order to use the product safely and successfully, please read this manual carefully.

PURPOSE

The anti-decubitus cushion filled with ecological buckwheat husk is used, among other things, in the treatment and prevention of pressure sores, limb paresis or paralysis, sacral pains, haemorrhoids, multiple sclerosis and in the postnatal period.

The central hole in the cushion relieves pressure on the parts of the body on top of it, protecting sore areas from pressure.

INDICATIONS: physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

CAUTION: Inspect the product for damage caused by transport. If such damage is noticed, DO NOT use the product. Contact the manufacturer for more information.

CAUTIONS AND WARNINGS**ATTENTION**

It is forbidden to use the product other than for its intended purpose.

ATTENTION

Flammable product!

ATTENTION

Product not permitted for use by children in the age below 3 years old.

**CAUTION:**

In the event of a device-related "serious incident" that has directly or indirectly led to, could have led to, or is likely to lead to any of the following:

- (a) death of the patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above 'serious incident' must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In the event of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, a healthcare professional should be consulted.

DIMENSIONS

AT03021 - 18x18cm

AT03022 - 43x43cm

TARGET PATIENT GROUP

The healthcare professional indicates the use of the product to adults and children at his own risk, taking into account the available variants/sizes/unnecessary functions/sizes and indications, bearing in mind the information provided by the manufacturer.

CLEANING/WASHING**AT03022**

The product should be hand washed at up to 40°C. Do not use bleach, dry clean, tumble dry, iron. Dump out all the buckwheat hulls before washing; once the cover is dry, put the buckwheat hulls back in

AT03021

The product can be cleaned using a damp cloth with a small amount of mild liquid. The cover must not be allowed to get completely wet.

NOTE

Before first use, place the cushion in an oven preheated to 100°C for 10 min.

STORAGE:

Store the product in a dry place, do not expose to direct sunlight/ Store at room temperature away from children.

HOW TO DISPOSE OF THE PRODUCT AT END-OF-LIFE

Once the product has been withdrawn from service, the medical device can be disposed of as normal household waste.

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WARRANTY CARD

Model:
 Serial number:
 Point of sale stamp:

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomical. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol.
7. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
8. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
9. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
10. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
 03-068 Warsaw, ul. Zawisłńska 43
 Tel. 22 518 36 00, Fax 22 518 36 30
 www.antar.net

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