AT03501/2/3/4

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage and personal injury.

CAUTION: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

APPLICATION/INDICATIONS

The rollers are helpful for rehabilitation treatments, corrective gymnastics, and massage. The rollers facilitate patient positioning as needed.

-The rollers can be used in the rehabilitation process, such as after orthopedic surgeries or during the treatment of sports injuries, to restore muscle and joint function.

-Reduce muscle tension and increase flexibility: By applying pressure to tight muscles and connective tissue, the roller can help relax these areas, helping to improve flexibility and range of motion.

-Regular use of the roller as part of a warm-up or cool-down after a workout can reduce the risk of injury by keeping muscles in good condition and increasing blood flow to muscles.

-The roller can speed up muscle recovery after intense exercise by reducing muscle soreness and improving blood circulation, which enables faster removal of metabolites such as lactic acid.

-Regular roller exercises can help build strength in the deep stabilizing muscles, which is important for improving balance and overall body stability.

-Regular exercises with the roller can help improve proprioception, or the ability to feel the body's position, movement and balance, which is especially important in sports and daily functioning.

CONTRAINDICATIONS

Physical or mental limitations (such as visual impairment) that prevent safe handling.

-Advanced varicose veins: use of the rehabilitation roller can put undue pressure on enlarged, painful varicose veins, which can worsen the condition.

-Blood clots (venous thrombosis): Massaging areas with blood clots can be risky, as there is a risk that the clot can break off and move, which can lead to serious complications like embolism.

-Rough skin inflammation, wounds, scars: Pressure on these areas can cause pain, worsening inflammation or tissue damage.

- People with osteoporosis have an increased risk of bone fractures, and intense massaging of muscles and bones can increase this risk.

- People with diabetes, especially those with complications such as peripheral neuropathy, may have a reduced ability to feel pain, increasing the risk of serious injury when using the roller.

-Severe heart failure or other serious cardiovascular disease: Intense exercise, including the use of the roller, can be too much strain on the heart in people with serious cardiovascular problems.

USE

Use the roller according to your needs, focusing on the body parts of your choice.

**FEATURES**

- Made of high quality polyurethane foam that does not deform
- shafts are equipped with covers made of easily washable leather-like material

SIZES:

AT03501 - 50 x 10 cm

AT03502 - 50 x 15 cm

AT03503 - 60 x 10 cm

AT03504 - 60 x 15 cm

COMPONENT PARTS: rehabilitation roller, instruction manual.

CAUTION:

In the event of a device-related "serious incident" that directly or indirectly led to, could have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a health care professional.

NOTE: It is forbidden to use the product other than in accordance with its intended use.

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or as a result of failure to follow the recommendations in this manual.

TARGET PATIENT GROUP

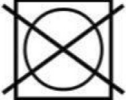
Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this product is intended (see the section on the purpose of the product of this user manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device

on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

CLEANING AND CARE

ATTENTION: DO NOT WASH THE FOAM!

To clean the product, unzip the cover with the zipper and clean the cover, according to the following instructions:

				
Wash at a temperature not exceeding 95°C	Do not dry clean	Do not bleach	Do not tumble dry	Iron at a maximum temperature of 150°C

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

Not suitable for children under three years of age. Store away from sources of fire.

HOW TO DISPOSE OF THE PRODUCT

After the device is taken out of service, the medical device can be disposed of as ordinary municipal waste.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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