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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage and personal injury.

**NOTE:** Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

#### AT04511

##### USE/INDICATIONS

The medical warming belt with cherry seeds is designed for people complaining of pain and tension in the lumbar region of the spine, it helps relieve muscle pain and tension. The belt reduces pain and leads to muscle relaxation. The belt can be used hot or cold. The product is indicated for people struggling with: sciatica, muscle pain, menstrual pain, lumbar pain rheumatic pain.

##### FEATURES

- easy adjustment, Velcro
- possible to use hot and cold
- Purified cherry seeds provide adequate massage to zones with pain and muscle tension
- pleasant material
- pockets for bags of cherry stones
- the belt is made of denim (100% cotton) in maroon color and raw cotton fabric

##### SPECIFICATION

- Thermoform dimensions: 50 cm x 20 cm
- Total length (with belt: 140 cm)
- Provides a Velcro fastener with an adjustable range of 80- 130 cm
- Universal **size**

##### COMPONENTS

Medical warming belt, User manual

#### AT04512

##### USE/INDICATIONS

The seed neck pad is designed to relieve muscle tension in the cervical region of the spine and improve blood circulation in this area. The thermoform **reduces pain** and leads to **muscle relaxation**. The product can be used hot and cold. The product is indicated for fever, bruises, migraine, pain in the neck area.

##### FEATURES

- easy to use
- possible to use hot and cold
- purified cherry seeds provide adequate massage to zones with pain and muscle tension
- pleasant material
- pockets for bags of cherry stones
- The thermophore is made of denim (100% cotton) in burgundy color and raw cotton fabric.

##### SPECIFICATION

- Dimensions: 50 cm x 18 cm
- Universal size

##### COMPONENTS

The seed neck pad, manual instruction

## USE AT04511/AT04512

### 1. USE FOR HEAT

To enjoy the warming effect of the product, place it in one of the heating devices.

a) Microwave

Place the product in a microwave with a power not exceeding 600W and heat the belt in the device for 2 minutes.

b) Oven

Place the product in the oven and heat the belt, baking it at 120 degrees C for 10 minutes. The oven should be set to up/down mode

c) Radiator/heater

Place the product on a heated radiator or heater and heat the belt for about 20 minutes.

**CAUTION:** To avoid burning, be careful when removing. Make sure the product is not too hot before use.

### 2. COLD APPLICATION

**NOTE:** Before placing in the refrigerator/freezer, the product should be placed in a plastic bag.

To enjoy the cooling effect of the product, place it in one of the cooling devices.

a) Freezer

Place the product in the freezer and pull it out after about 60 minutes

b) Fridge

Place the product in the refrigerator and take it out after about 120 minutes

### CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

Skin diseases: If there are open sores, inflamed skin, eczema, or psoriasis in the area where the belt or pad is used, heat or cold can aggravate these conditions or contribute to infection.

In people with diabetes, especially those with nerve complications (neuropathy), heat sensation may be impaired, increasing the risk of burns.

Hyposensitivity, or reduced sensation of pain and temperature, can increase the risk of burns/frostbite because a person does not feel when the heat/cold becomes too intense.

With active inflammation, such as acute arthritis, fever, or infections, the application of heat can aggravate inflammation and accelerate the development of infection.

If you are pregnant, it is a good idea to consult your doctor whether there are any contraindications to using these products.

Using a pacemaker or other electronic devices: heat/cold can affect the performance of some medical implants, including pacemakers.

### NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

(a) death of a patient, user or other person or

(b) temporary or permanent deterioration of the health of the patient, user or other person or

(c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

### NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

**NOTE: it is forbidden to use the product other than for its intended purpose .**

**NOTE:** The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.

### TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

**CLEANING AND WASHINGPECAUTION:** Do not wash the pouch with cherry seeds. The cover itself should be washed. Keep the seeds away from moisture. Wash the product by hand at a maximum temperature of 40 °C, do not bleach, do not dry clean, do not iron, do not tumble dry. Do not use strong detergents.

|                                                                                   |                         |
|-----------------------------------------------------------------------------------|-------------------------|
|  | Hand wash at up to 40°C |
|  | Do not bleach           |
|  | Do not iron.            |
|  | Do not tumble dry       |
|  | Do not dry clean        |

Product not suitable for children under 3 years old. Keep away from children

Keep away from sources of fire

### STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight. Keep the product away from children.

### METHOD OF DISPOSAL OF THE PRODUCT

Once the device is decommissioned, the medical device can be disposed of as ordinary municipal waste.

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# WARRANTY CARD

Model: .....  
Serial number: .....

Point of sale stamp  
and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawiańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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