AT51008
INSTRUKCJA UŻYTKOWANIA, ŁAWKA NAWANNOWA
USER MANUAL, BATH BENCH
BEDIENUNGSANLEITUNG, BADEBANK
NÁVOD K POUŽITÍ, LAVIČKA DO VANY
NÁVOD NA POUŽITIE, LAVIČKA DO VANE
MODE D'EMPLOI, BANC DE BAIN
GEBRUIKSAANWIJZING, BADBANK
MANUAL DE INSTRUCCIONES, BANCO DE BAÑO
ISTRUZIONI PER L'USO, PANCA DA BAGNO
BRUKSANVISNING, BADBÄNK

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EN

Thank you for purchasing our product. Please read the user manual carefully before use. Do not use the product without first reading and understanding this manual. If you do not understand the warnings, notes, and recommendations, please consult a healthcare professional or the seller to avoid product damage or bodily injury.

WARNING: Check all parts for any damage that may have occurred during transport. If such damage is found, DO NOT use the product. For more information, contact the manufacturer.

The bath bench, as an auxiliary equipment, is designed for disabled people with mobility impairments, elderly individuals, and those for whom standard use of a bathtub (sitting down and getting out) is problematic. It facilitates and assists with daily hygiene activities.

PRODUCT FEATURES:

Seat made from easy-to-clean material

Drainage holes and a space for soap

Handle to assist in standing up

Adjustable spreader legs

Durable construction

TECHNICAL PARAMETERS:

Seat dimensions: width/depth/height: 70.4 x 32 x 18 cm

Seat height: 5 cm

Seat size: 40.5 – 65 cm

Weight: 2.7 kg

Maximum load: 120 kg

INTENDED USE: The bath bench should be used by one person only. It is intended for use indoors, in a bathtub. The bench's design allows elderly individuals or those undergoing rehabilitation to use all its functions in a home environment. The bench cannot be used by individuals who are immobile.

The bench should not be used as a toy for children.

The bench must only be used in a horizontal position, on a non-slippery bathtub surface.

WARNING: Risk of injury—ensure the bench is securely attached to the bathtub using the rubber feet.

WARNING: Risk of injury—check that the bath bench fits securely onto the bathtub. Make sure it sits properly and there are no gaps or missing parts.

WARNING: Risk of injury—be cautious not to trap fingers between the bench and bathtub during installation.

INSTALLATION INSTRUCTIONS:

Place the rotating board on the edges of the bathtub.

Move the rubber feet to the appropriate position on the bathtub.

Ensure the bath bench is securely fastened to the bathtub.

REMOVING THE BATH BENCH FROM THE BATHTUB:

Unscrew the rubber feet on both sides of the bathtub.

Remove the bench from the bathtub.

TARGET PATIENT GROUP:

People suffering from diseases, dysfunctions, or injuries for which this product is intended (refer to the product's intended use section of this manual). The product can be purchased independently by the user or on the recommendation of a doctor, therapist, or other specialist. When purchasing the product, whether independently or as recommended by a healthcare professional, available sizes, necessary functions, product variants, indications, and contraindications for use should be considered, as well as the information provided by the manufacturer.

CONTRAINDICATIONS:

Coordination and balance disorders, injuries, and limb dysfunctions that prevent the use of the product.

WARNINGS:

Do not exceed the maximum weight limit for the bench (120 kg on the bench).

The bath bench should be used in a horizontal position on a non-slippery bathtub surface.

Keep away from children. The bath bench is not a toy.

If the bath bench is used by children or near them, as well as by individuals with disabilities, supervision is required.

Be cautious when installing the bath bench onto the bathtub, as there is a risk of pinching fingers during installation.

Ensure the bath bench is securely attached to the bathtub and is neither too short nor too long.

Check that the bench is securely fastened to the bathtub using the mounting clips.

Do not use the bath bench if any components of the product are damaged.

Service life: 5 years from the date of manufacture.

MAINTENANCE:

Clean with a damp cloth using mild detergents.

Remove any dirt from moving parts regularly.

Do not use abrasive cleaning agents!

Do not clean with high-pressure or steam devices!

WARNING: Do not use cleaning agents based on gasoline, solvents, acids, chlorine, etc.!

In the event of a "serious incident" related to the product, which directly or indirectly caused, could have caused, or may cause any of the following events: a) death of the patient, user, or another person, or

b) temporary or permanent deterioration of the patient's, user's, or another person's health, or

c) a serious public health threat,

the "serious incident" must be reported to the manufacturer and the competent authority of the member state where the user or patient resides. In Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

WARNING:

In the event of pain, allergic reactions, or other worrying symptoms, unclear to the user, related to the use of the medical device, consult a healthcare professional.

PRODUCT DISPOSAL AFTER USE:

After the product is withdrawn from use, the medical device can be disposed of as ordinary municipal waste, except for electrical products, which should be disposed of properly as electrical and electronic equipment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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