AT51013

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USER MANUAL ROLLATOR

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NÁVOD NA POUŽITIE - CHODÍTKO

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Thank you for purchasing our product. Before performing any operations, check the condition of the device. If the product works or looks abnormal, contact the retailer where you purchased the device immediately. Do not use the product before reading and understanding this manual. If you do not understand any warnings, cautions, or instructions, contact your healthcare professional or retailer to avoid damaging the product.

CHARACTERISTICS

The rollator is made of high-quality steel. The foldable frame makes its transport easier. The rollator is equipped with swivel front wheels and non-steering rear wheels with a brake and a parking clamp. The product is equipped with a seat, which allows you to rest in a sitting position. The rollator also has a convenient basket for the necessary things or small purchases, a stand and a tray that can be mounted on the seat.

APPLICATION

The rollator is designed to provide support and assistance to people who cannot or do not feel confident enough to walk without some support. It serves as an aid when moving.

CONTRAINDICATIONS:

Physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product

GENERAL SAFETY INFORMATION

1. Do not exceed the maximum user weight of 120 kg.
2. The rollator must not be used on traditional stairs and escalators.
3. Do not attempt to push the rollator while a person is sitting on the seat.
4. The seat must not be used to transport people or goods.
5. Do not sit on the seat when the rollator is on an inclined surface.
6. The rollator is for walking support only.
7. The rollator may only be used with original accessories and spare parts.
8. Items can only be transported in a basket. Do not hang heavy objects on the handles of the rollator, as this may impair the stability of the rollator.

GENERAL PRECAUTIONS

1. Make sure the rollator is properly unfolded before use.
2. Check the brakes before use.
3. Make sure all parts are in good condition and that moving parts are functional.
4. Improper use of the rollator may cause serious injury to the user and/or cause damage to the frame or wheels of the rollator.
5. Do not use the rollator to walk backwards.
6. Use caution when going over curbs and other obstacles, and when using the rollator on sloped, uneven, or slippery surfaces. Otherwise, serious injury to the user may result.
7. Do not make any adjustments while using the rollator.
8. Be careful if there are heavy items in the basket (max. weight 5 kg).
9. All wheels must always be in contact with the ground.
10. Apply the brakes before using the seat.
11. Apply the brakes when using the rollator in a stationary position.

Composition of the product

- steel frame
- PVC wheels
- plastic handles
- plastic seat

ATTENTION:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to, any of the following:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration of the patient's, user's or other person's health or
- (c) a serious threat to public health

this "serious incident" should be reported to the manufacturer and to the competent authority of the Member State where the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of the medical device, consult a healthcare professional.

NOTE: The manufacturer is not responsible for damage caused by neglected maintenance, improper servicing or failure to comply with the recommendations contained in this manual.

ATTENTION: It is forbidden to use the product in a way other than for its intended purpose!

GENERAL PRECAUTIONS DURING USE

Always ensure that the handles are at the correct height and securely fastened before use. To avoid the risk of falling, the rollator must not be used on uneven or inclined surfaces. Check condition of wheels and brakes. Finally, check the overall stability of the frame.

USE

The selection of the type of assistive device and its appropriate configuration must be made by qualified medical and technical personnel based on the diseases and conditions of use of the device by the user.

1. Place the rollator on the ground and unfold the frame until it is fully extended.
2. Place the handles in the holes and then adjust their height using the knob. (fig.1)
3. Install the basket, stand and trays. (fig.2,3 and 4)

1.

2.

3.

4.

5.

FOLDING

The rollator can be folded by following all the steps described above in reverse order. To fold the frame, press the red button. (fig.5)

BRAKES

6.

7.

1. To slow down or stop the rollator, pull the brake lever. Release the brake lever to move again (fig. 6).
2. To activate the parking brake, push the brake lever down until it locks (fig. 7).

SPECIFICATION

total width: 62 cm

width between handles: 48 cm

seat thickness: 2 cm

seat width: 38 cm

seat depth: 18 cm

seat height: 61 cm

handle height adjustment range: 80-92 cm

weight: 10.85 kg

user weight: 120 kg

This mark indicates the user's maximum weight!

INCLUDED

Rollator, basket, stand, tray, instruction manual

STORAGE AND DISPOSAL

Keep the rollator away from sources of light, high temperature and moisture. Avoid exposing the rollator to direct sunlight. Dispose of the rollator in accordance with applicable environmental protection and recycling regulations.

MAINTENANCE AND CLEANING

After each use, remove accumulated dirt using a mild cleaning agent, preferably a wet cloth with a mild soap, then wipe the rollator dry. Do not use sharp abrasive sponges. Periodically check the operation of the brakes, the condition of the seals of the fasteners and the condition of the welds.

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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