## **AT51018 (CA8882L)**

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

**CAUTION:** Inspect all parts for damage caused in transit. If such damage is observed, DO NOT use the product. Contact the manufacturer for more information.

### PRODUCT DESCRIPTION

The rollator, equipped with two solid rear wheels and two front swivel wheels and a brake with locking device. The rollator allows easy movement on any surface, indoors and outdoors. The product is equipped with a soft PVC-covered seat, folding footrests, double-sided handles. The frame is covered with a liquid aluminum coating.

### INDICATION

The product is designed to provide support during mobility for people with limited walking ability. The rehabilitation rollator is designed for one person. The product will be used for:

- Balance training: The rehabilitation rollator allows patients to practice balance maintenance, which is important in the rehabilitation process after injuries or neurological disorders.
- Posture maintenance: Patients can use the rehabilitation rollator to train correct posture, which is important especially for people with postural defects or problems maintaining body stability.
- Improving motor coordination: Exercises performed with a rehabilitation rollator can help improve motor coordination and control of movements, which is important for people with neurological disorders.
- Supporting rehabilitation after injury: A rollator can be used in the rehabilitation process after lower limb injuries, helping patients gradually regain mobility.
- Increasing independence: For some patients, a rollator can be a tool to increase independence in daily life, enabling independent movement.

### CONTRAINDICATIONS

Physical or mental limitations (such as visual impairment) that prevent safe handling.

- Lack of spinal stability: Individuals with severe spinal stability problems or spinal injuries may not be suitable candidates for using a rehabilitation rollator, as this may increase the risk of further injury.
- Uncontrolled seizures: In people with uncontrolled seizures or epilepsy, the use of a rehabilitation rollator can be dangerous for both the patient and those around them.
- Poor muscle strength: In people with very limited muscle strength, using a rehabilitation rollator may be difficult or ineffective. In such cases, a different approach to rehabilitation may be necessary, such as manual therapy or Muscle Strength Support Exercises.
- Balance problems: In patients with severe balance problems, the use of a rollator can be dangerous, as it can increase the risk of falls and further injury.

### PREPARATION FOR USE AND WARNINGS

Remove the product carefully from the package. Be careful not to cut or damage the product. The package contains the rollator, mounting screws, basket, and handles to assist in standing up/sitting down.

After removing the rollator from the package, unfold the rollator by pulling the frame of the rollator apart from side to side.

Then unfold the seat.


Install the main handles(with brakes). To mount the main handles, insert them into the dedicated hole.

Then choose the height that suits you.(Both handles must be set at the same height).

Screw the handles in place using the star knob screw.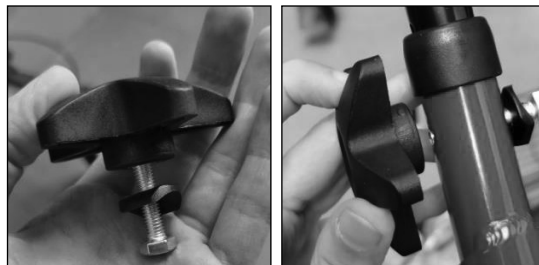

#### INSTALLATION OF SIT/STAND UP ASSIST HANDLES


To install the handles, insert them into the dedicated holes, and then screw the handles with the screws.

## BASKET

The basket can be mounted in two ways,

**First:** attaching the basket with hooks to the frame at the footrest.

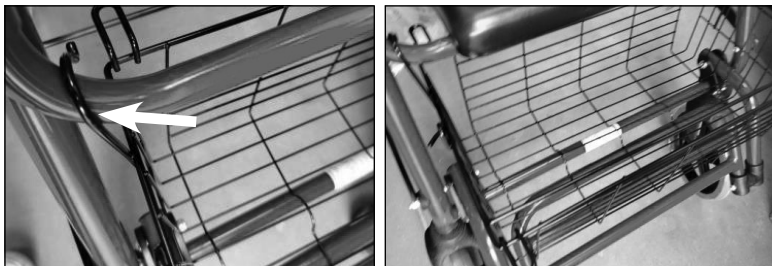
**Second:** attaching the basket under the seat.


## FOOTREST

Unfold the footrest by pulling it towards you.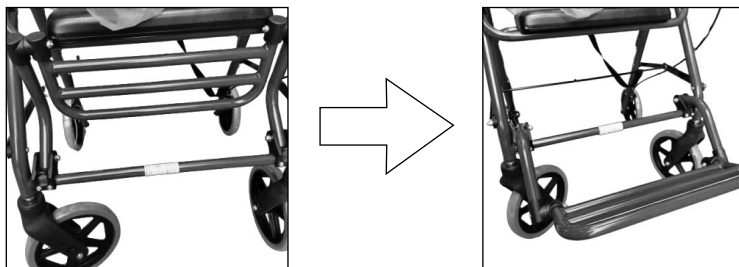

## FOLDING THE ROLLATOR

To fold the rollator, pull the cord located under the seat.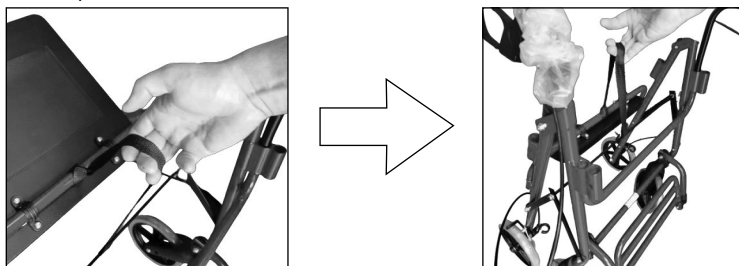

### OPERATING THE BRAKES


**NOTE:** To safely sit down and stand up from the rollator, the parking brakes must be applied.

1. to stop, gently press both brakes at the same time, which will slow and stop the rollator.


2. the rollator's brakes have a parking option. To activate the parking option, press both brakes down simultaneously. The brakes are now locked.

3. to unlock, simply pull the brake up again.

Main brake (when sliding): Please pull the brake rod upward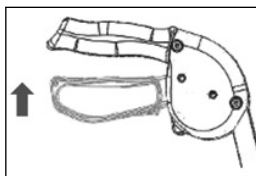

Parking brake (when parking): Please push the brake rod downwards

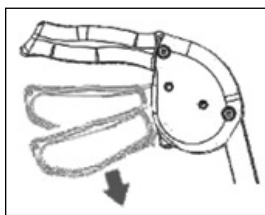**NOTE:** The rollator is designed for use on level surfaces. Using the rollator on slippery, wet, icy surfaces risks falling.

**ATTENTION:** The product must not be used after damage has been found

#### CAUTION

During the use and operation of the rollator, as well as during the assembly and adjustment of its mechanisms, there may be a danger of entrapment and/or compression of the user's/companion's body parts in the openings/gaps between the components.

Perform these operations with particular care. Once the adjustment is complete, stabilize the position by carefully tightening the nuts/bolts.

#### CAUTION

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

#### CAUTION

In the event of a device-related "serious incident" that has directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of the condition of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health, the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products

**ATTENTION:** The manufacturer shall not be liable for damage caused by negligent maintenance, inadequate servicing, or resulting from failure to observe the recommendations contained in this instruction manual.

**CAUTION:** It is forbidden to use the product other than in accordance with its intended use!

**CAUTION:** If the product is used incorrectly, there may be a risk of tipping over. Please follow the instructions for using the product. After adjustment, stabilize the position by tightening the nuts/bolts carefully.

**CAUTION:** It is forbidden to stand on the footrests of the walker. This may cause the walker and the user to tip over, resulting in severe injuries.

#### **MAINTENANCE OF THE PRODUCT**

The product should be cleaned periodically with water and cleaning agents. The product can be treated with disinfectants. The seat should be cleaned using mild detergents. When not in use, the product should be stored in a dry room. Avoid prolonged exposure to direct sunlight.

In order to ensure the long-term and safe operation of the product, it is recommended to regularly inspect all screws and mounting components. Before each use of the product, check that all screws are properly tightened and there are no signs of loosening.

Regularly check that the product is not missing any components and that all components are in good working condition. If you notice wear, mechanical damage or missing parts, immediately stop using the product and contact the manufacturer's service department to avoid the risk of an accident or further damage.

Pay attention to any unusual sounds or vibrations that may indicate loose parts or other technical problems. Remember to use only original parts recommended by the manufacturer when replacing screws or components.

#### **DISPOSAL OF USED PRODUCT**

After withdrawal from use, the product may be treated as municipal waste. It is recommended to segregate during disposal after the product is withdrawn from use.

#### **TECHNICAL PARAMETERS**

Height: 78 - 89 cm

Wheels with a diameter of 8"

Seat height: 53 cm

Seat width: 44 cm

Total width (with wheels): 61 cm

Maximum user weight: 120 kg.

Dimensions (length\*width\*height): 93\*60\*89

Weight: 8kg

THIS SIGN INDICATES THE MAXIMUM  
WEIGHT OF THE USER

#### **TARGET PATIENT GROUP**

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device independently or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

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## NOTATKI

A series of horizontal dotted lines for taking notes.

IU\_AT51018

# WARRANTY CARD

Model: .....  
Serial number: .....

Point of sale stamp  
and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawisłńska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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