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ENG

DESCRIPTION AND PURPOSE OF THE AT51020

The bedside table is designed for elderly and sick people lying in bed. A bedside table on wheels is an excellent way of improving care for an elderly, sick or disabled person. Thanks to its special design, it can easily be moved closer to the bed so that the patient can eat independently, use a computer or carry out simple manual tasks.

It has a sturdy construction and is adjustable in height. The frame of the table is made of powder-coated steel, while the 54x40 cm top is made of durable and aesthetically pleasing wood.

It is used for eating and additionally enables activities such as writing or reading.

DESCRIPTION AND PURPOSE OF THE AT51021

The bedside table with swivel top is designed for elderly and sick people lying in bed. The wheeled bedside table is an excellent way to improve care for an elderly, sick or disabled person. Thanks to its special design, it can easily be moved closer to the bed so that the patient can eat independently, use a computer or carry out simple manual tasks.

It has a sturdy construction and is adjustable in height. The table top consists of two parts: fixed and rotatable, so it can be customised. The frame of the table is made of powder-coated steel, while the 60x38 cm top is made of durable and aesthetically pleasing wood.

It is used for dining and additionally enables activities such as writing or reading. In addition, the product has additional slats that prevent meals and objects from sliding off. The wheeled bedside table is an excellent way to improve care for an elderly, sick or disabled person. Thanks to its special design, it can easily be moved closer to the bed so that the patient can eat independently, use a computer or carry out simple manual tasks.

It is characterised by its very simple installation.

COMMENTS AND WARNINGS

Before using the product, it is mandatory to read the contents of this manual

Do not exceed the maximum load on the table top.

The table is designed for use on flat and level surfaces

Standing on the table is not permitted.

Check the equipment regularly for damage and wear. In the event of any damage, stop using the product immediately.

Repair and disinfection should only be carried out by authorised persons!

Pay attention to the proper condition of the castors (damaged or missing castors pose a danger when using the table).

Before each use, check that all screws are properly tightened (otherwise the table may fold).

During use and handling, and when folding and adjusting the mechanisms, there may be a danger that parts of the user's/companion's body become trapped and/or pinched in the openings/gaps between the components.

PARAMETERS OF THE AT51020:

Product weight: 7.1 kg

Overall length of tabletop: 54 cm

Width of tabletop: 40 cm

Minimum height: 75 cm

Maximum height: 118 cm

Permissible table top load: 15 kg

Distance between the legs: 40 cm

It is equipped with 2 castors and 2 feet for stability.

PARAMETERS AT51021:

Product weight: 9.2 kg.

Overall length of top: 82 cm.

Length of tabletop swivel section: 60 cm.

Tabletop width: 38 cm.

Minimum height: 75 cm.

Maximum height: 118 cm.

Permissible table top load: 15 kg.

It is equipped with four castors, two of which have a brake.

THIS SIGN INDICATES THE MAXIMUM LOAD OF THE TABLETOP

INSTRUCTIONS FOR USE


1. the two silver tubes should be placed in the special holders on the underside of the tabletop so that the copper pins go into the correct places in the holders. (fig.1 and 2)

2. attach the two black table legs to the silver tubes using the knobs (you can adjust the height of the top using the knobs at a later stage of assembly) (fig3)

3. the smallest part of the C-shaped frame should be attached to the legs of the table using locking elements, placing them from above, in addition stabilising feet can be attached. (fig.7)

3. then attach the circles to the larger black C-shaped piece.

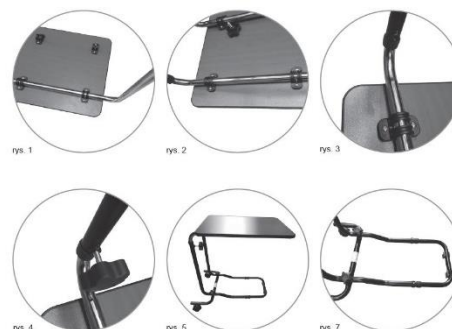4. Finally, the piece with the castors is attached to the table legs using the knobs. (fig.5)

MAINTENANCE AND CLEANING OF THE PRODUCT

Clean the frame and table with water and cleaning agents (avoid corrosive substances, use only mild detergents).
Rough objects and strong preparations must not be used.

STORAGE

Store away from direct sunlight, heat source, in a dry and cool room.

NOTES:

In the event of a product-related 'serious incident' that has directly or indirectly led to, could have led to or is likely to lead to any of the following:

- (a) the death of a patient, user or other person; or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

this 'serious incident' should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions or other distressing, unclear symptoms related to the use of the medical device, consult a healthcare professional.

ATTENTION: it is forbidden to use the product other than for its intended purpose

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/sizes/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

CONTRAINDICATIONS

physical or mental limitations (e.g. visual impairment) that prevent safe handling.

HOW TO DISPOSE OF THE PRODUCT

The medical device can be disposed of as normal municipal waste.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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Antar Medizin GmbH