AT51031

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INSTRUCTION MANUAL ROLLATOR

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v.1_1.08.07.2020

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The outdoor rollator is equipped with a parking brake and a seat with upholstered back. The handle height is adjustable and the rollator can be folded for transport or storage. Before folding or using the rollator for the first time, read the instruction manual carefully and keep it in a safe place. DO NOT fold or use the rollator before reading this instruction manual carefully.

DETERMINATION

The device serves as a walking, supporting and carrying aid for carrying small objects both indoors and outdoors. It is intended for use by users with limited mobility and their maximum body weight as specified below. To ensure the safety of the user of the rollator, the user should have appropriate physical, visual and cognitive abilities. In case of doubt, contact the seller / specialist who prescribed the use of the rollator. It is very important that the user carefully read the safety instructions in this manual before using the device for the first time. The instructions given here are intended to prevent possible bodily injury to the user or third parties, as well as damage to the rollator or objects placed next to it.

NOTES:

Do not stand on the individual parts of the frame. The frame of the device is not intended for such purpose.

SECURITY

The rollator can be used by a person with a weight of max. 135 kg.

IT SHOULD BE NOTED

- (1) Make sure that the brake of a rollator left unattended is always locked/engaged.
- (2) Before use, ensure that all wheels, including the moving wheels, are securely are securely fastened.
- (3) Make sure the brakes are working properly.
- (4) Before first use, check the handle height and its proper attachment.
- (5) Make sure the rollator is fully unfolded.
- (6) When unfolding the rollator, the frame should be placed in the correct position.
- (7) To avoid pinching fingers, keep them away from the folding mechanism away.
- (8) In order to adjust the rollator well to your needs, seek advice from qualified personnel.
advice.
- (9) Be careful when lifting or carrying the rollator.

IT IS FORBIDDEN:

- (1) Use the rollator to drive people - the device was not designed as a wheelchair.
- (2) Sit on the rollator without the brake locked - this can set the rollator in motion and cause bodily and it may cause bodily injury to the user or third parties.
- (3) Sitting on the rollator while tilted, without the parking brake engaged.
- (4) To sit on the seat with momentum - sit carefully and in a controlled manner, if Ask for assistance if needed.
- (5) Do not hang any objects on the brake levers or handles, this can impair the functioning of the brakes and/or the stability. Of the brakes and/or the stability of the rollator.
- (6) Standing on the rollator or using it as a footrest.

ASSEMBLY Instructions:

Make sure that the rollator has not been damaged during transport. If there is visible damage, do not use the rollator. In such case, contact the seller to have the rollator repaired or to return it.

- A. Backrest
- B. Brake lever
- C. Handles with height control
- D. Seat
- E. Bag
- F. Rear wheels diameter 30 cm
- G. Front wheels diameter 30 cm

1. Remove the packing. The rollator is folded.

2. Unfold the rollator, push the seat down on both sides - Fig. 4.

NOTE: Both sides must be locked in an appropriate position with the audible "click".

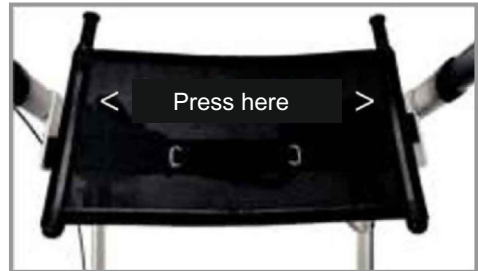
Abb. 4

3. Hang up the bag.

HEIGHTSETTING:

Appropriate adaptation:

Before using the rollator for the first time, make sure that it is fully folded and that the frame is properly locked and stiff. It is important to adjust the height of the rollator correctly to a comfortable position.

1. For this purpose, leave your hands hanging on the sides and adjust the handles of the rollators at the height of the wrists.
2. When gripping the handles, the elbows should be bent at an angle of 150-20°.
3. Are you forced to tilt or raise your arms to reach the rollator, correct the height.
4. To do this, turn the wing shudder out and adjust the handle at the correct height. Repeat the activity on the other side of the rollator and check before use that both handles are at the same height and properly locked.

HINT: To determine the correct height of the rollator, contact the doctor or retailer.

SETTING AND OPERATING THE BRAKES

Service brake (when sliding): Pull the brake levers upwards.

Parking brake (when parking): Push the brake levers down. (see Fig. 5).

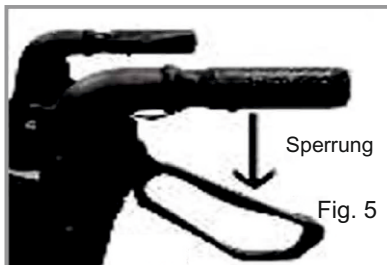
Fig. 6

Brake setting: To increase the voltage of the brake rope, turn the control screw counterclockwise; then - to maintain such a setting - turn the fastening button clockwise and hold the control screw (see Fig. 6).

NOTE: In case of any uncertainties, please contact Customer Service. Changes in the brake settings can lead to reduced braking force.

FOLDING AND STORAGE OF THE ROLLATOR

1. Pull the strap up in the middle part of the seat to fold the rollator - Fig. 7.

MAINTENANCE

1. Make sure that ALL equipment parts are always turned correctly.
2. Check that the screws, buttons and knobs of the wheels and the moving wheels are screwed in. Check that the brake is working properly and that the handles are properly secured.
3. Used or damaged parts must be replaced immediately.

CLEANING

To clean the rollator, use a damp cloth or sponge and warm water with soap. If necessary, common disinfectants can also be used. Make sure that there is no water or detergent residue soaked on the cleaned surfaces and that they are not smooth. Do not use rough detergents or scratching devices - they can damage the paint layer.

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