**Head washing tub
for the disabled**

AT51033

Notes and Warnings

BEFORE USING THE PRODUCT, IT IS MANDATORY TO READ THIS MANUAL.
KEEP AWAY FROM DIRECT SUNLIGHT AND HEAT.
CHECK EQUIPMENT REGULARLY FOR DAMAGE AND WEAR.
IN THE EVENT OF ANY DAMAGE, STOP USING THE PRODUCT IMMEDIATELY.

NOTE: Inspect all parts for damage caused in transit. If any such damage is found, DO NOT use the product. More information from the manufacturer.

GENERAL WARNING

Do NOT use the product before reading and understanding this manual. If you do not understand any warnings, cautions, or instructions, consult your healthcare professional, dealer, or service technician before attempting to assemble the product to avoid personal injury or damage to the product.

Application

The bathtub is intended for washing the hair of people with disabilities or reduced mobility.

Assembly / use

1. Inflate the bathtub. The valve is non-return and does not empty even when left open. To deflate it, press down on the valve and side walls until all air deflates.
2. Place the tray on a flat surface.
3. Place the drain hose in a bowl or sink, make sure it is lower than the tub to allow the water to flow.
4. Open the drain valve when using the drain pan.
5. Fill the water bag, the water flow in the hose is adjusted by unscrewing / screwing the tip.
6. Place the bathtub under the user's neck and adjust it until it feels comfortable.

Product features

Tub, 21 cm deep, double reinforced for greater durability and comfort. Includes drain valve and hose for easy and quick removal of water. In addition, in the absence of access to a water source, the product has a water bag with a maximum capacity of 10 liters.

The product includes: a bathtub with a hose, a pump, a water bag with a shower.

Material: PVC

Cleaning

Do not use sharp, abrasive objects or aggressive cleaning agents (e.g. acids, solvents) for cleaning. All parts are best cleaned with a damp cloth and mild household cleaners. Do not dry on a radiator or in direct sunlight.

Security

Use only warm water up to 38°C to avoid scalding!
Do not leave the patient unattended in the (filled tray).
Check the water-tightness of the tub regularly to avoid water damage.
The bathtub must not be used for purposes other than its intended purpose.

Dimensions

Length: 60cm
Width 50cm
Height: 20cm
Headrest height: 10cm
Drain hose length: 70cm

NOTE: Any activities while using the product require the assistance of the caregiver.

WARNING:

In the event of a device-related "serious incident", which directly or indirectly led to, could have led to, or is likely to lead to any of the following events:

- a) death of a patient, user or another person, or
- b) temporary or permanent deterioration of the health of a patient, user or another person, or
- c) a serious risk to public health, such a "serious incident" must be notified to the manufacturer and to the competent authority of the Member State, in which the user or patient resides. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

WARNING:

In the event of pain, allergic reactions or other alarming, unclear symptoms related to the use of the medical device, please consult a healthcare professional.

METHOD OF DISPOSAL OF THE PRODUCT AFTER REMOVING FROM SERVICE

After the device is withdrawn from use, the medical device can be disposed of as normal household waste, except for electrical products - proceed in the manner appropriate for the disposal of electrical and electronic equipment.

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent the safe handling of the product.

WARRANTY CARD

Model:

Serial number:

Point of sale stamp

and a legible signature of the Seller:

1) ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślńska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.

2) During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.

3) ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.

4) If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.

5) Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.

6) The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.

7) The product delivered to the service in connection with the complaint cannot cause an epidemic.

8) THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.

9) Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw
Zawiślńska 43
Poland
Tel.: +48 22 518 36 00
Fax: +48 22 518 36 30
www.antar.net

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Material: PVC

Cleaning

Do not use sharp, abrasive objects or aggressive cleaning agents (e.g. acids, solvents) for cleaning. All parts are best cleaned with a damp cloth and mild household cleaners. Do not dry on a radiator or in direct sunlight.

Security

Use only warm water up to 38°C to avoid scalding!

Do not leave the patient unattended in the (filled tray).

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