AT51037

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USER MANUAL ROLLATOR

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Thank you for purchasing our product. Before using, please read this user manual carefully. Incorrect use of the product may lead to health detriment, injuries or accidents. Therefore, in order to use the product safely and successfully, please read this manual carefully. Inspect all parts for damage in transport. If any such damage is found, **DO NOT** use the product. More information from the manufacturer.

GENERAL WARNING

DO NOT assemble or use the product before reading and understanding these instructions. If you do not understand the warnings, cautions or instructions, contact a healthcare professional, distributor or technician before attempting to assemble the product to avoid injury or damage to the product. The rollator is not designed to be steered by the user while sitting on it. When adjusting the height of the rollator, a doctor/therapist should be present to ensure that the user is supported and that the brakes are properly set and the screws are tightened. When the rollator is unfolded, always make sure that the tube under the seat is locked and properly connected to the frame, and that the brake cable is inside the side tube and tightly attached to it. Check the brake cable after each height adjustment to avoid loosening the cable. Make sure that the knobs are locked, in such a way that the main support structure will not move due to pressure.

ATTENTION:

When a product-related "serious incident" occurs, which directly or indirectly led, or could lead to, any of the following events:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of a medical device, consult a healthcare professional.

ATTENTION: It is forbidden to use the product other than in accordance with its intended purpose

INTENDED USE: The product is intended to support the movement of people with reduced walking ability. The rollator is designed for one person

CONTRAINDICATIONS: Physical or mental limitations (e.g. visual impairment) that prevent the safe handling of the rollator.

UNFOLDING

1. Extend the handles of the rollator outwards, press down on both sides of the seat so that the hooks are attached to the retaining bars, when you hear a click this means that they have been correctly attached.
2. The height of the handles and the backrest can be adjusted according to the needs of use by means of buttons and dials.
3. The shopping bag is made of opaque waterproof material, with a zip at the top. The shopping bag is detachable for easy carrying and cleaning.
4. It has strong forks of the front wheels, the rear wheels are equipped with effective and safe plastic brakes (including aluminium blocks), equipped with two pedals to help you overcome bumps and obstacles.
5. Lifting the plastic handle in the center of the seat may fold the product!

INCLUDES: rollator, bag, manual

THE SIGN MEANS THE MAXIMUM USER WEIGHT!

ADJUSTING THE HEIGHT OF THE HANDLES

Press the latch on the adjustment tube and slide it up or down to adjust the height. Make sure that the latch is in the correct hole after adjusting.

FOLDING

Raise the handle of the seat and press the armrests inwards to fold the product.

CAUTION: When using and handling the product, and when assembling and adjusting mechanisms, there may be a risk of entrapment and / or pinching of body parts of the user / accompanying person in openings / gaps between the components. These steps should be performed with extreme caution. After completing the adjustment, stabilize the position by tightening the nuts / bolts carefully.

USE OF HANDBRAKES

- A. WHEN STANDING - keep the brake handle up:
- B. WHEN MOVING the rollator - release the brake handle

LOCKING THE HAND BRAKES. UNLOCKING THE HAND BRAKES

1. Push down on the bottom of the brake handle as shown until you hear a click.
2. Refer to the UNLOCKING THE HANDBRAKES section in this manual for more information

UNLOCKING THE HAND BRAKES

Pull the brake handles upwards until released. Release the brake handle. Make sure the brake cable runs along the sidewall and the unused part is coiled in front of the rollator.

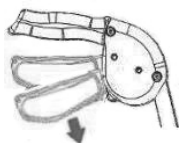
Fig. 1 Lock

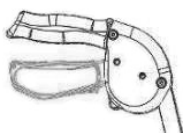
Fig. 2 Braking

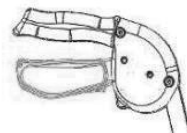
Fig. 3 Free movement

TECHNICAL SPECIFICATIONS

Foldable, four-wheeled aluminium rollator.

Weight: 7,4kg

Overall height: 81-93cm

Overall length: 74cm

Overall width: 65cm

Folded width: 25.5 cm

Wheel diameter: front 10 ", rear 8"

Seat height: 55cm

Seat depth: 23cm

Seat width: 46 cm.

Maximum user weight: 136 kg

CLEANING AND STORAGE: clean the rollator regularly with lukewarm soapy water. Do not use strong detergents, do not use rough objects for cleaning. Store in a dry and cool room away from direct sunlight.

MAINTENANCE

1. Please ensure that all attachments are always secured.
2. Check the bolt securing the seat to the side elements. Tighten as needed.
3. Replace any damaged or damaged parts immediately.

WHEEL MAINTENANCE

1. Wheels should be cleaned with warm water and a mild detergent. Pat dry with a dry cloth.
2. Lubricate periodically with 3-in-1 grease if the wheels begin to squeak.

UTILIZATION Municipal waste should be disposed of in accordance with the applicable disposal conditions.

TARGET PATIENT GROUP

The healthcare professional shall indicate the use of the device to adults and children on his/her own responsibility, taking into account the available variants/sizes/necessary functions/sizes and indications, bearing in mind the information provided by the manufacturer.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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