AT51038

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ROLLATOR USER MANUAL

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

CAUTION: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Consult the manufacturer for more information.

WARNINGS

When adjusting the height of the rollator, a doctor, therapist, specialist or dealer should be present to provide support to the user and to check that all screws are properly tightened.

When the rollator is unfolded, always make sure the tube under the seat is locked and properly connected to the frame.

Make sure the height adjustment knobs are locked so that the main support structure does not move under pressure. Failure to follow the above instructions may lead to injury.

Before use, check that ALL handle and seat adjustments have been secured and that the wheels and moving parts of the product are in good condition.

During use, ALL wheels must BE in contact with the ground AT ALL TIMES to ensure the stability of the rollator.

Failure to comply with the above recommendations may lead to tipping of the rollator and injury to the user.

The rollator is not designed for the user to steer it by himself while sitting on it.

CAUTION: During use and operation of the rollator, as well as when folding and adjusting its mechanisms, there may be a danger of entrapment and/or compression of the user's/companion's body parts in the openings/gaps between the components. Perform these operations with particular care. Once the adjustment is complete, stabilize the position by carefully tightening the nuts/bolts.

CAUTION:

In the event of a device-related "serious incident" that has directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient's, user's or other person's condition, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTE: In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

ASSEMBLING AND ASSEMBLY WARNINGS

After assembling the rollator, make sure the seat is locked and the two handle tubes - in the main support frame attached with nuts. Also make sure that both handles are at the same height and do not move.

INSTALLATION WARNINGS

Make sure the seat tube is securely fastened in the locking mechanism and the copper pins slide out of their respective holes. BEFORE use, always check that the walker and its components are securely locked in their proper positions.

NOTE: The manufacturer is not responsible for damage caused by negligent maintenance, inadequate servicing or resulting from failure to follow the recommendations in this manual.

CAUTION: It is forbidden to use the product other than in accordance with its intended use.

CAUTION: If the product is used incorrectly, there may be a risk of tipping over. Please follow the recommendations for getting on/sitting down/moving.

INDICATIONS

The product is designed to provide mobility support for people with limited walking ability. The rollator is designed for one person. The product will be used for:

- Balance training: The rollator allows patients to practice balance maintenance, which is important in the rehabilitation process after injuries or neurological disorders.
- Posture maintenance: Patients can use the rollator to train correct posture, which is important especially for people with postural defects or problems maintaining body stability.
- Improving motor coordination: Exercises performed with a rehabilitation rollator can help improve motor coordination and control of movements, which is important for people with neurological disorders.
- Supporting rehabilitation after injury: A rollator can be used in the rehabilitation process after lower limb injuries, helping patients gradually regain mobility.
- Increasing independence: For some patients, a rollator can be a tool to increase independence in daily life, enabling independent movement.

TARGET PATIENT GROUP

Individuals struggling with diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see the intended use section of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, it is important to consider the available sizes/necessary functions and variants of the device, the indications and contraindications for use, and the information provided by the manufacturer.

CONTRINDICATIONS: physical or mental limitations (e.g., visual impairment) that prevent safe handling of the walker.

- Lack of spinal stability: People with severe spinal stability problems or spinal injuries may not be suitable candidates for using a rollator, as this may increase the risk of further injury.
- Uncontrolled seizures: In people with uncontrolled seizures or epilepsy, the use of a rollator can be dangerous for both the patient and those around them.
- Poor muscle strength: In people with very limited muscle strength, using a rollator may be difficult or ineffective. In such cases, a different approach to rehabilitation may be needed, such as manual therapy or Muscle Strength Support Exercises.
- Balance problems: In patients with severe balance problems, the use of a rollator can be dangerous, as it can increase the risk of falls and further injury.
- Rollator mismatch: Improperly fitting a rollator to a patient can lead to additional injuries or worsening of the condition.

PRODUCT FEATURES:

- lightweight, durable folding aluminum construction
- one-hand folding / unfolding for easy transport and storage
- large cast swivel front wheels with sturdy fork for easy off-road mobility
- reflectors on the frame in the front for safety
- comfortable backrest, seat and large shopping bag
- non-slip comfortable handles
- in addition to the brake in the handles, the walker also has parking brakes
- 4-stage height adjustment capability

UNFOLDING

1. After unpacking the product, check the rollator for any damage. (Fig.1)
2. Position the rollator and unfold it, pointing the handles outward. (Fig.2)
3. Press down on the seat until the latches lock into place. (Fig.3)
4. Point the brake downward to lock the walker. (Fig.4)
5. Adjust the height of the backrest and then tighten the screws accordingly. (Fig.5)
6. Adjust the height of the handles and then tighten the screws accordingly. (Fig.6)
7. The height of the handles can be adjusted between 84cm and 94cm. (Fig.7)

Fig.1

Fig.2

Fig.3

Fig.4

Fig.5

Fig.6

Fig.7

Fig.8

FOLDING

1. To fold the rollator, lift the strap on the seat. (Fig.8)

TECHNICAL PARAMETERS

Maximum width: 80 cm
 Length: 77 cm
 Height adjustment: 84-94 cm (4 steps every 2.5 cm)
 Folded thickness: 19 cm
 Width between handles: 42 cm
 Seat width: 42 cm
 Seat height: 50 cm
 Diameter of the front swivel wheel: 25 cm
 Diameter of the rear non-steering wheel: 20 cm
 Weight: 8.4 kg
 Maximum load: 136 kg

THIS SIGN INDICATES THE MAXIMUM
WEIGHT OF THE USER

INCLUDED: ROLLATOR, BAG, USER MANUAL

STORAGE:

The product should be stored in a dry place, not exposed to direct sunlight.

MAINTENANCE

1. Ensure that all attachments are always secured.
2. Check all fixing screws, tighten if necessary.
3. Promptly replace all damaged or damaged components.

In order to ensure the long-term and safe operation of the product, it is recommended to regularly inspect all screws and mounting components. Before each use of the product, check that all screws are properly tightened and there are no signs of loosening.

Regularly check that the product is not missing any components and that all components are in good working condition. If you notice wear, mechanical damage or missing parts, immediately stop using the product and contact the manufacturer's service department to avoid the risk of an accident or further damage.

Pay attention to any unusual sounds or vibrations that may indicate loose parts or other technical problems.

Remember to use only original parts recommended by the manufacturer when replacing bolts or components.

Clean the wheels with warm water and a mild detergent. Dry with a dry cloth.

DISPOSAL

Dispose of municipal waste in accordance with the applicable disposal conditions. For its proper disposal, please contact your local authority or the nearest waste collection point to find out how to properly dispose of the product. By acting in accordance with local regulations, you are helping to protect the environment.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłńska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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