AT51043

INSTRUKCJA OBSŁUGI, CHODZIK
INSTRUCTION MANUAL, ROLLATOR
GEBRAUCHSANWEISUNG, ROLLATOR
NÁVOD K POUŽITÍ, CHODÍTKO
NÁVOD NA POUŽITIE, CHODÍTKO
MANUEL D'INSTRUCTION, DÉAMBULATEUR
GEBRUIKSAANWIJZING, ROLLATOR
MANUAL DE INSTRUCCIONES, ANDADOR
MANUALE DI ISTRUZIONI, DEAMBULATORE
BRUKSANVISNING, RULLATOR

IU_ AT51043

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

USE/INDICATIONS

The product is designed to support the movement of people with limited walking ability. The rehabilitation rollator is designed for one person. The product will find application in:

- Balance Training: The rehabilitation rollator allows patients to practice maintaining balance, which is important in the rehabilitation process after injuries or neurological disorders.
- Posture maintenance: Patients can use a rehabilitation rollator to train correct posture, which is important especially for people with postural defects or problems maintaining body stability.
- Improving motor coordination: Exercises performed with a rehabilitation rollator can help improve motor coordination and movement control, which is important for people with neurological disorders.
- Supporting rehabilitation after injuries: The rehabilitation rollator can be used in the rehabilitation process after lower limb injuries, helping patients gradually regain mobility.
- Increasing independence: For some patients, a rehabilitation rollator can be a tool to increase independence in daily life, enabling independent movement.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

- Lack of spinal stability: People with severe spinal stability problems or spinal injuries may not be suitable candidates for using a rehabilitation rollator, as it may increase the risk of further injury.
- Uncontrolled seizures: For people with uncontrolled seizures or epilepsy, the use of a rehabilitation rollator can be dangerous for both the patient and those around them.
- Poor muscle strength: For people with very limited muscle strength, using a rehabilitation rollator may be difficult or ineffective. In such cases, a different approach to rehabilitation may be necessary, such as manual therapy or Muscle Strength Support Exercises.
- Balance problems: In patients with severe balance problems, the use of a rollator can be dangerous because it can increase the risk of falls and further injury.
- Rehabilitation rollator mismatch: Improperly fitting a rehabilitation rollator to a patient can lead to additional injury or worsening of the condition.


USE


IU_AT51043

INSTALLATION OF HANDLES

Unfold the main frame and hold it in a stable position. Then press the orange button so you can insert the handle into the frame tube. Slide the handle in until you hear a click/lock.

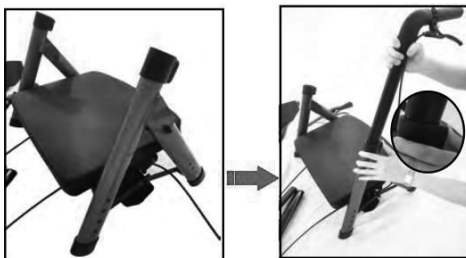**MOUNTING OF THE FRONT WHEELS**

Press the spring button so you can insert the front wheel into the main frame. Make sure that the spring button came out of the hole and you will hear a "click".

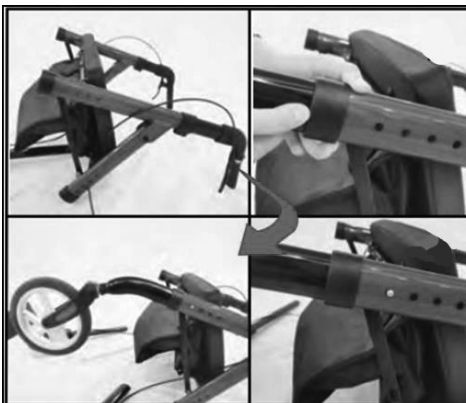**REAR WHEEL ASSEMBLY**

Press the spring button and insert the rear wheel tube into the main frame, you will hear a "click". Make sure the brake cable is not jammed.

ADJUSTABLE HANDLE HEIGHT

Press the orange button to adjust the height up or down.

**BACKREST ASSEMBLY**

Unscrew the screw of the backrest, and then insert the handle tube into the plastic parts. The knob is used to firmly fix the backrest to the rollator.

**HEIGHT ADJUSTMENT AT CASTERS**

Press the spring buttons on the front and rear tubes to adjust the height of the rollator.

FOLDING THE ROLLATOR

Pull the bar under the seat to fold the rollator.


SETTING PARKING/PARKING BRAKES


To apply the parking brake, push the brake handles down.

To release the parking brakes, grasp the brake handles again and pull them upward until they release.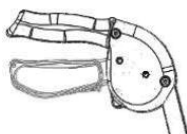

(unlocked state)

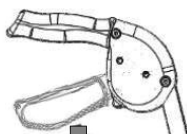(locked state)

USE OF HANDBRAKES

1. Pull the brake in the direction of the hand grips.
2. Perform one of the following actions:
 - A. NOT MOVING - hold the brake handle up:
 - B. MOVING - release the brake handle.

FEATURES

Solid rubber rear wheels, 8"

Front wheels, 10"

Frame: aluminum

Maximum load: 180 kg


Height to seat: 56 cm to 62 cm(4 degrees of adjustment)

Seat dimension: cushion-seat width 48 cm/seat width(distance between handles) 56 cm/ depth 36 cm/ height 5cm

Dimension of the walker: depth 79 cm/ width 68cm/ height 97 cm- 108 cm

Adjustable handles height: 97-108 cm(6 steps of adjustment)

Weight: 10 kg

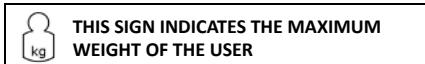COMPONENTS:

Rollator to assemble, backrest, set of wheels, Instruction manual

NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person or
- (b) temporary or permanent deterioration of the health of the patient, user or other person or
- (c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

NOTE: it is forbidden to use the product other than for its intended purpose .

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

WARNINGS

Be careful not to damage the rollator during accidental contact with obstacles encountered on the road. Always ensure that your rollator is fully unfolded and has no loose parts. If you are not using the rollator, ensure that the parking brakes are engaged. Before use, check the general condition of the wheels and ensure they are securely attached. Verify that the brakes are functional before use. When using a rollator, pay attention to your surroundings to move safely. Do not use the rollator as a wheelchair; it is designed to assist with walking. Do not sit on the rollator unless the parking brakes are engaged, as there is a risk of the rollator moving, which could lead to an accident. Sit on the seat carefully and slowly. If necessary, ask someone for help. Ensure that the adjustable handles are properly secured. Avoid placing your hands near the folding mechanism of the rollator to prevent injury. Do not hang any objects on the handles or brakes.

CLEANING AND CARE

Remember to check your rollator regularly. Before each use, check that all screws and knobs are tightened. Check that the brakes are working properly. Check the wear condition of the wheels. Check the looseness of the brake cable and the strength of the cable. Check whether any components are missing. If you find any defects or malfunctions, do not use the rollator, as it risks danger.

The rollator can be cleaned with a damp scrubber and mild soap. If necessary, you can use mild disinfectants and antibacterial agents. Do not rub with rough objects.

Be careful not to leave any cleaning residue on the handles or seat, as this can make these items slippery during use.

NOTE: DO NOT WASH THE SPONGE FROM THE SEAT, ONLY THE COVER SHOULD BE CLEANED.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

METHOD OF DISPOSAL OF THE PRODUCT

Once the product has been taken out of service, the medical device can be disposed of as normal municipal waste. If necessary, contact your local authority or the nearest waste collection point to find out how to properly dispose of the product. By acting in accordance with local regulations, you are helping to protect the environment.

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NOTATKI

A series of horizontal dotted lines for taking notes.

IU_AT51043

NOTATKI

A series of horizontal dotted lines for taking notes.

IU_AT51043

WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłńska 43 Tel. 22 518 36 00,
Fax 22 518 36 30 www.antar.net

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