**AT51110**  
**INSTRUKCJA OBSŁUGI, ROTOR REHABILITACYJNY**  
**USER MANUAL FOR REHABILITATION ROTOR**  
**BEDIENUNGSANLEITUNG FÜR REHABILITATION-ROTOR**  
**NÁVOD K POUŽITÍ REHABILITAČNÍHO ROTORU**  
**NÁVOD NA POUŽITIE, REHABILITAČNÝ ROTOR**  
**MODE D'EMPLOI, ROTOR DE RÉHABILITATION**  
**GEBRUIKSAANWIJZING, REVALIDATIEROTOR**  
**MANUAL DE USO, ROTOR DE REHABILITACIÓN**  
**MANUALE D'USO, ROTORE PER RIABILITAZIONE**  
**BRUKSANVISNING, REHABILITERINGSROTOR**

**MD**

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EN

The rehabilitation rotor is a compact exercise bike that can be easily stored when needed. It is primarily used for the rehabilitation of people with limb paralysis caused by, for example, an accident or stroke. Regular use stimulates blood circulation and increases muscle strength. It's a great way to do moderate cardiovascular exercise. Place it on a table to exercise your arms, or on the floor to cycle without leaving your chair.

**Technical Specifications:**

- **Frame:** Powder-coated steel
- **Rotor Pedals:** Nylon
- **Dimensions:** 36x50x24 cm
- **Weight:** 2.4 kg






Maximum user weight: 120 kg. The rotor has a resistance adjustment function with the included screw.

**THIS SIGN INDICATES THE MAXIMUM USER WEIGHT**

**Monitor Functions**

1. Unscrew and remove the screws on both sides of the screen, then carefully remove the screen from the housing.
2. Lift the battery latch located at the back of the screen and slide out the battery.
3. Insert a new 1.5 V AG13 battery with the positive pole facing down, and close the latch.
4. Place the screen back into the housing and tighten the screws.

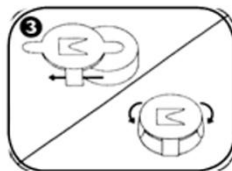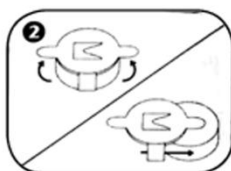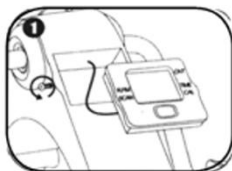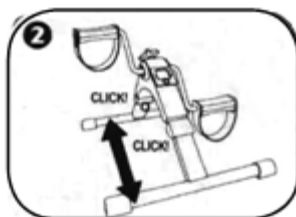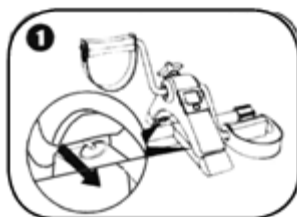
**Easy Setup in 2 Steps!**

Firmly pull the ring located at the connection of the stand legs.

### Display Functions

To activate the display, press the red button once or start pedaling. The display automatically turns off after 4 minutes of inactivity. All display functions activate when you start pedaling and deactivate when you stop. To switch between them, press the red button until the arrow points to the desired function.

- **CNT:** Number of rotations
- **TIME:** Duration of training
- **CAL:** Estimated number of calories burned
- **R.P.M.:** Rotations per minute
- **SCAN:** Switch (TIME>CNT>RPM>CAL)

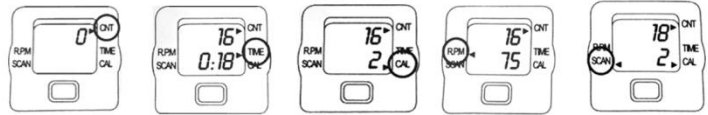### WARNING:

In the event of a "serious incident" related to the product, which directly or indirectly led to, could have led to, or may lead to any of the following:

- a) death of the patient, user, or another person, or
- b) temporary or permanent deterioration of the health of the patient, user, or another person, or
- c) serious threat to public health,

such "serious incident" must be reported to the manufacturer and the competent authority of the member state where the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

### WARNING:

If any pain, allergic reactions, or other disturbing, unclear symptoms related to the use of the medical device occur, consult a healthcare professional.

### DISPOSAL METHOD AFTER END OF USE

After the product is withdrawn from service, it can be disposed of as regular municipal waste, except for electric products – follow proper disposal methods for electric and electronic equipment.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawiślańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
[www.antar.net](http://www.antar.net)

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