AT51111

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ROLLATOR-USER MANUAL

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

RECOMMENDATIONS FOR. TO MAINTAIN SAFETY

To stay safe when using the product, THEN follow the guidelines below.

GENERAL WARNINGS

DO NOT assemble or use the product before thoroughly reading and understanding these instructions. If you do not understand the warnings, cautions, or instructions, contact a healthcare professional, retailer, or technician before attempting to assemble the device. This precaution is necessary to avoid injury or damage to the product. Each user should consult with his or her doctor or therapist to determine the appropriate adjustments and usage options for this product.

The rollator is not designed for the user sitting on it to steer it himself.

A doctor/therapist, other health care professional/vendor should be present when adjusting the height of the walker to ensure that the user is supported and that the screws are properly tightened.

When the rollator is unfolded always make sure that the tube under the seat is locked and properly connected to the frame.

Ensure that the height adjustment knobs are locked securely to prevent any movement of the main support structure due to pressure. Failure to secure these knobs may result in injury.

Before use, verify that ALL handle and seat adjustments have been secured and that the wheels and moving parts of the product are in good condition.

During use, ALL wheels must BE in contact with the ground AT ALL TIMES to ensure the stability of the rollator. Neglecting to follow these recommendations may lead to the rollator tipping and potential injury to the user.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device independently or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

CHARACTERISTICS

The AT51111 rollator is designed for indoor and outdoor use. The device is designed to serve only as a rollator with seating for one person. It is prohibited to carry more than one person.

- lightweight construction, aluminum frame
- easy installation
- comfortable backrest, seat and large shopping bag
- half-full wheels
- parking brakes
- easy transportation
- LED light
- 6 levels of handle adjustment
- twist front wheels

INDICATIONS

The rollator is designed for people with limited mobility who still have the ability, strength and balance to leave their residence with the help of a wheeled rollator.

CONTRAINDICATIONS

Cases in which the use of a rollator is not advisable:

- Perception problems
- Severe balance problems
- No use of hands
- Lack of ability to sit down

NOTES:

During the use and operation of the rollator, as well as when folding and adjusting mechanisms, there may be a danger of trapping and/or squeezing parts of the user's/companion's body in the openings/gaps between components.

These activities should be performed with particular care.

Once the adjustment is complete, stabilize the position by tightening the nuts/bolts carefully.

SPECIFICATION

Seat width 45cm

Seat height 54cm

Total width 65cm

Total length 71cm

Total height/handle height(adjustable in 6 positions) 81cm-94cm

Maximum user weight 136kg

Maximum pannier load 5kg

Dimensions after folding(L x W x D) 71cm x 25cm x 81cm

Front wheels, swivel 25cm x3.5cm

Rear wheels braking, untwisted 21cm x3.5cm

Aluminum frame

Seat nylon

Backrest nylon

Polyurethane tires

This symbol indicates the maximum weight of the user!

NOTE

Make sure that the supplied rollator is complete and in good condition. If the rollator is damaged, refrain from using it. In this case, contact your dealer immediately.

COMPONENTS

Delivery includes the following, major components:

- 1 outer package
- wheeled rollator with back strap and tilt footrests
- mesh fabric pouch with reflector and zipper
- 2 horizontal handles, with padding
- instructions for use
- 1 position LED lamp

NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of the condition of a patient, user or other person or
- (b) temporary or permanent deterioration of the health of the patient, user or other person or
- (c) a serious threat to public health


the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTE: The manufacturer is not responsible for damage caused by negligent maintenance, inadequate servicing or resulting from failure to follow the recommendations contained in this manual.

CAUTION: It is forbidden to use the product other than in accordance with its intended use!

CAUTION: If the product is used incorrectly, there may be a risk of tipping over. Please follow the recommendations for getting on/sitting down/moving. After adjustment, stabilize the position by tightening the nuts/bolts carefully.

BASIC INFORMATION

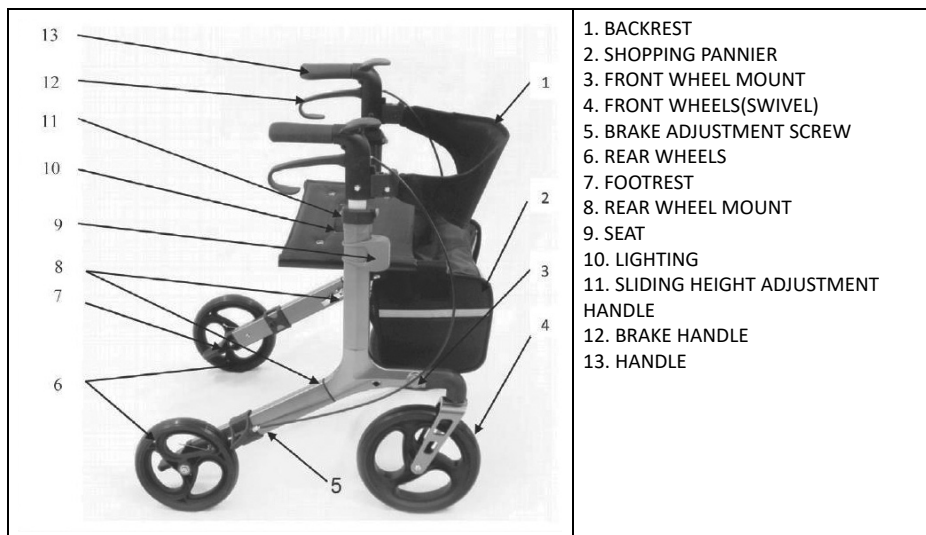INSTALLATION OF ROLLATOR AT51111

HANDLE HEIGHT

The height of the handle can be adjusted to your own height. This action does not require the use of any tools.

Under each handle there is a red lock (Figure 3). The lock must be pressed and held down.

In this setting, the handle can be lowered or raised.

Release the lock. Pull or push the lock again until you hear a click indicating that the lock is in the right place and that it cannot be moved any further.

Repeat the above steps on the other side so that both handles are at the same height.

3

SHOPPING PANIER

The rollator is supplied with a pannier installed. The pannier can be easily removed Or hung over the mountings (Figure 4).

4

IU_AT51111

BRAKES

NOTE: HAVE A SPECIALIST ADJUST THE BRAKES, AS IMPROPER INSTALLATION OF THE BRAKES CAN BE LIFE-THREATENING.

INCREASE / DECREASE IN BRAKING FORCE

1. loosen the allen head screw (Figure 1, item 1).
2. Tighten or loosen the Bowden cable by pulling or pushing it out (Figure 5, item 2).
3. tighten the allen head screw.

4 Minor adjustment:


Loosen the counter-nut (Figure 6, item 1) so that the adjustment screw (Figure 6, item 2) can be turned.

Turn the adjustment screw clockwise: decrease the braking force.
Turn the adjustment screw counterclockwise: increase the braking force.


Finally, re-tighten the counter-nut.

Repeat the steps at the second brake.

Adjust the brakes so that they do not cause resistance when moving, but allow easy movement. Locked brakes prevent the rollator from moving.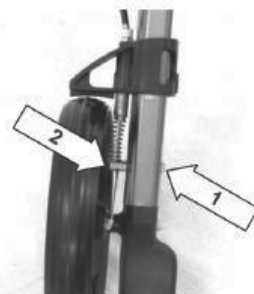

5

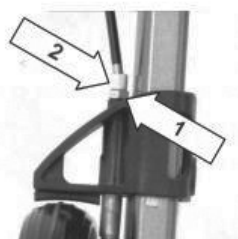6

USE OF AT51111 ROLLATOR

OPENING AND FOLDING THE ROLLATOR

To save space during storage, the rollator can be folded.

To use it again, the rollator must be opened by pressing on the outer edges of the seat (Figure 7). There is a locking mechanism at the bottom of the rollator to prevent it from folding unintentionally. An audible click when unfolding the rollator indicates that the locking mechanism is working correctly. To fold the rollator, pull up the handle in the seat (Fig. 8).

7

8

TRANSPORTATION AND STORAGE

To make it easier to transport the rollator and save space when storing it, the front wheels can be turned and the attachment of the rear wheels can be removed (Figure 9).

9

FRONT WHEELS

To release the front caster lock, press the red latch (Figure 10). Then pull the front wheels forward and rotate them 180° upward. To use the rollator again, pull the front casters forward and rotate them downward. A click means that the casters are in the right position; you can now adjust the seat accordingly.


10

REAR WHEELS

Then fold the lever (Figure 11). In the last step, press the button and slide the rear caster mechanism downwards. Note that the rear wheels are always connected to the rollator via the brake cable.

To unfold them, push the rear caster mechanism back into the frame. A click means that the wheels are in the right position. The lever should now be folded back.

Make sure that the rear wheels are properly secured, and that the brake works properly.

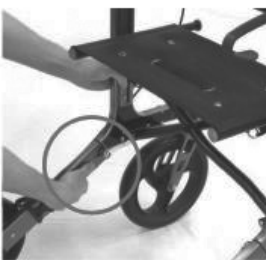11

MOVING AROUND WITH THE HELP OF A WHEELED ROLLATOR

First, check the operation of the rollator on a level, dry surface. If possible, ask another person to help. At first, take time to test the use of the rollator, and once you become confident in using it, you can start using it on inclined surfaces.

NOTE: When moving with the rollator, be sure not to move too far away from the handles. Do not hunch over.

You choose the speed and direction of movement by pushing the rollator yourself. To reduce the speed of movement, use the brakes. Always use both brakes when braking. Otherwise, the rollator may turn on one wheel and cause the user to fall.

Obstacles that appear on the road should be circumvented. If there are steps in the road, look for a ramp, if the obstacle is stairs, use an elevator. If this is not possible, the rollator should be moved over the obstacle. As far as possible, another person should be asked to help.

FOOTRESTS FOR EASY TILTING

The rollator is equipped with two footrests for easy tilting (Figure 12), designed to make it easier to get over curbs. Stepping on the footrests with your foot and pulling the handles with your hands at the same time raises the front wheels. This makes it easier to go over curbs and over small obstacles.

When performing such a maneuver, remember to maintain balance.

12

BRAKE AND PARKING BRAKE

The rollator is equipped with a hand-operated brake and a parking brake to prevent accidental movement of the rollator.

To easily and effortlessly stop the rollator, pull up both brake levers (Figure 13).

13

The effectiveness of braking depends on the force with which the brake levers are pressed. To activate the parking brake, press the brake levers downward; a click means the lever is locked in the correct position. (Figure 14).

14

To release the parking brake, pull the brake levers upward. (Figure 15).

15

SEAT ON THE ROLLATOR

The rollator is equipped with a seat with a backrest. Before you sit on the seat, make sure the parking brake is engaged. Otherwise, the rollator may roll away causing you to fall. Do not use extra cushions; they may slip off causing a fall.

LIGHTING (OPTIONAL)

To light the way in front of the rollator, attach a battery-powered spotlight to its frame. The headlight is turned on with a switch. The headlight has an additional flashing function.

MOVING THE ROLLATOR

To move the rollator, for example, over a curb, fold it. Lift the rollator with the handles or by grasping the frame. Be aware that body parts may then be caught.

SECURITY INFORMATION:

Before each use, check that the rollator is not damaged and that the brakes work properly.

The rollator is not suitable for use on escalators (danger of falling!).

The rollator is made of non-flammable materials.

The maximum user weight is 136 kg (300 lbs); the maximum pannier load is 5 kg (11 lbs). The rollator cannot be used to transport cargo. The pannier is not suitable for transporting animals.

To save space, the rollator can be folded. Please note that when folding and unfolding, there is a danger of catching body parts. When folding or unfolding the rollator, do not pull on the parts attached to it (brake levers, wheels, pannier, etc.), but only on its frame.

Please note that the seat and backrest material exposed to sunlight may become hot and cause skin burns on contact. Therefore, these parts or the entire stroller should be covered thus protecting them from the sun's rays.

CLEANING

Frame parts can be cleaned with a damp cloth. For hard-to-remove dirt, you can use a mild cleaner.

The wheels can be cleaned with a damp brush with plastic bristles (do not use wire brushes!).

The backrest, seat and pannier can be washed with water and mild soap.

NOTE: Pressure washers and steam cleaners must not be used for cleaning. After cleaning, the rollator should be dried with a towel or allowed to dry.

DISINFECTION

The rollator can be disinfected with common household disinfectants. Approved alcohol-based spray disinfectants or wipe disinfectants should be used for this purpose.

After disinfecting, set the rollator aside to air-dry, observing the action times specified by disinfectant manufacturers.

In order to ensure the long-term and safe operation of the product, it is recommended to regularly inspect all screws and mounting components. Before each use of the product, check that all screws are properly tightened and there are no signs of loosening.

Regularly check that the product is not missing any parts and that all components are in good working order. If you notice wear, mechanical damage or missing parts, immediately stop using the product and contact the manufacturer's service department to avoid the risk of an accident or further damage.

Watch for any unusual sounds or vibrations that may indicate loose components or other technical problems. Remember, when replacing screws or components, to use only original parts recommended by the manufacturer.

RE-USE.

The AT51111 rollator is reusable. The rollator should first be cleaned and disinfected according to the attached information on maintaining its hygiene. When transferring the rollator to another person, also remember to give the new user all the necessary technical documents. The rollator must be inspected by an authorized specialist and transported in proper condition.

STORAGE:

Store the product in a dry place, do not expose to direct sunlight.

UTILIZATION

Once the product is decommissioned, the medical device can be disposed of as normal municipal waste. To dispose of it properly, please contact your local authority or the nearest waste collection point to find out how to dispose of the product properly. By acting in accordance with local regulations, you are helping to protect the environment.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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