AT51117 / AT51117B

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ENG

Intended use:

The shower chair (AT51117, AT51117B) is designed for people with reduced mobility, to make it easier for them to use the shower.

The stool on legs is height adjustable to ensure maximum comfort for the user. The product has non-slip caps. After the user has finished using the stool, care should be taken to ensure the cleanliness of the product by washing it thoroughly. Standard cleaning and disinfecting agents may be used. Once the product has been taken out of service, it must be dealt with in the manner prescribed for municipal waste.

SPECIFICATION

colour	white / blue
seat width	32,5 cm
seat depth	32,5 cm
seat height	38 - 56 cm
maximum user weight	115 kg
weight	1,5 kg

This symbol means: "Maximum user weight"

ATTENTION:

In the event of a product-related

"serious incident" that directly or indirectly led to, could have led to or is likely to lead to any of the following:

- a) death of a patient, user or other person, or
- b) a temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

This 'serious incident' should be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products..

ATTENTION:

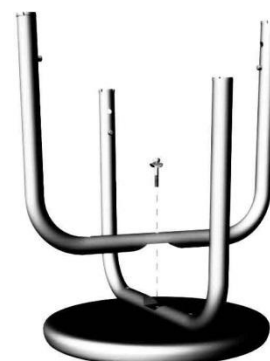
In the event of pain, allergic reactions or other distressing symptoms, unclear to the user, related to the use of the medical device a health professional should be consulted.

ATTENTION: Before use, check that all elements have been assembled correctly.

ATTENTION: it is forbidden to use the product in a way other than for its intended purpose

CONTRAINDICATIONS

physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

Pic. 1

Pic. 2

TARGET GROUP OF PATIENTS

The healthcare professional indicates the use of the product to adults and children at their own risk, taking into account the available variants / sizes / necessary functions / size and indications, taking into account the information provided by the manufacturer.

Installation:

1. To simplify installation, place the shower stool on a flat surface..
2. Place the legs of the stool on the seat and fasten them with a screw.
3. place a washer on each hole and secure the stool parts with the remaining screws.
4. You can now insert the legs of the shower stool. Ensure that the spring pins are clearly seated and are at the same height.

Comments:

1. During use and when assembling and adjusting mechanisms there may be a danger of entrapment and/or compression of parts of the user's/companion's body in openings/gaps between components.
2. Once the adjustment is complete, stabilise the position by tightening the nuts/bolts firmly.
3. Before each use of the product, always check that all adjustable parts are securely fastened.
4. Be careful on wet or slippery surfaces.
5. Do not insert your fingers into the holes in the seat for draining water. There is a risk of cutting or getting your finger stuck..
6. Do not use a defective or broken product.
7. Keep the product out of the reach of children.
8. Only an authorised service centre may replace missing, damaged or worn parts.
9. Do not use high-pressure cleaners or aggressive, corrosive cleaning agents for cleaning.
10. Special care should be taken when using the product.
11. Check that the product is properly assembled.
12. Do not lean excessively, especially backwards, as this may result in tipping over.

The maximum weight of the user is 115 kg

CLEANING/STORAGE

The product should be cleaned with delicate detergents, do not use rough cleaning objects. Store at room temperature, in a dry place, away from direct sunlight and moisture.

HOW TO DISPOSE OF THE PRODUCT AT END-OF-LIFE

Once the device has been taken out of service, the medical device can be disposed of as normal municipal waste.

NOTE

The manufacturer is not responsible for damage caused by neglected maintenance, improper servicing or failure to comply with the recommendations contained in this manual.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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