AT51120

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DESCRIPTION AND PURPOSE

The shower chair with swivel seat (AT51120) is an assistive device facilitating the performance of sanitary activities. It is intended for people with reduced mobility for whom it is impossible or difficult to perform daily activities independently. The stool is recommended for use in showers, baths, bathrooms and other humid rooms, at home and in hospitals. The stool legs are height-adjustable to ensure maximum comfort for the user.

The product has anti-slip caps. The stool should be cleaned thoroughly after use. Standard cleaning and disinfecting agents may be used. After the product has been taken out of use, it should be handled in the way provided for municipal waste.

KEY FEATURES

- 360° swivel function allows the user to turn around without having to stand up.
- seat made of durable and easy-to-clean material; lightweight aluminium frame (corrosion-resistant),
- seat diameter 36 cm;
- 5-stage height adjustment (every 2.5 cm) makes possible to adjust the height of the seat to the user's height (from 40 to 52 cm);
- anti-slip rubber caps with increased contact surface with the ground guarantee high stability;
- easy and convenient stool folding system without the use of tools.

SPECIFICATIONS:

Basic material: aluminium / plastic

Adjustable height: Yes

Load capacity: 136 kg

Seat height: 40 - 52 cm

Seat dimensions: 36 cm

Distance between legs when the seat is raised to the minimum: L 32 x W 32 cm

Distance between legs when the seat is raised to the maximum: L 33 x W 33 cm

Total weight: 2.38 kg

THIS SIGN MEANS A MAXIMUM
WEIGHT OF THE USER!

INCLUDED: chair, user manual

ASSEMBLY

1. To simplify the installation, place the shower stool on a flat surface.
2. Place the legs of the stool in the designated holes on the reverse side of the seat (snap fastening).
3. Place the stabilising strut in the intended location (snap-on fitting).
4. You can now adjust the height of the shower stool. Make sure that the latches are locked and are at the same height.

DISASSEMBLY

Follow the steps described under „ASSEMBLY“ in reverse order.

NOTES

1. During use and when assembling and adjusting the mechanisms there may be a danger of entrapment and/or compression of parts of the user's/companion's body in openings/gaps between components.
2. Always check that all adjustable parts are securely fastened before using the product.
3. Be cautious on wet or slippery surfaces.

4. Do not insert your fingers into the holes in the seat for draining water. You risk cutting or getting your finger stuck.
5. Do not use a defective or broken product.
6. Keep the product away from children.
7. Only an authorised service centre should replace missing, damaged or worn parts.
8. Do not use high-pressure cleaners or aggressive, corrosive cleaning agents for cleaning.
9. Special care should be taken when using the product.
10. Check that the product is correctly assembled.
11. Do not lean excessively, especially backwards, as there is a risk of tipping over.

CAUTION:

In the event of a device-related „serious incident“ that has directly or indirectly led to, could have led to, or is likely to lead to any of the following:

- a) death of a patient, user or other person, or
- b) temporary or permanent deterioration of the health of a patient, user or other person, or
- c) a serious risk to public health

the above „serious incident“ must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In case of Poland, the competent authority is the Office for Registration of Drugs, Medical Devices and Biocidal Products.

CAUTION:

If you experience pain, allergic reactions or other worrying symptoms that are unclear to you, related to the use of the medical device, consult your healthcare professional.

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/sizes/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

MAINTENANCE

1. Ensure that all attachments are always secure.
2. Promptly replace any damaged or deteriorated components.

CLEANING

Clean the product with a damp cloth and mild detergent. Do not use rough tools or strong detergents for cleaning.

HOW TO DISPOSE OF THE PRODUCT AT END-OF-LIFE

After the device has been taken out of service, the medical device can be disposed of as normal household waste.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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