AT51128
INSTRUKCJA UŻYTKOWANIA
STOPIEŃ PRZYWANNOWY DLA OSÓB NIEPEŁNOSPRAWNYCH
INSTRUCTION MANUAL
BATHTUB STEP FOR THE DISABLED
GEBRAUCHSANWEISUNG
BADETREPPE FÜR BEHINDERTE
NÁVOD K POUŽITÍ STUPÍNEK K VANĚ NASTAVITELNÝ
NÁVOD NA POUŽITIE
KÚPEĽNÝ SCHODÍK PRE ZDRAVOTNE POSTIHNUTÉ OSOBY
MANUEL D'INSTRUCTION
MARCHE DE BAIN POUR HANDICAPÉS
HANDLEIDING
OPSTAPJE VOOR GEHANDICAPTEN
MANUAL DE INSTRUCCIONES
ESCALÓN DE BAÑO PARA DISCAPACITADOS
MANUALE DI ISTRUZIONI
GRADINO DA BAGNO PER DISABILI
BRUKSANVISNING
BADSTEGE FÖR FUNKTIONSHINDRADE

PL

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EN

Thank you for purchasing our product, please read the instruction manual carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions or instructions, please contact your health care professional or dealer to avoid damage to the product.

APPLICATION:

A bathtub step is a special device that enables people with disabilities to enter a bathtub safely and comfortably. The product can greatly improve the comfort of daily life.

Indications

The product is indicated for people who have limited mobility due to illness, injury, trauma, age, disability. The product provides support for getting into the bathtub.

FEATURES:

- Wide footrest platform guarantees comfortable and convenient use
- Anti-slip caps provide safety and stability
- Platform surface with perforation for water drainage
- Adjustable height (3 degrees of adjustment)

CONTRAINDICATIONS

Physical or mental limitations (such as visual impairment) that prevent safe handling.

1. too much weight of the user - destruction of the product, fall of the user.
2. improper installation - instability of the product, resulting in a fall of the user.
3. Contamination of the product - failure to clean the product, resulting in bacterial contamination.
4. Improper positioning of the product - uneven surface, product unstable, resulting in user fall.
5. contaminated surface - the surface on which the product is placed is contaminated with slippery substances (e.g. oil, soap) the product unstable as a result of the fall of the user.

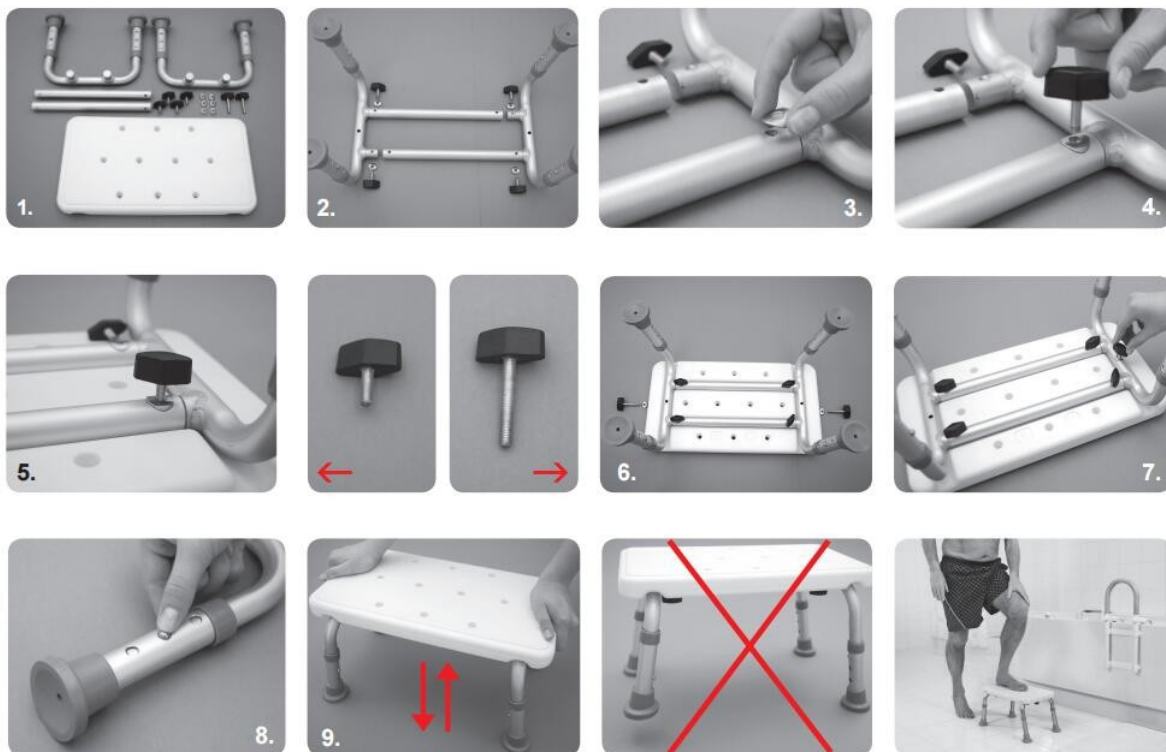
PACKAGE INCLUDES: plastic platform (1pcs), U-shaped aluminum legs (2pcs), straight tubes (2pcs), set of butterfly type screws, short knobs (4pcs), long knobs (2pcs), metal washers (6pcs), instruction manual.

ASSEMBLING

First of all, check that the package contains all the necessary components.

Next, screw the aluminum legs to the frame components using short knobs and metal washers (2,3,4,5).

Then attach the platform to the aluminum structure using long knobs and washers (6,7). The height of the step can be adjusted, set the same height on each leg (8,9,10).

DISASSEMBLY

All the steps described above should be performed in reverse order.

TECHNICAL PARAMETERS:

- Platform area: 40 x 22 cm
- Platform height: 21-26 cm
- Total width: 41,5-43,5 cm
- Maximum user weight: 110 kg
- Weight: 1.5 kg
- Package size: 42*10*26 cm

This symbol indicates the maximum weight of the user

NOTICE:

It is forbidden to use the product other than as intended.

CAUTION:

Before use, check that all components have been assembled correctly.

ATTENTION: The manufacturer is not responsible for damage caused by negligence in cleaning, storage of the product or resulting from failure to follow the recommendations contained in this manual.

CAUTION:

In the event of a device-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

STORAGE

Store the device in a dry cool room, away from moisture and direct sunlight.

MAINTENANCE AND CLEANING

Clean the product with a damp cloth and mild detergent. Do not use rough objects for cleaning, it may scratch the elements of the product.

CAUTION: Periodically perform an inspection of the product to determine whether the product is suitable for use. There is a danger of loose screws. In this case, tighten all loose parts.

Before each use, verify that the bathtub step stands stable, does not slip. If you find damage or deformation of the anti-slip caps, platform or metal frame of the product, stop using the product.

HOW TO DISPOSE OF THE PRODUCT AFTER DECOMMISSIONING

After the device is taken out of service, the medical device can be disposed of as ordinary municipal waste

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the intended use of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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