AT51201, AT51202, AT51203, AT51204, AT51205

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AT51201, AT51202, AT51203, AT51204, AT51205

If any defects or damage occur, please inform the supplier immediately. Do not use the product if you think it is damaged or has missing parts.

Toilet lift seats are for people with arthritis, after hip surgery, and others who have difficulty moving, standing, and sitting. The raised toilet seat has a modern restraint system that allows easy stabilization in the toilet. The ergonomic surface ensures an even distribution of compressive forces over the user's body. A slight inclined tilt of the seat forces the correct posture and reduces push-ups in the hip and knee joints. The reinforcement of the toilet has two incisions at the front and back, which facilitates hygienic activities in a sitting position.

Product features:**AT51201, AT51202, AT51203**

- two mounting flaps adapt to the toilet bowl
- the installation of the raised toilet seat is easy, you just need to screw two hooks
- the product is available in 2 heights: 10 and 15 cm (**AT51201, AT51202**)

AT51204:

- a secure attachment to the toilet with tightened clamps (more stable than with clippers)
the mounting does not require any tools
- equipped with hinged handles to help a disabled person getting up from the toilet
- the seat riser fits to the most toilet bowls, it also has 3 levels of height adjustment

AT51205:

- secure attachment to the toilet
- with folding armrests to help a disabled person get up
- the raised toilet seat fits most toilet bowls

Functionality:

- large openings at the front and back facilitate intimate hygiene
- can open the cover without touching the booster seat
- Can take a sitting position with a low lid

Technical data AT51201, AT51202:

Size (width x length): 40 x 36 cm
 Maximum user weight: 180 kg
 Product weight: AT51201 -1.2 kg, AT51202 -1.4 kg
 Hole dimensions: 21 x 27 cm
 Height AT51201: 10 cm
 Height AT51202: 15 cm

Technical data AT51203

Size (width x length): 40 x 36 cm
 Maximum user weight: 136 kg
 Product weight: 2.2 kg
 Height: 10 cm
 Material: 100% PU (polyurethane)
 Hole dimensions: 23 x 26 cm

Technical data AT51204:

Total height (cm): 37-59 (with raised cover) cm
 Size, width x length (cm): 40 (seat riser)/57 (with armrests) x 46
 Adjustable seat height (cm): 6, 10, 14
 Width between the upper parts of the armrests (cm): 47
 Product weight: 4.2 kg
 The user's weight and the maximum load must not exceed 150 kg
 Hole dimensions: 23 x 27 cm

Technical data AT51205:

General depth. (cm): 35-60 (with raised bowl) cm x 46
 Size, width x length (cm): 40 (booster seat)/54 (with armrests)
 Width between the top of the armrests (cm): 47
 Product weight: 3.6 kg
 The weight of the user and the maximum weight of the load must not exceed 136 kg
 Hole dimensions: 23 x 24 cm
 Height: 10 cm


THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER

Assembly AT51201 , AT51202, AT51203:

Connect the front member of the raised toilet seat to the front of the toilet bowl, then screw the two fasteners with the side screws. The ergonomic and robust construction of the screws allows a precise and permanent installation of the raised toilet seat. The reinforcements were secured against slippage.

The raised and lower covers are secured against accidental disassembly.

AT51201, AT51202

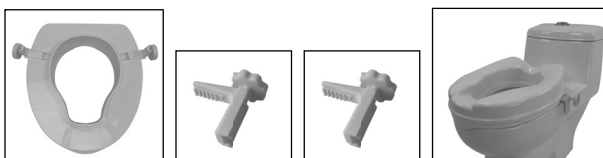
AT51203


Assembly AT51204:

- remove the two front plastic plugs
- loosen the screw and place the crossbar at the required height
- tighten it using two screws
- fix the rear crossbar in a similar way, both should be at the same height
- remove the existing toilet seat, insert two threaded rods in the place of its fixing on the toilet bowl
- tighten the seat riser using plastic caps
- before first usage, make sure that the seat is correctly and securely mounted.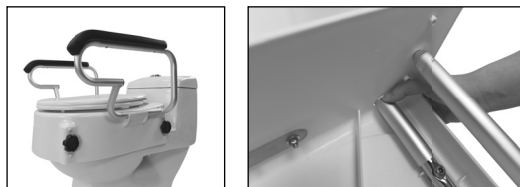

Assembly AT51205:

- armrests should be attached to the toilet lift with screws located in the assembly with the armrests (requires allen key size 4)
- Connect the front part of the raised toilet seat to the front of the toilet bowl, then firmly screw the accessory to the front of the raised toilet seat using the screw. The ergonomic and robust construction of the screw allows a precise and permanent installation of the raised toilet seat. The handle was secured against slipping.
- The raised and lower covers are secured against accidental disassembly.
- before the first use, make sure that the seat has been properly and safely mounted.

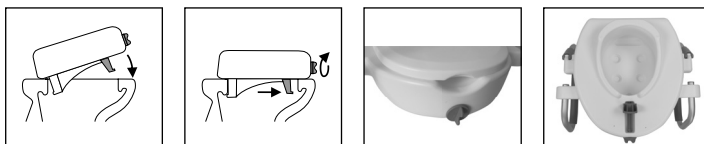Cleaning:

- easy to clean, free of external rough surfaces
- do not use abrasive or scouring cleaning agents.
- if any defects or damage appear, please inform the supplier immediately. Do not use the product if it is suspected of being damaged or missing parts.
- clean it with hot water and soap or with soft detergents.

Disinfection by wiping

With commercially available disinfectants according to EN 12720

1. Put on protective gloves before preparing the product
2. Spray the areas to be treated with a disinfectant
3. Rub the treated areas with a paper towel, if necessary try again
4. Discard protective gloves and paper towels

Contraindications

- Troubled perceptual
- Severe balance disorders
- Inability to sit

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

DISPOSAL OF THE PRODUCT

The product must be disposed of in accordance with the applicable disposal conditions.

NOTES

1. During use and during the assembly and adjustment of the mechanisms, there may be a risk that parts of the user/caregiver's body will be trapped and/or compressed in the openings/spaces between the elements.
2. After adjustment, stabilize the position by tightening the nuts/bolts.
3. Before each use of the product, check that all adjustable parts are securely attached.
4. Beware of wet or slippery surfaces.
5. Do not place your fingers in seat holes that drain water. This can cause your fingers to be compressed or pinched.
6. Do not use a defective or broken raised toilet seat.
7. Keep the product away from children. The maximum load does not apply to the cover.
8. The blanket is not intended to sit.
9. The raised toilet seat is not intended to be seated permanently.
10. By lowering the armrests, clothes or body parts can be pinched.
11. Missing, damaged or worn parts can only be replaced by an approved service.
12. Do not use pressure washers or harsh, corrosive detergents for cleaning.

WARNING:

In the event of a device-related "serious incident", which directly or indirectly led to, could have led to, or is likely to lead to any of the following events:

- a) death of a patient, user or another person, or
- b) temporary or permanent deterioration of the health of a patient, user or another person, or
- c) a serious risk to public health, such a "serious incident" must be notified to the manufacturer and to the competent authority of the Member State, in which the user or patient resides. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

WARNING:

In the event of pain, allergic reactions or other alarming, unclear symptoms related to the use of the medical device, please consult a healthcare professional.

NOTE: Before use, check that all components are properly assembled.

ATTENTION: The manufacturer is not responsible for damage caused by neglect of maintenance, inadequate servicing or resulting from failure to comply with the recommendations contained in this manual.

NOTE: It is forbidden to use the product contrary to its intended use!

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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