AT51209
INSTRUKCJA UŻYTKOWANIA, NASADKA TOALETOWA PODWYŻSZAJĄCA
USER MANUAL, RAISED TOILET SEAT
BEDIENUNGSANLEITUNG, ERHÖHTER TOILETTENSITZ
NÁVOD K POUŽITÍ, NÁSTAVEC NA WC
NÁVOD NA POUŽITIE, ZVÝŠENÁ TOALETNÁ DOSKA
MODE D'EMPLOI, SIÈGE DE TOILETTE SURÉLEVÉ
GEbruikSAANWIJZING, VERHOOGDE TOILETBRIJL
MANUAL DE INSTRUCCIONES, ASIENTO ELEVADO PARA INODORO
ISTRUZIONI PER L'USO, SEDILE WC RIALZATO
BRUKSANVISNING, FÖRHÖJD TOALETSTOL

PL

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EN

Thank you for purchasing our product. Please carefully read the user manual. Do not use the product before reading and understanding this manual. If you do not understand the warnings, notes, and recommendations, contact a healthcare specialist or the seller to avoid damaging the product or causing personal injury.

WARNING: Check all parts for any damage caused during transport. If such damage is noticed, DO NOT use the product. More information is available from the seller.

TARGET PATIENT GROUP

People suffering from diseases, dysfunctions, or injuries for which this product is intended for treatment, rehabilitation, or compensation (see the section on the intended use of the product in this user manual). The product may be purchased by the user independently or based on recommendations from a doctor, therapist, or other specialist. Whether purchased independently or recommended by a doctor/therapist/specialist, available sizes, necessary functions, product variants, indications and contraindications for use, and the information provided by the manufacturer should be considered.

USE/INDICATIONS

It is designed for people with arthritis, those recovering from hip surgery, and others who have difficulty moving, standing up, or sitting down. The raised toilet seat can increase the height of the toilet, making it easier to sit down and stand up, particularly for those with joint problems such as rheumatism or arthritis.

Reduction of fall risk: For individuals who struggle with balance, a stable raised seat can significantly reduce the risk of falling in the bathroom, one of the most common places for accidents at home.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product.

Hygiene issues: While raised toilet seats are designed to improve hygiene, some may not be sufficiently clean or may be used by multiple individuals, which could lead to the spread of bacteria and infections. Always ensure the seat is properly cleaned and disinfected.

User comfort: Some raised seats may be uncomfortable for people with certain health conditions, such as hemorrhoids or anal fissures. In such cases, hard materials may cause additional discomfort or pain.

Reduced sensory perception: Individuals with limited sensation in the lower body should be cautious of seats that may be too hard or cold, as this could lead to pressure sores or other skin injuries.

Inability to sit down.

Severe balance disorders.

FEATURES

Easy installation

Sturdy handles

Greater comfort in the bathroom with a raised seat and two solid handles with an anti-slip coating. The seat has a front cutout to facilitate intimate hygiene. The smooth surface makes it easy to clean. Fits all standard toilets available on the market.

Material: plastic, aluminum

Base color: white

Length: 43 cm

Width: 49.5 cm

Maximum user weight: 113kg

This symbol indicates the maximum user weight.

PARTS LIST

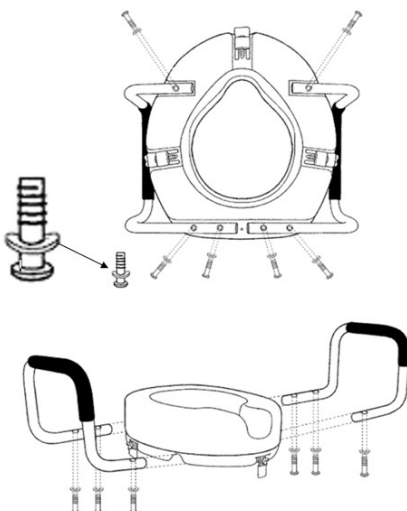
Raised toilet seat, pads, screws, armrests, user manual.

ASSEMBLY

Place the raised seat upside down on a flat surface and position the armrests so that the holes in the armrests align with the threaded sleeves on the bottom of the seat, see the diagram below.

Place the curved washers on the screws. Insert the screws through the holes in the armrests and attach the armrests to the raised seat. Do not overtighten the screws to avoid damaging the threads.

Place the installed raised seat with the front cutout over the toilet bowl, facing forward. Tighten the clamps on the seat until it is stable and securely fastened.

WARNING:

The armrests are designed to assist with balance when sitting down and getting up from the toilet. Make sure you apply even pressure to both armrests. The armrests are not designed to support your full body weight.

Regularly check that the raised toilet seat is stable and securely fastened to the toilet, as it may loosen over time. Tighten any loose clamps as needed to avoid the risk of injury.

Do not sit on the raised seat with a wet bottom after bathing or showering to avoid the risk of slipping.

WARNING:

In the event of a "serious incident" related to the product, which directly or indirectly resulted in, could have resulted in, or may result in any of the following events:

- a) the death of the patient, user, or another person, or
- b) temporary or permanent deterioration of the health of the patient, user, or another person, or
- c) a serious public health threat,

the "serious incident" must be reported to the manufacturer and the competent authority of the member state where the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

WARNING:

If pain, allergic reactions, or other concerning or unclear symptoms related to the use of the medical device occur, consult a healthcare professional.

WARNING: It is prohibited to use the product in a manner other than as intended.

CLEANING AND MAINTENANCE

Use a mild household cleaning agent for cleaning, and if necessary, a regular disinfectant. Do not use aggressive, corrosive, or abrasive cleaning agents. Thoroughly dry the raised seat before use.

To ensure long-term and safe operation of the product, it is recommended to regularly inspect all screws and mounting elements. Before each use, check that all screws are properly tightened and that there are no signs of loosening.

Regularly check whether any parts of the product are missing and whether all components are in good technical condition. If you notice wear, mechanical damage, or missing parts, stop using the product immediately and contact the manufacturer's service department to avoid the risk of accidents or further damage.

Pay attention to any unusual sounds or vibrations that may indicate loose elements or other technical problems.

Remember to use only original parts recommended by the manufacturer when replacing screws or components.

STORAGE

Store the product in a dry place and do not expose it to direct sunlight.

DISPOSAL

Once the medical device has been decommissioned, it can be disposed of as regular household waste. For proper disposal, please contact your local authority or the nearest waste collection point to find out how to properly dispose of the product. By following local regulations, you help protect the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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