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**SKŁAD**  
PVC

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Thank you for purchasing our product. Before using, please read this user manual carefully. Incorrect use may lead to health detriment, injury or accidents. Therefore, in order to use your wheelchair safely and successfully, please read this manual carefully.  
NOTE: Inspect all parts for damage caused in transit. If any such damage is found, DO NOT use the product. More information from the manufacturer.

#### RECOMMENDATIONS SAFETY

To maintain safety when using the product, you SHOULD follow the guidelines below.

##### ATTENTION:

When a product-related "serious incident" occurs, which directly or indirectly led, or could lead to, any of the following events:

- a) death of a health condition of a patient, user or other person, or
- (b) temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

##### ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of a medical device, consult a healthcare professional.

**NOTE:** In the event of pain, great discomfort, or an allergic reaction, please contact a healthcare professional as soon as possible.

**ATTENTION:** The manufacturer is not responsible for damage caused by neglect of maintenance, inadequate servicing, or resulting from failure to comply with the recommendations contained in this manual.

**ATTENTION:** It is forbidden to use the product other than in accordance with its intended purpose

**CAUTION:** There may be a risk of tipping over if the product is used improperly. Please follow the boarding / dismounting / moving instructions.

**CAUTION:** flammable product.

NOTE: the product may not be used by children under 3 years of age.

**INTENDED USE:** The product is intended for people who require rehabilitation or lead a sedentary lifestyle. Sitting on the ball forces you to correct your body posture (the position of the pelvis and spine), which leads to an upright sitting position.

**CONTRAINDICATIONS:** physical or mental limitations (e.g. visual impairment) that prevent the safe handling of the product

##### STORAGE:

The product should be stored in a dry place, not exposed to direct sunlight.

##### MAINTENANCE

The ball and stand should be cleaned with a damp cloth with a mild detergent.

##### ATTENTION

Do not use strong detergents, sharp objects or solvents for cleaning.

##### UTILIZATION

Municipal waste should be disposed of in accordance with the applicable disposal conditions.

##### Technical specifications

AT51406 (55cm): maximum user weight 170KG  
AT51407 (65cm): maximum user weight 170KG  
AT51408 (75cm): maximum user weight 180KG  
AT51409 (85cm): maximum user weight 180KG  
AT51410 (55cm): maximum user weight 170KG  
AT51411 (65cm): maximum user weight 170KG  
AT51412 (75cm): maximum user weight 180KG

This sign indicates the maximum weight of the user.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warszawa, ul. Zawiańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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Date of issue of the manual: August 31, 2022  
v1-3 08/31/2022