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Model:

Numer Serii:

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Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warszawa, ul. Zawiańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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SKŁAD: PVC

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Thank you for purchasing our product. Before using, please read this user manual carefully. Incorrect use may lead to health detriment, injury or accidents. Therefore, in order to use the product safely and successfully, please read these instructions carefully.

NOTE: Check the product for damage caused during transport. If any such damage is found, DO NOT use the product. More information from the manufacturer.

ATTENTION:

When a product-related "serious incident" occurs, which directly or indirectly led, or could lead to, any of the following events:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of a medical device, consult a healthcare professional.

NOTE: In the event of pain, great discomfort, or an allergic reaction, please contact a healthcare professional as soon as possible.

ATTENTION: The manufacturer is not responsible for damage caused by neglect of maintenance, inadequate servicing, or resulting from failure to comply with the recommendations contained in this manual.

ATTENTION: it is forbidden to use the product other than in accordance with its intended purpose

CAUTION: Flammable.

NOTE: The product may not be used by children under 3 years of age.

INTENDED USE: The product is intended for people who require rehabilitation or lead a sedentary lifestyle.

CONTRAINDICATIONS: Physical or mental limitations (e.g. visual impairment) that prevent the safe handling of the product

PRODUCT FEATURES:

- set of 2 air-filled balance balls
- spiky surface ideal for sensory stimulation
- Suitable for all ages and adapted to the user's skill level
- latex and phthalate free - suitable for people who are sensitive or allergic to these materials

APPLICATION:

- prevention of injuries such as ankle sprain
- flat feet,
- calcaneal spur,
- improves the ankle stability
- improves coordination and deep feeling
- increases the strength of the shoulder girdle

STORAGE

The product should be stored in a dry place, not exposed to direct sunlight.

MAINTENANCE

The pillow should be cleaned with a damp cloth with a mild detergent.

ATTENTION

Do not use strong detergents, sharp objects or solvents for cleaning.

UTILIZATION

Municipal waste should be disposed of in accordance with the applicable disposal conditions.

Technical parameters AT51424

- Diameter of the dome - 16cm
- Maximum user weight - 180 kg

This sign indicates the maximum weight of the user.

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