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EN

Introduction

Thank you for purchasing our product. Please read this user manual carefully before using it. Incorrect use may lead to damage to health, injury or accidents. Thus, in order to use the product safely and successfully, please read this manual carefully.

NOTE

Check all parts for damage caused during transit. If such damage is noticed, DO NOT use the product. Contact the manufacturer for more information.

ATTENTION

In the event of a device-related "serious incident" that has directly or indirectly led to, could have led to, or is likely to lead to any of the following:

- (a) death of the patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above 'serious incident' must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTE

In the event of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

NOTE

The manufacturer is not responsible for damage caused by negligent cleaning, improper use or as a result of not following the recommendations contained in these instructions for use.

ATTENTION

- Use of this product other than for its intended purpose is prohibited.
- Flammable product!
- Product not permitted for use by children under 3 years of age.

PURPOSE

The rehabilitation mat has many needles, which give the impression of a massage, which gives the feeling of complete relaxation and reduces various types of pain.

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

APPLICATION

- Back pain
- Sciatica
- Knee pain
- Muscle and joint pain
- Insomnia
- Headache
- Getting rid of fat and cellulite

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see the intended use section of this manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/sizes/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

USE

Place the mat and cushion on a flat, dry, clean and non-slip surface then lie on the mat and cushion for an appropriate amount of time. Do not hold your breath while lying on the mat. The duration of exercise on the mat should be agreed with your doctor.

NOTE (applies to product AT51431)

Before using the mat and cushion each time, remove the lavender bag and then heat it in the microwave for 5 seconds, place it back in the cushion.

STORAGE:

Store the product in a dry place, do not expose to direct sunlight.

CLEANING

Use a damp cloth with mild soap to clean.

Keep away from fire, heat sources, rough, sharp and abrasive surfaces.

Never use a damaged or worn product.

Caution: spikes may be sharp.

DISPOSAL

Dispose of municipal waste in accordance with the applicable disposal conditions.

DIMENSIONS:**AT51425**

Mat length: 75 cm

Width of mat: 44 cm

AT51426

Cushion length: 23 cm

Width of cushion: 10 cm

AT51431

Length of mat: 65 cm

Width of mat: 40 cm

Length of cushion: 36 cm

Width of cushion: 13 cm

DELIVERY RANGE

AT51425 : mat, manual

AT51426 : cushion, instructions

AT51431 : mat, cushion with lavender insert, manual

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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