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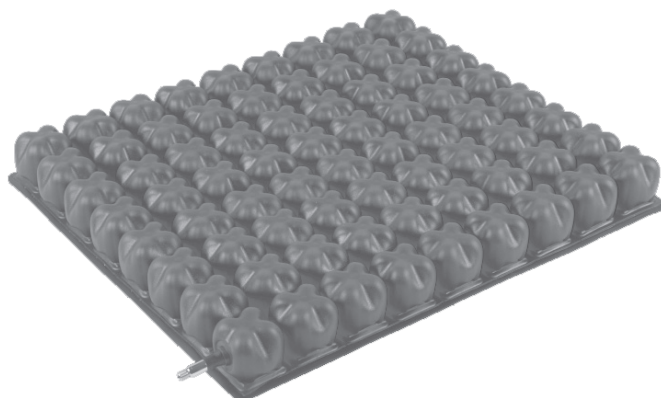
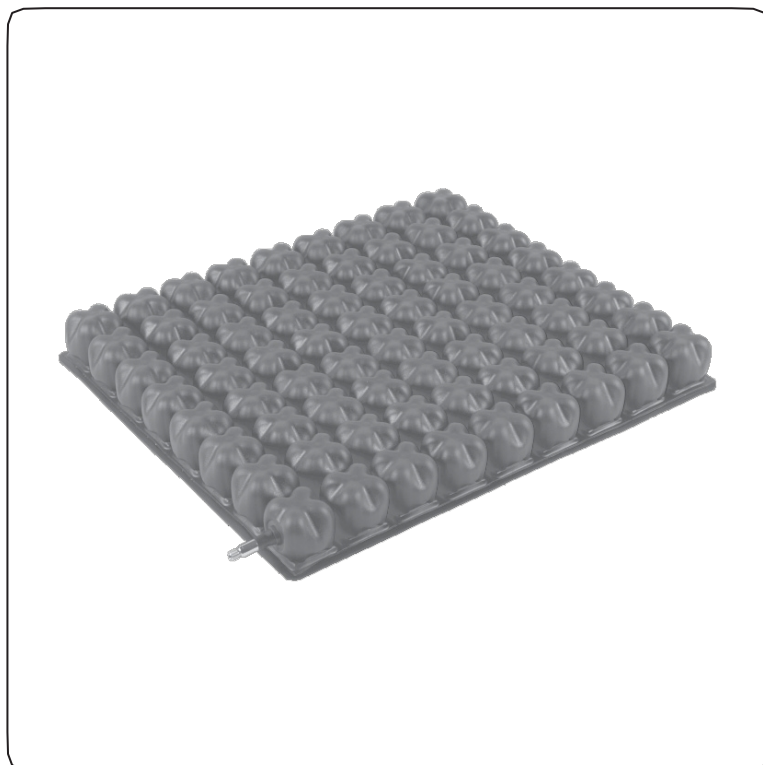
INSTRUCTIONS FOR USE
PNEUMATIC DECUBITUS CUSHION
AT52114

Important notes and warnings

Thank you for purchasing our product. The instruction manual contains instructions for the proper use of the product and valuable user safety tips. Please read the manual carefully before using the product. If you have any questions, please contact your dealer for advice and assistance. The product should be checked regularly for any damage that has occurred, in case of any defects, please stop using the product immediately.

GENERAL WARNING

Do not use the product before reading and understanding these instructions. If you do not understand the warnings, cautions or instructions, contact a health care professional or your dealer to avoid damage to the product.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

NOTE: In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration in the health of a patient, user or other person or
- c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices
The report is based on the report of the Committee on Biocidal Products and Biocidal Products.

CAUTION: Do not use needles, stitching tools or other sharp instruments in contact with the cushion.

NOTE: It is forbidden for children to use the pillow without adult supervision!

NOTE: It is forbidden to use the product other than for its intended purpose.

NOTE: Do not move the cushion (inflated/uninflated) when the patient is on it.

NOTE: Check the cushion daily (pressure check) to ensure that the user does not fall on it on a hard surface.

CAUTION: Flammable product. Keep away from sources of fire.

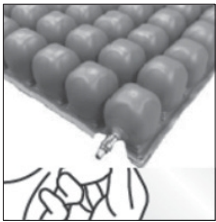
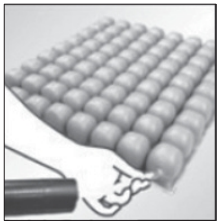
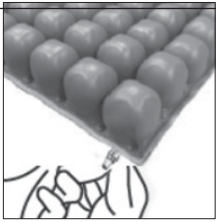
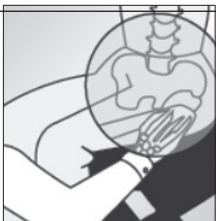
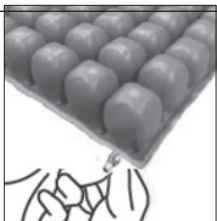
NOTE: If you experience pain, allergic reactions, or other worrisome, unclear symptoms related to the use of the medical device, consult a healthcare professional.

Product purpose/features:

The pillow provides comfort and stability by evenly dispersing the pressure of the hips, sitting on the pillow causes the user to sink into it. The pillow is designed for body contact use, especially for patients who have a high risk of developing pressure sores. Inflated chambers provide a more balanced and stable body position. The cushion is designed for the disabled, the elderly, office workers, drivers, people who spend a long time in a sitting position, etc.


Orthopedic Pillow AT52114 is a unique solution, combining all the advantages of traditional anti-decubitus pillows with great comfort. Unlike traditional pillows, it does not cause discomfort.

Rules for preparing the pillow for use/use:

	
1. open the cushion valve counterclockwise while holding it with the other hand, To prevent damage to the valve hose.	2 Attach the soft pump hose to the valve, holding the valve with the other hand so as not to damage the hose Valve, slightly inflate the cushion with a pump. valve hose.
	
3 Turn the valve clockwise to close.	4. after inflating, sit on the cushion in a natural position. Sitting with properly arranged- Wifed arms and feet.
	
5. the hand should be placed between the cushion and the user's tailbone to position the cushion In the right position.	6. slowly open the valve to let the air out. Close the valve when the distance between the widest point and the seat base is at least 2 fingers.

Specification/Scope of Delivery:

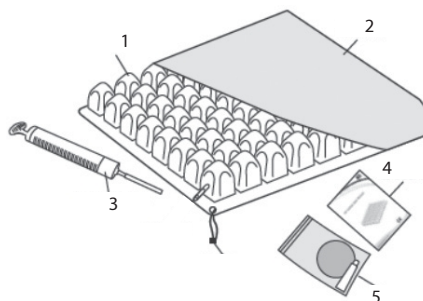- pillow (1)
- cover (2)
- inflator (3)
- instruction manual (4)
- repair kit (5)

Sizes:

40cm x 40cm x 8cm,
40cm x 45cm x 8cm,
45cm x 45cm x 8cm

Maximum load: 120 kg

1. Air chambers: made of medical-grade PVC. 9 rows of 9 air cells.
2. Sheath: made of polyester fabric, prevents the surface of the cushion from protruding beyond the seat and slipping off the cushion.
3. Hand pump: to fill the cushion with air.
- 3a. Vent: each pillow contains a valve, used to adjust the amount of air in the pillow, and thus the softness of the pillow.
4. Instruction manual: contains instructions for adjustment, maintenance and use.
5. Repair kit: two patches to repair the cushion in case of small holes.

Material: cushion:100% PVC, cover: 62% polyester, 38% PVC

THIS SIGN INDICATES THE USER'S MAXIMUM WEIGHT

Maintenance:

● **Pillow:** wash the **pillow** with warm soapy water at least once a week, rinse with lukewarm water and leave to dry or wipe with a towel, clean regularly, store in a dry place when not in use, keep away from sharp objects.

Note: Do not use bleach, mechanical cleaning, tumble drying, sun exposure, ironing. It is forbidden to use detergents and other solvents. Do not heat the pillow and do not use autoclave.

● **Cover:** Hand wash at up to 40°C, do not iron, close the zipper when cleaning, clean regularly, store in a cool and dry environment.

Note: Do not use bleach, mechanical cleaning, tumble drying, sun exposure, ironing. It is forbidden to use detergents and other solvents. Do not heat the pillow and do not use autoclave.

Note: Check the cushion daily (pressure check) to ensure that the user does not fall on it on a hard surface.

Note: The product and its accessories should be maintained in a dry environment.

Disposal: municipal waste, dispose of in accordance with the applicable disposal conditions.

Repairing leaks:

1. Make sure the cushion and all 4 corners are fully inflated.
2. Dip the inflated pillow in water to locate the leak.
3. If you locate a leak, mark it.
4. Remove the pillow and get rid of the air inside.
5. Allow the pillow to dry.
6. Place a patch over the leak.
7. Press the patch firmly to seal it well. Do not use the cushion for 30 min.
9. when you start using the cushion again, fill it with air following the instructions. Patches will not change the performance of the cushion, but if your patch is not satisfactory and has not solved the problem, or if you have more serious damage, contact the service center.

WARRANTY CARD

Model:

Serial Number:

Seal of the point of sale

and legible signature of Seller:

1) ANTAR Sp. J., 03-068 Warsaw, Zawisłańska 43 St., hereby provides a 12-month warranty on the pillow, starting from the date of purchase of the product by the Buyer. For products, the purchase of which is reimbursed by the National Health Fund, the warranty period is extended and covers half of the period of use specified in the Regulation on Guaranteed Services, provided that the period of use is specified in the Regulation in months or years.

2) During the warranty period, ANTAR undertakes to perform the repairs necessary to restore the proper functioning of the product, within a period of 14 days from the date of receipt of the product by the service along with the proof of purchase and the warranty card.

3) ANTAR reserves the right to replace the product if it is determined that the repair costs are not cost-effective. Repair or replacement of defective products does not extend the warranty period.

4) If the product is found to be malfunctioning, it should be sent immediately to ANTAR's address (given in Section 1) or contact the point of sale.

5) Before use, read the instructions carefully and follow their provisions. Use of the product contrary to its intended use and recommendations will result in irrevocable loss of warranty.

6) The warranty does not cover damage caused by external mechanical force, loading in excess of the user's maximum permissible weight as described in the manual, and contact of the product with alcohol, oily substances or gasoline. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.

7) The product submitted to the service for a complaint must not cause an epidemic.

8) THIS WARRANTY IS VALID ONLY WITH PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A CLAIM, THE PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO RECOGNIZE THE EXTENDED WARRANTY PERIOD FOR PRODUCTS THE PURCHASE OF WHICH WAS REIMBURSED BY THE NATIONAL HEALTH FUND, A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES THAT ARE ORTHOPEDIC OBJECTS AND AIDS MUST BE ATTACHED.

9) Claims submitted without proof of purchase and warranty card with product serial number will not be accepted.

ANTAR Sp. J.
03-068 Warsaw, 43 Zawisłańska St.
Poland
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net e-mail: antar@antar.net

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