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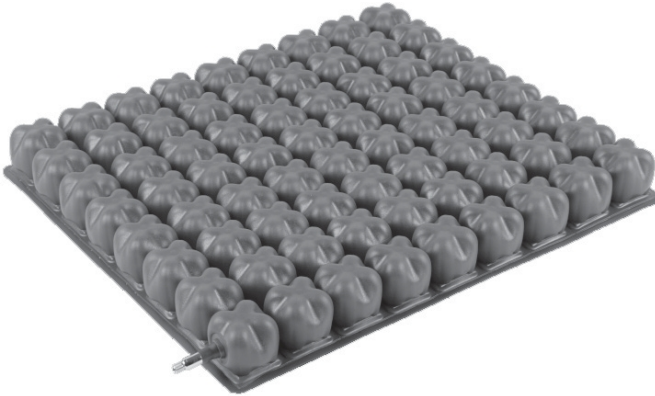
INSTRUCTIONS FOR USE
PNEUMATIC DECUBITUS CUSHION
AT52115

Important Notes and Warnings

Thank you for purchasing our product. The user manual contains instructions for the correct use of the product and valuable safety advice for the user. Please read the instructions carefully before using the product. If you have any questions, please contact your dealer for advice and assistance. The product should be checked regularly for any damage that may have occurred, and in the event of any fault, the product should be discontinued immediately.

GENERAL WARNING

Do not use the product until you have read and understood these instructions. If you do not understand the warnings, cautions or instructions, contact your healthcare professional or dealer to avoid damage to the product.

NOTE: Inspect all parts for damage caused in transit. If such damage is noticed, DO NOT use the product. Contact the manufacturer for more information.

NOTE: In the event of a product-related "serious incident" that directly or indirectly led to, could have led to, or is likely to lead to any of the following:

- a) death of a patient, user or other person, or
- b) a temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

this 'serious incident' should be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

CAUTION: Do not use needles, stitching tools or other sharp instruments in contact with the cushion.

NOTE: It is forbidden for children to use the cushion without adult supervision!

NOTE: It is forbidden to use the product in any other way than for its intended purpose.

NOTE: Do not move the cushion (inflated/uninflated) while the patient is on it.

NOTE: Check the cushion daily (pressure check) to ensure that the user does not fall on it on a hard surface.

CAUTION: Flammable product. Keep away from sources of fire.

NOTES:

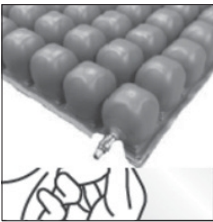
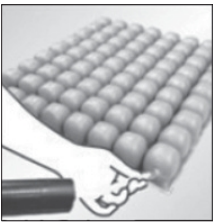
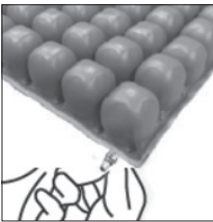
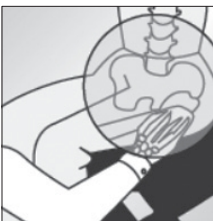
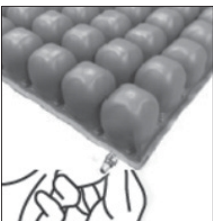
If you experience pain, allergic reactions or other distressing, unclear symptoms related to the use of the medical device, consult a healthcare professional.

Product purpose/features:

The cushion provides comfort and stability by dispersing hip pressure evenly, sitting on the cushion causes the user to sink into it. The cushion is designed for body contact use, particularly for patients who have a high risk of pressure sores. The inflated chambers provide a more balanced and stable body position. The cushion is suitable for people with disabilities, the elderly, office workers, drivers, people who sit for long periods of time, etc.

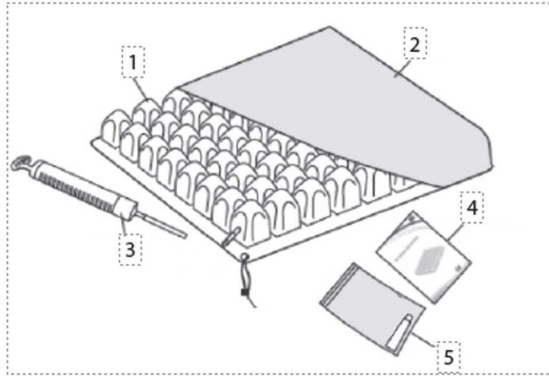
The Orthopaedic Pillow AT52115 is a unique solution, combining all the advantages of traditional decubitus pillows with great comfort. Unlike traditional pillows, it does not cause discomfort.

Principles of cushion preparation/use:

	
1. open the cushion valve counterclockwise, holding it with the other hand, to prevent damage to the valve hose.	2. attach the soft pump hose to the valve, holding the valve with the other hand so as not to damage the valve, slightly inflate the cushion with a pump. valve hose.
	
3 Turn the valve clockwise to close.	4. Once inflated, sit on the cushion in a natural position. Sit with the cushion properly positioned. with arms and feet.
	
5. the hand should be placed between the cushion and the user's tailbone to position the cushion in the correct position.	Slowly open the valve to let the air out. Close the valve when the distance between the widest point and the seat base is at least 2 fingers.

Specification/scope of supply:

- cushion (1)
- cover (2)
- inflator (3)
- instruction manual (4)
- repair kit (5)

Sizes:

40cm x 40cm x 5cm,
40cm x 45cm x 5cm,
45cm x 45cm x 5cm

Maximum load: 120 kg

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER

1. Air chambers: made of medical-grade PVC. 9 rows of 9 air cells.
2. Sheath: made of polyester fabric, prevents the surface of the cushion from protruding beyond the seat and slipping off the cushion.
3. Hand pump: to fill the cushion with air.
- 3a. Vent: each cushion contains a valve to regulate the amount of air in the cushion and thus the softness of the cushion.
4. User manual: includes instructions for adjustment, maintenance and use.
5. Repair kit: Glue to repair the cushion in case of small holes.

Material: Cushion:100% PVC, cover: 62% polyester, 38% PVC

Maintenance:

● **Pillow:** wash the pillow with warm soapy water at least once a week, rinse with lukewarm water and allow to dry or wipe with a towel, clean regularly, store in a dry place when not in use, keep away from sharp objects.

Note: Do not use bleach, mechanically clean, tumble dry, expose to sun, iron. The use of detergents and other solvents is prohibited. Do not heat the cushion and do not use an autoclave.

● **Cover:** Hand wash at up to 40°C, do not iron, close the zip when cleaning, clean regularly, store in a cool and dry environment.

Note: Do not use bleach, mechanically clean, tumble dry, expose to sun, iron. The use of detergents and other solvents is prohibited. Do not heat the cushion and do not use an autoclave.

Note: Check the cushion daily (pressure check) to ensure that the user does not fall on it on a hard surface.

Note: The product and its accessories should be maintained in a dry environment.

Disposal: Municipal waste, must be disposed of in accordance with the applicable disposal conditions.

Repairing leaks:

1. Make sure the cushion and all 4 corners are fully inflated.
2. Dip the inflated cushion in water to locate the leak.
3. If you locate a leak, mark it.
4. Remove the cushion and get rid of the air inside.
5. Allow the cushion to dry.
6. Shake the glue bottle before opening it.
7. Fill the leaking area with glue, but not too much.
8. Allow the glue to flow inside the leaking area.
9. Smooth the surface of the repaired area so that it is even and free of burrs.
10. Leave it for 12 hours to dry completely.
11. When you start using the cushion again, inflate it according to the instructions.

WARRANTY CARD

Model:

Serial Number:

Seal of the point of sale

and legible signature of the Seller:

1) ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby provides a 12-month warranty for the cushion, starting from the date of purchase of the product by the Purchaser. For products, the purchase of which is reimbursed by the National Health Fund, the warranty period is extended and covers half of the period of use specified in the Regulation on Guaranteed Services, provided that the period of use is specified in the Regulation in months or years.

2) During the warranty period, ANTAR undertakes to carry out the repairs necessary to restore the proper functioning of the product, within a period of 14 days from the date of receipt of the product by the service, together with the proof of purchase and the warranty card.

3) ANTAR reserves the right to replace the product if it is considered that the repair costs are not cost-effective. Repair or replacement of defective products does not extend the warranty period.

4) If the product is found to be malfunctioning, it should be sent immediately to the ANTAR address (given in point 1) or contact the point of sale.

5) Read the instructions carefully before use and follow the instructions. Use of the product contrary to its intended use and recommendations will result in irrevocable loss of warranty.

6) The warranty does not cover damage caused by external mechanical force, loading in excess of the user's maximum permissible weight as described in the instructions, or contact of the product with alcohol, oily substances or petrol. The guarantee does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.

7) A product submitted for service in connection with a complaint must not cause an outbreak.

8) THIS GUARANTEE IS ONLY VALID IF ACCOMPANIED BY PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN THE EVENT OF A CLAIM, THE PROOF OF PURCHASE MUST BE ENCLOSED. IN ORDER TO RECOGNISE THE EXTENDED WARRANTY PERIOD FOR PRODUCTS WHOSE PURCHASE WAS REIMBURSED BY THE NATIONAL HEALTH FUND, A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES THAT ARE ORTHOPAEDIC AND AUXILIARY EQUIPMENT MUST BE ATTACHED.

9) Complaints submitted without proof of purchase and a warranty card with the product serial number will not be accepted.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43 Poland
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net e-mail: antar@antar.net

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