AT52301

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INTRODUCTION

Before using your wheelchair, please read this user manual carefully. Incorrect use of the wheelchair may lead to health damage, injury or accidents. Thus, in order to use the wheelchair safely and successfully, please read this manual carefully.

SAFETY

The safest operation of the product is on a flat surface free of any obstacles. If possible, always use ramps when entering or leaving a building. Keeping your hands on the grip rims allows maximum balance.

Leaning

Do not lean over the side of the wheelchair as this can lead to loss of balance and falling.

Moving up hills

- Do not turn while moving on hills because of the risk of tipping over.
- The attendant should not push the wheelchair on hills diagonally.
- If the wheelchair user is going up a hill on his own, he should avoid moving on the hill diagonally.
- If it is necessary to go down a steep hill, the attendant should walk backwards with the wheelchair above him.
- If the user is going down a hill on his own, he should control the speed of the wheelchair at all times and not allow the product to overspeed. If necessary, another person should provide an assistance.

Kerbs

- If possible, always be assisted by a carer (if moving with a carer, the wheelchair must be provided with push handles).
- Avoid strong shocks when moving down off the kerb. Shocks can damage the product.
- Do not under any circumstances attempt to move onto high kerbs.
- Always exercise caution when passing kerbs or making turns on hills and never move backwards on a wheelchair without assistance from a carer.

Stairs

- Always have the assistance of two or more carers. Tilt the product slightly backwards for balance.
- Do not climb the stairs with the wheelchair.
- Do not use escalators. Use a lift.

General information

- Always check that BOTH brakes are blocked when getting in or out of the wheelchair.
- Do not stand on the footrests when getting on or off the product.
- It is not permitted to carry more than one passenger. Wheelchairs are designed for one user.
- Please take care that clothing does not block the wheels of the wheelchair.

Upholstery

- Use warm soapy water to clean the upholstery.
- Upholstery cleaner/foam can also be used, but avoid using solvents.
- Keep the wheelchair away from sources of fire that may cause injury to the user and damage the product.

CAUTION:

In the event of a device-related 'serious incident' that has directly or indirectly led to, could have led to or may lead to any of the following:

- (a) death of the patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above 'serious incident' must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In the event of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

ATTENTION: The manufacturer cannot be held responsible for damage caused by negligent maintenance, inadequate servicing or as a result of not following the instructions in this manual.

ATTENTION: It is forbidden to use the product other than in accordance with its intended use!

ATTENTION: During use and handling of the product and when assembling and adjusting the mechanisms, there may be a danger of entrapment and/or compression of parts of the user's/companion's body in openings/gaps between components. Perform these operations with particular care.

NOTE: There may be a risk of tipping over if the product is used incorrectly. Please follow the instructions for getting on/off/moving. Once the adjustment is complete, stabilise the position by tightening the nuts/bolts carefully.

PURPOSE / INDICATIONS

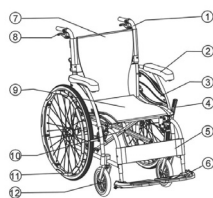
The wheelchair is intended for people with disabilities or mobility difficulties. It serves as a support during mobility.

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling.

GENERAL WARNINGS

- When getting on or off the wheelchair, make sure that both brakes are blocked.
- If the wheelchair is not being pushed or used make sure that both brakes are blocked.
- Avoid steep and uneven surfaces as this can cause the product to tip over.
- Avoid soft surfaces as the wheels may collapse.
- Move the wheelchair in a smooth manner in all conditions.
- Always observe your surroundings to identify hazards, such as furniture, doors, holes, hills, kerbs, entrances, etc.
- Always keep both feet on the footrests and ensure clothing is properly tucked and not blocking the heels.
- Always move in a careful manner on hills.
- Always keep both hands on the push handles for proper control of the wheelchair (applies to the carer).
- Always remember to maintain balance.
- Check the frame and wheelchair components regularly and always follow the instructions in this manual during and after the warranty period.
- Keep the wheelchair away from sources of fire.
- Do not, under any circumstances, overload the product.
- Do not hang any heavy objects on the back of the wheelchair as this may affect the stability of the product, particularly when climbing hills.
- The wheelchair may only be used by one person.
- Do not use the wheelchair when the patient is unconscious.
- Do not use other means of transport to push the wheelchair.
- Do not lift the front wheels while the wheelchair is in motion, as this may result in a fall and serious injury or death.
- Do not get into the wheelchair while it is moving.
- Do not move the wheelchair too fast and do not make sudden turns.
- Do not move the wheelchair across a hill (in either direction) with a gradient of more than 7 degrees.
- Do not stand on the footrests when getting on or off the product.
- Do not climb stairs using a wheelchair, either alone or with a carer.
- Do not attempt to lift the wheelchair with moving (removable) parts. Lifting with moving parts of the product may lead to injury to the user and/or carer or damage to the product.
- Do not use the wheelchair if the backrest is folded. This changes the centre of gravity of the wheelchair and may cause the wheelchair to tip backwards. The backrest should only be folded when storing or transporting the wheelchair.
- No objects should be hung on the backrest-folding connector as this may cause the backrest to fold down and cause serious injury to the user.

1. push handle
2. armrest
3. side panel
4. brake
5. calf rest strap
6. footrest base
7. backrest upholstery
8. additional brake
9. seat upholstery
10. wheelchair crossbar
11. rear wheel
12. front wheel

Pic.1

TECHNICAL PARAMETERS:

Width (steel wheelchair): 66 cm
 Width of seat: 46 cm
 Width when folded: 26 cm
 Seat depth: 40 cm
 Backrest height: 42 cm
 Maximum height: 94 cm
 Height with backrest folded: 66 cm
 Front wheel (cast): 8"
 Rear wheel (cast): 22"
 Weight of wheelchair: 13.40 kg
 Maximum load: 115 kg

THIS SIGN MEANS THE MAXIMUM USER'S WEIGHT

Equipment: belt for the patient, brakes for the carer, foldable backrest, soft armrests, pocket, reclining footrests (3 degrees of adjustment)

USE OF THE PRODUCT

- Preparing the wheelchair for use

Folding out

- Hold the handle and unfold the rear connector until a distinctive sound can be heard. (See figure 1.)
 - Holding the armrests, pull outwards (See Fig. 2).
 - Pointing your fingers inwards while keeping your hand on the frame push downwards until the seat lowers completely. Make sure the frame is securely locked in the handle. (See figure 3). Alternatively, stand at the side of the wheelchair and carry out step 1 by pulling the nearest armrest towards you pushing the other armrest in the opposite direction. To fold the product proceed in the reverse order.

Pic. 2


Pic. 3

Pic. 4

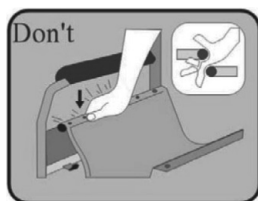When carrying out step 2, do not place your fingers on the sides of the product or put them under the seat, as they could be pinched between the seat and the frame.

Pic. 5

MOVING THE WHEELCHAIR

Grasp the grip rims around the outside of the large wheels at the front of the wheelchair and pull them towards you to make the wheelchair move. To release/brake, grasp the rim at the rear and hold gently. Caution, if the wheelchair moves quickly, injuries or burns to the hands (due to friction) may occur. The use of protective gloves is recommended if the wheelchair is used intensively.

INSPECTION AND MAINTENANCE

Daily inspection

Before driving, check the following items. If any damage is found, contact your dealer for further inspection of the product before use.

- Check brakes for wear or damage. Replacement or adjustment of brakes should be carried out by a professional service.
- Check that the hand grips for the carer are not damaged and securely attached.
- Check that all bolts and nuts are tight before using the wheelchair. Check all parts for wear and replace if necessary. Check the alignment of all parts. Check the brakes to make sure they work properly.
- Wheels and tyres should be checked periodically for cracks and wear and replaced if necessary by a professional service.

CLEANING

To prolong the life of your wheelchair, it is recommended that you clean it regularly (preferably daily) especially if it is used in rainy or snowy weather. We recommend that the product is not used in rain or snow.

Wipe with a damp cloth.

Upholstery

- Use warm, soapy water to clean the upholstery.
- Upholstery cleaning products/foam can also be used, but avoid solvents and strong detergents.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

DISPOSAL OF THE PRODUCT

The product must be disposed of in accordance with the applicable disposal conditions.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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