AT52302

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Before using the wheelchair, please read this user manual carefully. Incorrect use of the wheelchair can lead to health disorder, injury or accidents. Therefore, in order to use your wheelchair safely and successfully, please read this manual carefully.

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

GENERAL WARNINGS

- When entering or leaving the wheelchair, make sure that both brakes are applied.
- When the wheelchair is not being pushed or used, make sure both brakes are applied.
- Be careful when approaching curbs and making turns on a slope.
- Steep and uneven surfaces should be avoided as this may cause the wheelchair to tip over.
- Avoid soft surfaces as the wheels may collapse.
- Move the wheelchair smoothly under all conditions.
- Always observe your surroundings to identify hazards, such as furniture, doors, potholes, slopes, curbs, entrances, etc.
- Always keep both feet on the footrests and ensure that clothing is properly tucked away and does not block the wheels.
- Always move carefully uphill.
- Always keep both hands on the push handles to control the wheelchair properly (for the assistant).
- Always remember to keep your balance.
- Regularly inspect the frame and components of your wheelchair, and always follow the instructions set in this manual during and after the warranty period.
- Keep the wheelchair away from fire sources
- Store the wheelchair away from fire sources.
- Under no circumstances should the wheelchair be overloaded.
- Do not hang any heavy objects on the back of the wheelchair as this may affect the stability of the wheelchair, especially when climbing slopes.
- Only one person may use the wheelchair.
- Do not use the wheelchair while the patient is unconscious.
- Do not use any other means of transport to push the wheelchair.
- Do not raise the front wheels while driving, as this may cause a fall and serious injury.
- Do not get into the wheelchair while it is in motion.
- Do not move the wheelchair too fast and do not make sudden turns.
- Do not drive the wheelchair across a slope (in any direction) greater than 7 degrees.
- Do not stand on the footrests when entering or leaving the wheelchair.
- Do not use the trolley to climb stairs, either alone or with an attendant.
- Do not attempt to lift the wheelchair with moving (dismountable) parts. Lifting with the moving parts of the wheelchair can lead to injury to the user and / or caregiver or damage to the wheelchair.
- The wheelchair must not be used with the backrest folded down. This changes the center of gravity of the wheelchair and may cause the wheelchair to tip over backwards. The backrest should only be folded down when storing or transporting the wheelchair.
- Do not hang any objects on the rear connector as this may cause the backrest to fold down and seriously injure the user.

SECURITY

Entrances, doors

The safest way to operate the wheelchair is on a horizontal floor free from any obstacles. Whenever possible, use the ramps when entering or exiting the building. Keeping your hands on the push handles allows you to maintain maximum balance. Do not force yourself to avoid obstacles by driving, and never use the side of the door to push yourself away.

Leaning over

Do not lean out to the side of the wheelchair as this may cause you to lose your balance and fall.

Navigating hills

- Do not turn on slopes because of the risk of tipping over.
- When assisting, go straight up. If you need to go downhill, walk backwards until the user is on level ground.

Curbs

- If possible, always use the help of an attendant (in the event of moving with the attendant, the stroller must be equipped with handles for pushing).
- Avoid heavy jolting when driving off the curb. Shocks may adversely affect the wheelchair.
- Under no circumstances should you attempt to climb over the curb above the normal height.
- Always exercise caution when passing curbs or turning on a hill, and never reverse the wheelchair without the assistance of an attendant.

Stairs

- Two or more attendants should always be used. Tilt the wheelchair to the point of balance. One of the attendants (at the back) holds the wheelchair on the first step with his arms firmly raised. The second attendant, holding the fixed part of the front frame, lifts the wheelchair above the steps, while the first attendant places one foot on the next step and repeats the action.
- It is not allowed to move on stairs in forward direction.
- Do not use escalators. Use an elevator.

General information

- Always make sure that BOTH brakes are applied when entering or exiting the wheelchair.
- Do not stand on the footrests when entering or exiting the wheelchair.
- It is not allowed to transport more than one passenger. Wheelchairs are intended for a single user.
- Please make sure that clothes do not block the wheels of the stroller.

Upholstery

- Use warm soapy water to clean the upholstery.
- Upholstery cleaner / foam can also be used, but avoid the use of solvents.
- The wheelchair must be kept away from fire sources that may injure the user and damage the wheelchair.

WARNING:

In case a serious incident that has occurred in relation to the medical device, that directly or indirectly led, might have led or might lead to any of the following:

- a) the death of patient, user or other person,
- b) the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- c) a serious public health threat;

the above mentioned serious incident should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is resident;

WARNING:

In case any pain, allergic reaction or other worrying symptoms, unclear for the user which have occurred in relation to the medical device, the user should consult a healthcare professional;

ATTENTION: It is forbidden to use the product other than in accordance with its intended purpose

NOTE: Before use, check that all components are properly assembled.

ATTENTION: The manufacturer is not responsible for damage caused by neglect of maintenance, inadequate servicing or resulting from failure to comply with the recommendations contained in this manual.

NOTE: There may be a risk of tipping over if the product is used improperly. Please follow the boarding / dismounting / moving instructions.

ATTENTION: During use and handling of the product, as well as during its assembly and adjustment of mechanisms, there may be a risk of entrapment and / or pinching of body parts of the user / accompanying person in the openings / gaps between the elements. These steps must be performed with extreme caution. After completing the adjustment, stabilize the position by tightening the nuts / bolts carefully

NOTE: It is forbidden to stand on the footrests as this can lead to a dangerous fall and injury.

Composition:

Frame: aluminum

Seat: polyurethane, polyethylene; 1.5 cm thick

Armrests: polyurethane; 3 cm thick

Backrest: polyurethane, polyethylene; 2 cm thick

Wheels: PU

Preparing for use

Hold the handle and unfold the rear connector until you hear a distinctive sound. Holding the armrests, pull them away from you. With your fingers inward, keeping your hand on the frame, push downward until the seat is fully lowered. Make sure the frame is firmly seated in the holder.

Use

1. Unfolding

Lay the wheelchair on its side, then insert the quick release wheel to remove the axle, do the same with the other wheel. (fig. 1)

Place the wheelchair in front of you, push it outwards against the armrests. Press with your hands from above on the side struts of the seat to unfold them completely. Make sure that the sides of the seat are resting firmly on the plastic joints. Otherwise, components may be damaged. (fig. 2)

Then mount the footrests on the sides, fold them forward until they stop in the right place. (Figures 3 and 4)

Fig. 1

Fig. 2

Fig. 3

Fig. 4

2. Folding

Fold the footrest platforms upwards, grasp the front and back parts of the seat and pull them upwards (fig. 5 & 6)

Fig. 5

Fig. 6

3. Side panels

To facilitate access to the wheelchair from the side, the armrest can be folded up or removed completely by pressing the release levers and lifted upwards. (figs. 7 and 8)

By pulling the rear locking levers, the side part can be completely removed (fig. 9 and 10). To reattach the armrest, proceed in the reverse order.

Fig. 7

Fig. 8

Fig. 9

Fig. 10

4. Tilting aid

The lever on the rear wheel makes it easier for the accompanying person to pass through small obstacles. Press down on the lever with your foot, hold the handles with your hands and slide the wheelchair over the obstacle. (fig. 11 and 12)

Fig. 11

Fig. 12

5. parking brake

The wheelchair has a parking brake on both sides. On both sides, press the black brake levers forward until the brake bolts press the tires. (Fig. 13) To release the brake, pull the lever back up. (fig. 14)

Fig. 13

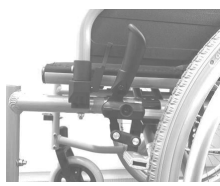
Fig. 14

Settings

1. Seat height (back)

The height of the seat can be adjusted (6 levels of adjustment every 2.5 cm). Remove the rear wheel, use a spanner to loosen the nut on the inside. Remove the rear wheel holder and insert it to the desired height. Tighten the screw.

Fig. 15

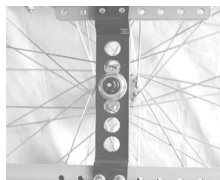
Fig. 16

2. Height of the front wheels

The height can be adjusted (5 degrees of adjustment every 1 cm). Loosen the screw with an Allen key and a flat wrench, and then pull it out of the holder. Adjust to the desired height and tighten the screw. Make sure both wheels are at the same height.

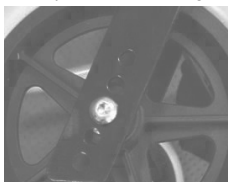
Fig. 17

Fig. 18

3. Seat inclination angle

By adjusting the height of the front and rear wheels, you can adjust the angle of the seat by changing the position of the wheels downwards and the inclination of the seat is increased.

4. The steering angle of the wheels

To prevent the front wheels from shaking after changing the angle of the seat, adjust the steering angle of the wheels. For this, the front wheel holder must be positioned exactly at a 90 ° angle. Loosen the bracket bolts with two flat wrenches, position the bracket perpendicular to the floor, and reattach the bolts.

Fig. 19

Fig. 20

5. Seat depth

The depth of the seat can be adjusted (6 steps every 2.5 cm). After changing the seat depth, the seat upholstery should be properly adjusted with Velcro.

Fig. 21

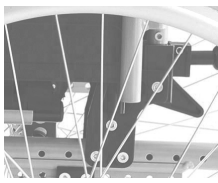
Fig. 22

Fig. 23

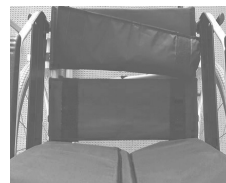
Fig. 24

6. Leg support

The length of the footrest can be adjusted (5 steps every 2.5 cm). Loosen the hexagonal bolt, remove it from the pipe. Then push the tube with the footrest to the desired length of the appropriate hole. Tighten the screw.

Fig. 25

Fig. 26

7. Footrests

The footrests are foldable and removable, they can be adjusted at an angle. Loosen the hexagonal socket screw, adjust the angle and tighten the screw.

Fig. 27

Fig. 28

8. Rear wheels

The wheels can be adjusted backwards or forwards (7 steps every 2.5 cm).

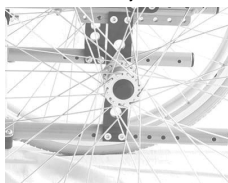
Fig. 29

Fig. 30

9. Backrest

The backrest can be adjusted in the upholstery by means of Velcro fasteners. (5 steps every 2.5 cm).

Fig. 31

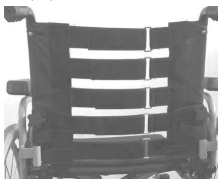
Fig. 32

Technical data:

Size 41 (cm)	Size 43 (cm)	Size 46 (cm)	Size 49 (cm)
Total length: 101 cm	Total length: 101 cm	Total length: 101 cm	Total length: 101 cm
Total height: 92 cm	Total height: 92 cm	Total height: 92 cm	Total height: 92 cm
Seat depth: 38-53 cm	Seat depth: 38-53 cm	Seat depth: 38-53 cm	Seat depth: 38-53 cm
Height from the ground to the seat: 43-55.5	Height from the ground to the seat: 43-55.5	Height from the ground to the seat: 43-55.5	Height from the ground to the seat: 43-55.5
Maximum payload: 120 kg	Maximum payload: 120 kg	Maximum payload: 120 kg	Maximum payload: 120 kg
Seat width: 41 cm	Seat width: 43 cm	Seat width: 46 cm	Seat width: 49 cm
Overall width: 61.5 cm	Overall width: 63.5 cm	Overall width: 65.5 cm	Overall width: 68 cm
Folded width: 32	Folded width: 32	Folded width: 32	Folded width: 32
Backrest height: 48-38 cm	Backrest height: 48-38 cm	Backrest height: 48-38 cm	Backrest height: 48-38 cm
Front wheels: cast wheels, 8"	Front wheels: cast wheels, 8"	Front wheels: cast wheels, 8"	Front wheels: cast wheels, 8"
Rear wheels: cast wheels, 24"	Rear wheels: cast wheels, 24"	Rear wheels: cast wheels, 24"	Rear wheels: cast wheels, 24"
Net weight: 15.3 kg	Net weight: 15.8 kg	Net weight: 16.3 kg	Net weight: 16.8 kg
Gross weight: 17.3 kg	Gross weight: 17.8 kg	Gross weight: 18.3 kg	Gross weight: 18.8 kg

This sign indicates the maximum weight of the user

Inspection and maintenance

Before driving, be sure to check the following items. If any malfunctions are found, contact your dealer for an additional inspection of your wheelchair before it is used.

- Check the brakes for wear and damage. Brake replacement or adjustment should be performed by an authorized service center.
- Check that the handgrips are undamaged and securely attached.
- Before using the cart, check that all bolts and nuts are tightened. Check all parts for wear and replace if necessary. Check the alignment of all parts. Check the brakes for proper operation.
- Wheels and tires should be checked periodically for cracks and wear and, if necessary, replaced with new ones by an authorized service center.

Cleaning: To prolong the life of your stroller, it is recommended that you clean it periodically (preferably daily) especially if it is used in rainy or snowy weather. We recommend that you do not use the stroller in rain or snow.

If you accidentally encounter rain while moving the stroller, it is advisable to find shelter. If there is no possibility of finding shelter, proceed to the destination point.

Frame: Wipe clean with a damp cloth and a mild detergent.

Upholstery: To clean the upholstery, use warm soapy water, do not use solvents.

Shelf life

Based on the tests performed, the manufacturer indicates a 5-year shelf life of the medical device.

ATTENTION: When using and operating the wheelchair, there may be a risk of entrapment and / or pinching of body parts of the user / companion in openings / gaps between the components. These steps must be performed with extreme caution.

SAFE DISPOSAL OF THE MEDICAL DEVICE

When the device has been withdrawn from use it can be treated as municipal waste except for electric medical devices – in such case the electric medical devices shall be disposed in the way described by the law for electrical and electronic waste disposal.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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