AT52307

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ENG

ENTRY

Before using the wheelchair, please read this user manual carefully. Incorrect use of the wheelchair can lead to personal injury, injury or accidents. Therefore, for the safe and successful use of your wheelchair, please read this manual carefully.

SECURITY

The safest handling of the product is on a level surface free of any obstacles. Whenever possible, always use ramps when entering or exiting a building. Keeping your hands on the push handles allows for maximum balance.

BENDING

Do not lean to the side of the wheelchair as this may cause you to lose your balance and fall.

MOVING ON HILLS

- Do not make turns on slopes due to the risk of overturning
- The assisting person should not push the wheelchair on slopes diagonally.
- If a wheelchair user climbs a hill on his own, he should avoid driving on the diagonal of the hill.
- If it is necessary to go down a steep slope, the assisting person should walk backwards with the wheelchair above them.
- If the user descends the slope by himself, he should control the speed of the wheelchair at all times and not allow the product to accelerate excessively. Seek help from another person if necessary.

CURBS

- If possible, always use the help of an attendant (when moving with an attendant, the wheelchair must be equipped with push handles).
- Avoid strong shocks when driving down the curb. Shock can have an adverse effect on the product.
- Under no circumstances should you attempt to climb high curbs.
- Always be careful when going over curbs or making turns on hills, and never walk backwards in your wheelchair without the help of an attendant.

STAIRS

- Always use two or more guardians. Tilt the product slightly backwards to maintain balance.
- It is not allowed to climb stairs while going forward.
- Do not use escalators. You must use the elevator.

GENERAL INFORMATION

- Always check that BOTH brakes are applied when getting in or out of the wheelchair.
- Do not stand on the footrests when getting on or off the product.
- It is not allowed to carry more than one passenger. Wheelchairs are designed for one user.
- Please make sure that clothing does not block the wheels of the wheelchair.

ATTENTION:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to, any of the following:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration of the patient's, user's or other person's health or
- (c) a serious threat to public health

this "serious incident" should be reported to the manufacturer and to the competent authority of the Member State where the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of the medical device, consult a healthcare professional.

NOTE: The manufacturer is not responsible for damage caused by neglected maintenance, improper servicing or failure to comply with the recommendations contained in this manual.

ATTENTION: it is forbidden to use the product in a way other than for its intended purpose!

ATTENTION: During the use and handling of the product, as well as during its assembly and adjustment of mechanisms, there may be a risk of entrapment and/or squeezing of the user's/assistant's body parts in the openings/gaps between the elements. These steps should be performed with particular care.

NOTE: If the product is used incorrectly, there may be a risk of tipping over. Please follow the boarding/dismounting/moving instructions. After adjustment, stabilize the position by carefully tightening the nuts/bolts.

INTENDED USE / INDICATIONS

The wheelchair is intended for people with disabilities or mobility difficulties. It serves as support while moving. Also recommended for the elderly.

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

GENERAL WARNINGS

- When getting in or out of the wheelchair, make sure that both brakes are engaged.
- Ensure that both brakes are engaged when the wheelchair is not being pushed or used.
- Steep and uneven surfaces should be avoided as this may cause the product to tip over.
- Avoid soft surfaces as the wheels may sink in.
- Move the wheelchair smoothly in all conditions.
- Always observe your surroundings for hazards such as furniture, doors, holes, elevations, curbs, entrances, etc.
- Always keep both feet on the footrests and ensure that clothing is properly stowed and does not block the wheels.
- Always drive carefully on inclines.
- Always keep both hands on the push handles to properly control the wheelchair (carer related).
- Always remember to keep your balance.
- Check the wheelchair frame and components regularly and always follow the instructions in this manual during and after the warranty period.
- Keep the wheelchair away from sources of fire.
- Under no circumstances should the product be overloaded.
- Do not hang any heavy objects on the back of the wheelchair, as this may affect the stability of the product, especially when climbing hills.
- The wheelchair may only be used by one person.
- Do not use the wheelchair when the patient is unconscious.
- Do not use other means of transport to push the wheelchair.
- Do not lift the front wheels while driving, as this may result in a fall and serious injury or death.
- Do not get into the wheelchair while moving.
- Do not move the wheelchair too fast or make sudden turns.
- Do not move the wheelchair across a slope (in any direction) steeper than 7 degrees.
- Do not stand on the footrests when getting on or off the product.
- Do not use a wheelchair to climb stairs, either alone or with an attendant.
- Do not attempt to lift the wheelchair with moving (removable) parts. Lifting with the moving parts of the product may lead to injury to the user and/or the caregiver or damage to the product.
- Do not use the wheelchair with the backrest folded down. This changes the wheelchair's center of gravity and may cause the wheelchair to tip over backwards. The backrest should only be folded down when storing or transporting the wheelchair.
- Do not hang any objects on the rear connector, as this may cause the backrest to fold down and cause serious injury to the user.
- The wheelchair is only suitable for indoor use or on level outdoor surfaces. The device cannot be used for prolonged driving on uneven roads. If you need to navigate slopes, steep terrain and steps, please ask for assistance from the guardian. If it is necessary to overcome an obstacle, the caregiver should grip the wheelchair handles with both hands, and then press the rear pedal of the wheelchair so as to lift the cat's front, which allows safe overcoming of the obstacle. When driving on inclines: When going downhill, avoid pushing down on the wheelchair. Maneuver the wheelchair using your body position to control speed and smoothness. Do not drive on inclines greater than 15 degrees.
- For people with a higher body weight, it is recommended to use the reverse driving method on inclines. Reverse driving allows safe speed control, thus ensuring the safety of the user. In the case of overcoming particularly steep slopes, you should use the help of a guardian. Raise the wheelchair by pressing on the rear foot, tilt back to lift the front wheels, and then apply gentle force to push the wheelchair up the hill. The wheelchair can also be lifted by two people - one supporting at the front and the other holding the handles at the back. Thanks to this, it is possible to overcome obstacles backwards.

COMPONENT DESCRIPTION

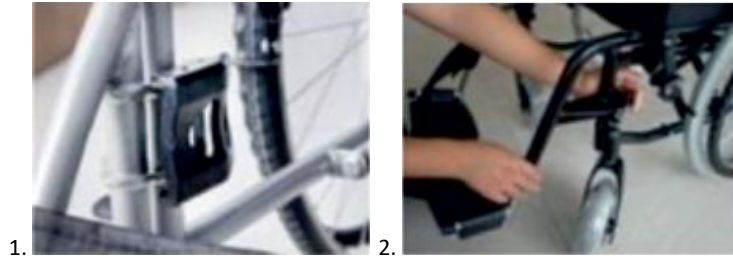
1. Backrest
2. Armrest
3. Detachable footrest
4. Footplate
5. Front wheel
6. Rear wheel
7. Drive rim
8. Handbrake
9. Side cover
10. Handle with an additional brake

USER MANUAL

1. Unfolding

In order to unfold the wheelchair, open the backrest until you hear a characteristic sound, then press the movable sides of the seat with both hands until the wheelchair unfolds completely. Finally, place the footrests in the corresponding holes on the frame, rotate inwards until the footrest is locked.

2. Folding

Fold the footrests first. Then fold the backrest and finally grab the center of the seat with both hands and pull it up until it is completely folded.

3. Footrest height adjustment.

Loosen the locking screw with an Allen key, unscrew the screw, set the desired height of the footrest, then tighten the screw. (fig. 3 and 4)

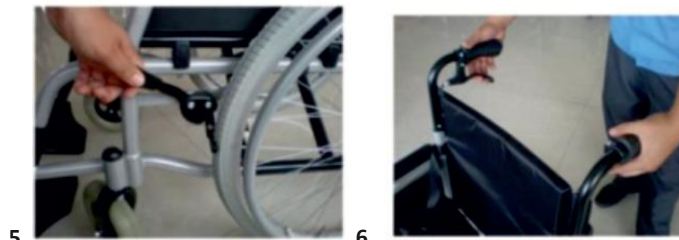
BRAKES

Remember to always use the brakes when getting in and out of the wheelchair. Hold the brake handle and move it forward or backward. Getting on and off the wheelchair is only allowed when the wheels are locked.

Note: Remember not to use the brake when moving on an incline due to the risk of the wheelchair tipping over. (Fig.5)

When using the handbrake, hold the brake lever with both hands and apply sufficient pressure to control the braking force and adjust the speed of the wheelchair. (Fig.5)

The wheelchair is equipped with brakes for the caregiver (fig.6)

QUICK REMOVAL FUNCTION OF THE BACKREST, WHEELS AND INSTRUCTIONS FOR THE SAFETY BELT

- The wheelchair is equipped with a folding backrest function. To do this, fold the backrest down by pressing the handles with both hands. (fig.7)
- To remove the wheel, hold it, press the black quick release button on the frame and remove the wheel. (fig. 8). Press the button to lock the wheel again. Always check the position of the locks to confirm that the wheels are correctly fitted.
- Adjust the seat belt to the user and fasten the buckle. (fig.9)

TERMS OF USE OF THE WHEELCHAIR

1. It is recommended to use the wheelchair only on flat surfaces outdoors or indoors. Avoid using the wheelchair outdoors in adverse weather conditions. If you need to use your wheelchair in bad weather, we recommend that you seek shelter as soon as possible.
2. Avoid bad roads such as muddy, snowy, sandy, gravel and uneven roads. Avoid driving on roads with heavy traffic and exposing the wheelchair to sunlight for long periods of time as tire damage may occur.
3. Remember to strictly comply with traffic regulations and maintain safety when using the wheelchair on public roads.
4. Remember to walk on sidewalks and use pedestrian crossings.
5. It is recommended to store and use the stroller in normal weather conditions. The permissible temperature range is -15°C ~ 40°C, and the relative humidity is 60 ± 35% rh.

SPECIFICATION

Total length: 100 cm
Total width: 70 cm
Total height: 92.5 cm
Folded width: 33.5 cm
Seat width: 46cm
Seat depth: 39.5 cm
Backrest height: 43.5 cm
Height from the ground to the seat: 46
Maximum user weight: 120 kg
Solid front wheel: 7"
Solid rear wheel: 24"
Net/gross weight: 19.8/22.2 kg
Package size: 82x32x78cm
Frame: steel

THIS MARK MEANS THE MAXIMUM USER WEIGHT!

EQUIPMENT

- patient safety belt
- brakes for the attendant
- soft armrests
- back pocket
- hinged footrests with the option of height adjustment and disassembly
- double cross
- rear wheels on quick couplers
- foldable backrest

DELIVERY RANGE

Wheelchair, 2 footrests, wrenches for adjusting the footrests, instruction manual

MAINTENANCE, CLEANING AND STORAGE

1. Avoid moisture and remember to wipe the wheelchair dry after each use. Store the stroller in the right conditions and take care of the proper storage conditions to ensure maximum durability of the device.
2. Before using the wheelchair, check that the bolts are properly tightened, especially the nuts securing the rear axle. Remember to check the condition of the spokes in the wheels.
3. The wheelchair should be routinely inspected quarterly to ensure that all components are in good working order. Remember to apply grease to the moving parts of your wheelchair on a weekly basis.
4. Remember to check the condition of your tires every time. In case of damage, please contact the service.
5. When the wheelchair is not in use, it should be stored in a safe place, protected from direct sunlight and weather conditions. It is recommended to store the wheelchair in a cool and dry place. Avoid placing heavy objects on the wheelchair.

METHOD OF DISPOSAL OF THE PRODUCT AFTER WITHDRAWAL FROM USE

After the device is decommissioned, the medical device can be disposed of as ordinary municipal waste.

TARGET PATIENT GROUP

People suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/sizes/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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