AT52308

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ENTRY

Thank you for purchasing our product. Before using the wheelchair, please read this user manual carefully. Incorrect use of the wheelchair can lead to personal injury, injury or accidents. Therefore, for safe and successful use of the product, please read this manual carefully.

GENERAL WARNINGS

- When getting in or out of the wheelchair, make sure that both brakes are engaged.
- Ensure that both brakes are engaged when the wheelchair is not being pushed or used.
- Carefully handle curbs and make turns on hills.
- Steep and uneven surfaces should be avoided as this may cause the wheelchair to tip over.
- Avoid soft surfaces as the wheels may sink in.
- Move the wheelchair smoothly in all conditions.
- Always observe your surroundings for hazards such as furniture, doors, holes, elevations, curbs, entrances, etc.
- Always keep both feet on the footrests and ensure that clothing is properly stowed and does not block the wheels.
- Always drive carefully on inclines.
- Always keep both hands on the push handles to properly control the wheelchair (assistant).
- Always remember to keep your balance.
- Check the wheelchair frame and components regularly and always follow these instructions during and after the warranty period.
- Keep the wheelchair away from sources of fire.
- Under no circumstances should the wheelchair be overloaded.
- Do not hang any heavy objects on the back of the wheelchair as this may affect stability of the product, especially when climbing hills.
- The wheelchair may only be used by one person.
- Do not use the wheelchair when the patient is unconscious.
- Do not use other means of transport to push the wheelchair.
- Do not raise the front wheels while driving, as this may cause a fall and serious injury.
- Do not get into the wheelchair while moving.
- Do not move the wheelchair too fast or make sudden turns.
- Do not stand on the footrests when getting in or out of the wheelchair.
- Do not attempt to lift the wheelchair with moving (removable) parts. Lifting with the moving parts of the product may lead to injury to the user and/or the caregiver or damage to the wheelchair.
- Do not hang any objects on the rear connector, as this may cause the backrest to fold down and cause serious injury to the user.

NOTE: Inspect all parts for shipping damage. If you notice such damage, DO NOT use the product. More information from the manufacturer.

ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of the medical device, consult a healthcare professional.

ATTENTION:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to, any of the following:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration of the patient's, user's or other person's health or
- c) a serious threat to public health

this "serious incident" should be reported to the manufacturer and to the competent authority of the Member State where the user or patient resides. In case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

APPLICATION OF THE PRODUCT

The product is designed for the comfortable transport of people with limited or no walking ability. The wheelchair is designed to transport one person.

APPLICATION/INDICATIONS

The wheelchair is used for individual transport of disabled people after spinal cord injury, with cerebral palsy, and with various injuries of the lower limbs, neuromuscular diseases, after brain and cranial injuries or for geriatric patients. It is also designed for people who want to actively and comfortably participate in everyday life.

CONTRAINDICATIONS

Contraindications to the sitting position. Conditions requiring stabilization of the upper spine and head. Patients with mental disorders should not be left unattended in the wheelchair. Physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

TARGET GROUP OF PATIENTS

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

SECURITY

Entrances, doors

The safest handling of the wheelchair is on a level floor free of any obstructions. Whenever possible, always use ramps when entering or exiting a building. Keeping your hands on the push handles allows for maximum balance.

Bending over

Do not lean to the side of the wheelchair, as this may cause you to lose your balance and fall

Moving up hills

- Do not make turns on slopes due to the risk of overturning.
- When assisting, move straight up. If you need to move downwards, walk backwards until you are on level ground.

Curbs

- If possible, always use the help of an attendant (when moving with an attendant, the wheelchair must be equipped with push handles).
- Avoid strong jolts when driving off the curb. Shock can have an adverse effect on the wheelchair.
- Always be careful when going over curbs or making turns on hills, and never walk backwards in your wheelchair without the help of an attendant.

Stairs

- Always ask for help two or more assistants. Tilt the wheelchair to the balance point.
- Do not climb stairs while driving forward.
- Do not use escalators. You must use the elevator.

BRAKE MECHANISM


- To activate the brake, pull the brake lever towards the front wheel until the brake is tight against the rear wheel.
- Make sure that the brake lever will not kick back when trying to move the wheelchair.
- To disengage the brake, pull the brake lever towards the rear wheel.
- To change the position of the brake mechanism, loosen the screw with a spanner.
- Move the entire mechanism towards or away from the wheel and tighten the screw.
- When the brake is re-applied, the wheel should not rotate (test for correct adjustment on the wheelchair with the patient).

USE

- After unpacking the package, check that it contains all the parts according to the list of components.
- Make sure that all elements have no visible mechanical damage. If in doubt, please contact your retailer.
- Do not use the wheelchair until all its components are properly connected.
- The wheelchair can only be lifted by gripping fixed elements such as the main frame, push handles.

ASSEMBLING THE WHEELCHAIR

Place the wheelchair on a level surface. Then grab the handles and unfold the frame outwards. Gently push the seat down so that the wheelchair unfolds. (fig.1)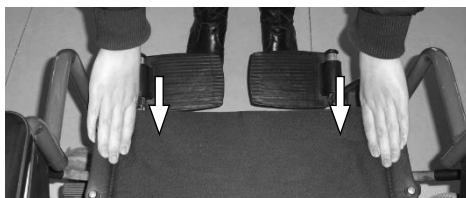


1.

ASSEMBLY OF THE FOOTRESTS

To attach the footrests, hang the footrest mounting brackets on the footrest hangers and turn them maximally inward until the footrest lock is locked and a characteristic click is heard (fig. 2). Then insert the footplates into the holes and set the appropriate height, tighten the lock nut. (fig.3)


The calf stabilizing belt should be fastened with Velcro on the footrest frames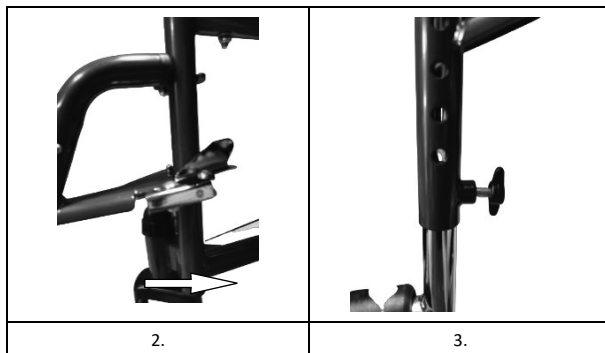

FOLDING THE WHEELCHAIR

The product should be folded in the opposite way than described above in the sections ASSEMBLING THE WHEELCHAIR and ASSEMBLY OF THE FOOTRESTS.

ATTENTION: During the use and handling of the product, as well as during its assembly and adjustment of mechanisms, there may be a risk of entrapment and/or squeezing of the user's/assistant's body parts in the openings/gaps between the elements. These steps should be performed with particular care. After adjustment, stabilize the position by carefully tightening the nuts/bolts.

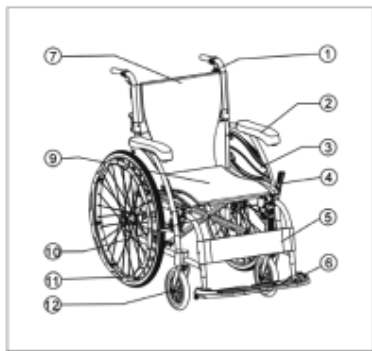List of elements:

1. Push handle
2. Armrest
3. Side panel
4. Brake
5. Calf strap
6. Step of the footrest
7. Back upholstery
9. Seat upholstery
10. Cross frame
11. Cast rear wheel
12. Cast front wheel

Tipping Lever

The wheelchair is equipped with a special tipping lever located at the rear of the frame.

This lever allows the attendant to easily lift the front wheels of the wheelchair by pressing it with their foot.

This function facilitates overcoming obstacles such as curbs or thresholds.

PARAMETERS

STEEL STRUCTURE

	41	43	46	49	H51	H56	H61
Total length	103cm	103cm	103cm	103cm	103cm	103cm	108cm
Total width	58cm	60.5cm	63cm	65.5cm	68cm	73cm	78cm
Total height	91cm	91cm	91cm	91cm	91cm	91cm	94cm
Folded width	26.5cm	26.5cm	26.5cm	26.5cm	26.5cm	26.5cm	28cm
Seat width	41cm	43cm	46cm	49cm	51cm	55cm	61cm
Seat depth	40cm	40cm	40cm	40cm	40cm	40cm	45cm
Backrest height	41cm	41cm	41cm	41cm	41cm	44cm	45cm
Height from the floor to the seat	50cm	50cm	50cm	50cm	50cm	50cm	50cm
Maximum user weight	130kg	130kg	130kg	130kg	150kg	150kg	150kg
Weight	18.1kg	18.3kg	18.5kg	19kg	20kg	20.5kg	23.6kg
Cast front wheels	8"	8"	8"	8"	8"	8"	8"
Cast rear wheels	24"	24"	24"	24"	24"	24"	24"

This symbol indicates the user's maximum weight!

DELIVERY RANGE

- Wheelchair, two footrests, calf strap, instruction manual.

MAINTENANCE

Periodically check that parts and screws on the wheelchair have not become loose due to use. If they become loose, adjust them accordingly. The moving parts of the wheelchair should be lubricated with grease for better use.

In order to maintain the performance parameters of the product, it is extremely important to keep all its elements clean and to store it properly. Storage in rooms with very high humidity (baths, laundries, etc.) or in the open air, combined with lack of hygiene, will result in a quick loss of functional and aesthetic parameters.

Do not expose the wheelchair to extreme weather conditions (rain, snow, very strong sun), as some elements may change color.

CLEANING

To prolong the life of your wheelchair, it is recommended that you clean it periodically (preferably daily), especially if it is used in rainy or snowy weather. The wheelchair should be cleaned using a soft and dry cloth to prevent rust. "The product can be wiped with a damp cloth and a mild detergent, but it should be wiped dry."

Dirt, e.g. threads, hair, etc., should be removed from moving parts of the wheelchair (wheels, footrests) on an ongoing basis. Regularly remove dirt from textiles.

DO NOT USE:

- sharp brushes and hard objects
- aggressive cleaning agents/solvents
- strong detergents
- pressure or steam aggregates
- disinfectants

STORAGE

The product should be stored in a dry, cool room, away from moisture and direct sunlight.

ATTENTION:

The manufacturer is not responsible for damage caused by neglected maintenance, improper servicing or resulting from failure to comply with the recommendations contained in this manual.

UTILIZATION

Municipal waste, please dispose of it in accordance with the applicable general disposal conditions.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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