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Thank you for purchasing our product. Please read this user manual carefully before using it. Incorrect use may lead to damage to health, injury or accidents. Thus, in order to use your wheelchair safely and successfully, please read this manual carefully.

**NOTE:** Check all parts for damage caused during transit. If such damage is noticed, DO NOT use the product.

#### GENERAL WARNINGS

DO NOT assemble or use the product before reading and understanding these instructions. If you do not understand the warnings, cautions or instructions, contact your healthcare professional, retailer or technician before attempting to assemble the device to avoid injury or damage to the product. Each user should ask their doctor or therapist to determine the appropriate adjustment and use option.

Before use, check that ALL adjustments have been secured and that the wheels and moving parts of the product are in good condition. During use, ALL castors must be in contact with the ground AT ALL TIMES to ensure the stability of the wheelchair. Failure to do so may lead to the wheelchair tipping over and injury to the user.

**WARNING:** During use and handling of the product and when assembling and adjusting the mechanisms, there may be a danger of trapping and/or pinching parts of the user's/companion's body in the openings/gaps between components. Perform these operations with particular care. Once the adjustment has been completed, stabilise the position by tightening the nuts/bolts firmly.

#### NOTES:

In the event of a product-related 'serious incident' that directly or indirectly led to, could have led to or is likely to lead to any of the following:

- (a) the death of a patient, user or other person; or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

this 'serious incident' should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

#### NOTES:

If you experience pain, allergic reactions or other distressing, unclear symptoms related to the use of the medical device, consult a healthcare professional.

**ATTENTION:** The manufacturer cannot be held responsible for damage caused by negligent maintenance, inadequate servicing or as a result of failing to follow the instructions in this manual.

**NOTE:** There may be a risk of tipping over if the product is used incorrectly. Please follow the instructions for getting on/off/moving.

**ATTENTION: it is forbidden to use the product other than for its intended purpose**

**PLEASE NOTE:** this product is not permitted for use by children under 3 years of age.

#### COMMENTS AND WARNINGS:

- When getting in or out of a wheelchair, make sure the brakes are on.
- Care must be taken when negotiating kerbs and making uphill turns.
- Steep and uneven surfaces should be avoided as this can cause the wheelchair to tip over.
- Move the wheelchair smoothly in all conditions.
- Keep both feet on the footrests at all times, and ensure that clothing is properly tucked and does not block the wheels.
- Always drive carefully on hills.
- Always keep both hands on the push handles for proper control of the wheelchair (applies to the helper).
- Keep the wheelchair away from sources of fire
- Under no circumstances should the wheelchair be overloaded.
- Do not hang any heavy objects on the back of the wheelchair as this may affect the stability of the product especially when climbing hills.
- The wheelchair can only be used by one person.
- Do not lift the front wheels while driving as this can cause a fall and serious injury

**PURPOSE:** The product is designed to comfortably transport people with limited or no walking ability. The wheelchair is designed to transport one person.

**INDICATIONS:** Contraindications to sitting position. Conditions requiring stabilisation of the upper spine and head. Patients with mental disorders should not be left unattended in the wheelchair. Physical or mental limitations that prevent safe use of the device.

#### FEATURES:

- Steel frame
- Easy-to-clean upholstery
- Removable table
- Wheelchair foldable

#### USE OF THE PRODUCT:

We recommend using the wheelchair on flat surfaces or indoors. It is recommended that you avoid using your wheelchair outdoors during bad weather conditions. Avoid surfaces such as mud, snow, sand, gravel and uneven surfaces. Avoid driving on roads with excessive traffic and avoid exposure to sunlight, which can cause the wheelchair to wear out more quickly or damage the tyres.

When using a wheelchair on the road, road safety regulations must be observed. Use the wheelchair on pavements and use pedestrian crossings.

When the product is not in use, do not expose it to direct sunlight, avoid wind, rain, fog and water. Store the device in a cool and dry place. Do not place heavy objects on the wheelchair.

**NOTE:** THE PHOTOS USED TO PRESENT THE FUNCTION OF THE WHEELCHAIR ARE FOR ILLUSTRATIVE PURPOSES ONLY, THEY MAY DIFFER FROM THE ACTUAL APPEARANCE OF THE PRODUCT.

## DISPLAY

The wheelchair is unfolded by extending the seat outwards, then pressing the seat down at the sides with both hands until it is completely unfolded. Next, unfold the backrest and attach e its corresponding holes to the top of the backrest. Finally, attach the head cushion with Velcro.

(Fig.1 and 2)

FIG. 1

FIG. 2

FIG. 3

FIG. 4

FIG. 5

## SUBMISSION

To fold the wheelchair for transport, remove the high backrest and fold down the lower backrest. Grasp the centre of the seat and then lift it upwards to fold the wheelchair. (Figures 6 and 7) To fold the wheelchair completely, start by removing the footrests and then follow the steps described above.

FIG. 6

FIG. 7

## FOOTSTOOLS

To fit the footrests, place them in the appropriate holes and turn them inwards so that they lock into the correct position. (fig. 8). Once fitted, check the stability of the footrests by pulling. To remove the footrests, pull the handle backwards (fig. 9), then turn the footrest towards the outside and pull out the footrest. To remove the footrest, lift it up. (Fig. 10)

FIG. 8

FIG. 9

FIG. 10

## FOOTREST ANGLE ADJUSTMENT

To adjust the desired angle, raise the footrest by gripping the metal tube from underneath(Fig. 11). If you need to adjust downwards, press the adjustment lever (Fig. 12) Once you have set the correct leg position, release the adjustment lever. (Fig. 13).

FIG. 11

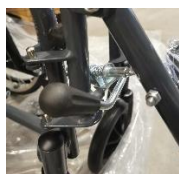
FIG.12

FIG.13

## HEIGHT ADJUSTMENT OF THE FOOTRESTS

Loosen the screw located at the footrest plate with a hex key, remove the screw, adjust the height of the footrest and then tighten the screw (Fig. 10 and 11). It is also possible to adjust the height of the calf support - 3 steps of adjustment. (Fig. 12).

FIG. 10

FIG. 11

FIG. 12

## BACKREST ANGLE ADJUSTMENT

The carer should adjust the angle of the backrest according to the physical condition of the wheelchair user using the red levers located at the handles. (Fig. 13 and Fig. 14)

FIG. 13

FIG. 14

## Table assembly and disassembly

Once the wheelchair is unfolded, attach the table holders to the tubes on either side of the frame. Do not tighten the screws; screw the two tubes into the corresponding holes in the table. (Fig. 15)

The table should then be slid into the latch hole and the screws tightened to the maximum. When lifting, the table should be pulled upwards and lifted. (Fig. 16)

FIG. 15

FIG. 16

## Brake user manual

The brakes must be applied each time you get into or out of the wheelchair. Hold the brake handle and move it forwards (unlocking) or backwards (locking). Once locked, you can get in and out of the wheelchair.

**NOTE:** Please note that the brake should not be used when driving down a hill, as this may cause a risk of overturning. When using the handbrake, hold the brake lever with both hands, tighten the grip to adjust the braking force or control the speed of the wheelchair.

**NOTE:** The wheelchair is also equipped with brakes for the carer.

## Quick-folding backrest function

The fold-back function allows the wheelchair to be stored in narrow spaces. Press the handle with both hands and then tilt the push handles of the wheelchair to fold. When opening the wheelchair, push the handles upwards until they lock.

## PROBLEM SOLVING

| Problem                                                               | Control and problem solving                                                                                                                                                                                                         |
|-----------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 1. wheelchair changes direction of travel to the right or to the left | <ul style="list-style-type: none"><li>• The condition of the wheels should be checked.</li><li>• Check the angle of the caster bracket</li><li>• Please check that the rear wheels are evenly spaced from the seat frame.</li></ul> |
| 2. Reduced turning or driving speed                                   | <ul style="list-style-type: none"><li>• Check the tightness of the bolts and nuts.</li><li>• Check that the wheel turns normally</li></ul>                                                                                          |
| 3. Wheel problems - Wobbling                                          | <ul style="list-style-type: none"><li>• Check the tightness of the bolts and nuts.</li><li>• Check the angle of the caster bracket.</li></ul>                                                                                       |
| 4. Creaking and other disturbing sounds                               | - Check the tightness of the bolts and nuts. Inspect the wheel for rust or other defects.                                                                                                                                           |
| 5. wheelchair gaps                                                    | - Check the tightness of the bolts and nuts.                                                                                                                                                                                        |

## TECHNICAL SPECIFICATIONS

|                                          |         |                           |        |
|------------------------------------------|---------|---------------------------|--------|
| Overall length<br>(recumbent position)   | 1640mm  | Seat depth                | 430 mm |
| Overall length (sitting<br>position)     | 1189mm  | Seat width                | 460mm  |
| Length after folding                     | 1189mm  | Distance between armrests | 480 mm |
| Overall width                            | 700mm   | Seat height               | 450mm  |
| Width when folded                        | 300mm   | Front wheels              | 8'     |
| Overall height                           | 12000mm | Rear wheels               | 23'    |
| Backrest height                          | 450mm   | Weight                    | 25 kg  |
| Backrest height<br>(including head rest) | 770mm   | Maximum user weight       | 120 kg |

## CLEANING

Clean the frame, seat, backrest and other components with a damp cloth and mild detergent. When cleaning the wheelchair, always wipe it dry.

## WHEEL MAINTENANCE

1. Clean the wheels with warm water and a mild detergent. Dry with a dry cloth.
2. Lubricate periodically with grease if the wheels start to squeak.

**NOTE:** STRONG DETERGENTS AND SHARP OBJECTS MUST NOT BE USED FOR CLEANING.

This sign indicates the maximum weight of the user.

## STORAGE:

Store the product in a dry place, do not expose to direct sunlight

## COMPOSITION

The frame is made of steel and the seat and back are made of polyurethane leather, with sponge inside.

## SCOPE OF SUPPLY:

TROLLEY, TWO FOOTRESTS, TABLE, HEAD REST, INSTRUCTION MANUAL.

## UTILISATION

Dispose of municipal waste in accordance with the applicable disposal conditions.

## TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/sizes/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawisłańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
[www.antar.net](http://www.antar.net)

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