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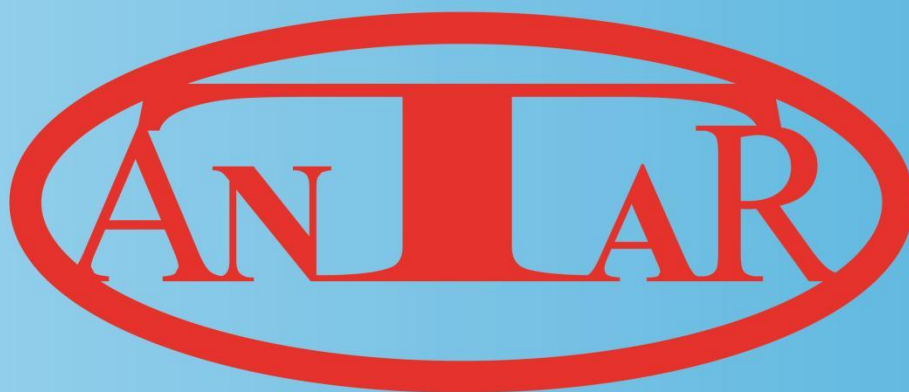
ENG

Thank you for purchasing our product. Please carefully read the user manual. Do not use the product before reading and understanding this manual. If you do not understand the warnings, notices, and recommendations, contact a healthcare specialist or the seller to avoid product damage or personal injury.

WARNING: Check all parts for any damage caused during transport. If such damage is found, DO NOT use the product. More information is available from the seller.

SAFETY PRECAUTIONS. TO MAINTAIN SAFETY

Ensure safety by: Read and understand the instructions. Seek guidance from a healthcare professional or therapist. Check all parts before use. Keep all castors in contact with the ground during use.

CAUTION

When using, assembling, or adjusting the product, exercise caution to prevent body parts from getting trapped in gaps. After adjustments, firmly secure the position by tightening nuts/bolts.

NOTE

In the case of a device-related "serious incident" leading to or likely to lead to:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

Report the mentioned 'serious incident' to both the manufacturer and the relevant authority in the user or patient's residing Member State. In Poland, contact the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

ATTENTION For any pain, allergies, or unclear symptoms related to using a medical device, consult a healthcare professional.

ATTENTION The manufacturer is not responsible for damage resulting from negligent maintenance, inadequate servicing, or failure to follow the instructions provided.

ATTENTION Using the product incorrectly may lead to tipping over. Follow instructions for proper use during getting on/off/moving. It's prohibited to use the product in ways other than its intended purpose.

TARGET PATIENT GROUP

Individuals with conditions for which this device is intended (refer to instructions) can purchase it independently or based on a doctor's, therapist's, or specialist's recommendation. Regardless of the purchase method, consider available sizes, necessary functions, variants, indications, contraindications, and manufacturer-provided information.

WARNINGS

- It is forbidden to use the wheelchair if its use is contrary to local law
- It is prohibited to use the product other than in accordance with its intended use
- Horizontal driving on a slope is prohibited
- Overloading is prohibited and the maximum load capacity is 120 kg
- It is forbidden to cross the safety line of any telescopic parts
- Manual contact with electrical components is prohibited and it must be ensured that the wheelchair is switched off before making contact
- Abrupt turning, significant change of direction and instability is prohibited, which may cause danger
- Avoid driving in adverse weather conditions, such as heavy rain, snow, wind, and on slippery or muddy surfaces.
- Children under the age of 14 are prohibited from driving alone (the permissible driving age depends on local law)
- It is prohibited to ride without a safety helmet. Wear a helmet or other protective equipment in accordance with local regulations
- Do not release the handlebar grip, rest your body on it, or lower the foot pedal while riding.
- It is forbidden to modify parts on your own and to ride a modified wheelchair on the road
- Avoid driving in hazardous environments with flammable gases, vapors, liquids, or dust that may lead to fire or explosion.
- It is not recommended to transport the wheelchair by air.

USE

1. Check power supply, tire pressure, and brakes before driving. 2. Insert the key, turn to "ON" for power activation. 3. Gradually adjust the speed control lever with your right hand for driving. 4. To stop, reduce speed, brake, turn key to "OFF," and remove it from the socket.

CAUTION The maximum speed is around 6 km/h, with a limited turning radius. Making sharp turns at high speeds can destabilize the center of gravity, leading to potential tipping and hazards.

PURPOSE power wheelchair - a vehicle designed for the mobility of a disabled person. Powered by engine whose design limits the speed of movement to the speed of the pedestrian person.

PRECAUTIONS The AT52317 wheelchair is a stable, lightweight folding four-wheel travel tool with advanced design technology, promoting environmental protection. It features a lamp, tilting armrests, a basket, and a bag and is suitable for various age groups.

UNFOLDING

1. set the wheelchair on a flat surface and pull the lower small red ball handle to unlock the folding lock, then pull it backwards and forwards to finish unfolding the wheelchair.(fig.2 and 3)
- 2.To proceed to mounting the seat, first mount the rear basket holder to the main frame with 2 large silver screws and nuts. (fig.6)
3. then install the seat in the frame, a characteristic clicking sound means that the seat has been installed correctly.(fig.1)
4. armrests should be inserted into the correct holes (first remove the screws from them) and then tighten the screws with a wrench.(fig.5)
5. Then install the rear basket with 4 black screws and round nuts, and install the basket cover with 2 smaller silver screws and black nuts. (6b)
6. Secure the front basket frame to the handlebars and attach the basket using Velcro. Additionally, fasten the basket to the handlebar using Velcro (6a).
7. Finally, connect the 2 plugs of the lamp wires to the steering wheel wires and mount the lamp holders on the steering bar and tighten the screws.(fig.4)

ASSEMBLING

On a flat surface, remove the seat part and raise the large red ball on the right to fold the wheelchair. Use the directional rod to tow it after folding. For car transportation, open the directional release handle and adjust the nut to its lowest height.

STEERING ROD ADJUSTMENT

Open the mounting bracket and adjust the directional lever to match your height, tightening the nut to secure the bracket. While adjusting the steering rod, avoid exposing the safety line marking.

FUNCTION DIAGRAM

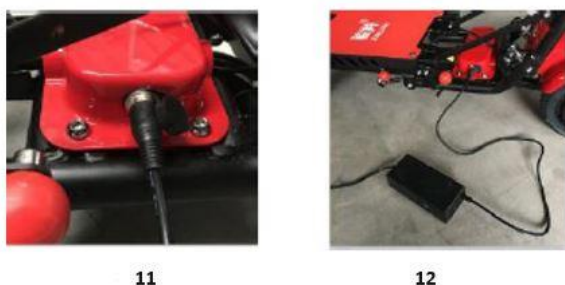
The wheelchair features an LCD instrument cluster, lock, LED headlights, and a power display. To operate, turn the key once clockwise for power on, twice for LED headlights (fig.10). Utilize the horn button for sound (fig.8). Adjust acceleration with the right speed control handle (fig.9). The "R" switch activates reverse gear, and the parking system secures the brake, preventing movement on uneven surfaces (fig.7).

MAINTENANCE AND CLEANING

If stains appear on the surface of the wheelchair body, wipe with clean water or mild detergent and then wipe with a dry cloth.

Note: Avoid using alcohol, petrol, or harsh chemical solvents for cleaning, as they can damage the appearance and internal structure. Refrain from using high-pressure water guns to prevent damage to internal components. Do not attempt to disassemble or modify the product, as this can void the warranty. Use only indicated spare parts to maintain warranty coverage and contact the company or local dealer for repairs or replacements.

Battery maintenance

Avoid exposing the product to temperatures above 40°C or below -20°C, and never place the battery in a fire to prevent battery failure or overheating. To extend the battery life, avoid completely depleting and recharging it, using the device frequently. Using the battery at room temperature will maximize its driving range and performance, whereas in temperatures below 0°C, its life and performance are reduced. Prolonged non-use may damage the battery, so charge it at least once a month. Failure to charge for a prolonged period can cause irreparable battery damage.

STORAGE

Store the product indoors in a dry, cool, enclosed space when not in use, avoiding prolonged outdoor storage. Exposure to sunlight, heat, or cold outdoors can accelerate tire aging and reduce battery life. Non-professionals should not attempt to dismantle the battery to avoid serious safety risks. Contact the company or local distributor for professional battery replacement services.

DISPOSAL OF THE DEVICE

Once the medical device has been withdrawn, it must be handled in a manner appropriate for the disposal of electrical and electronic equipment.

COMPONENTS

WHEELCHAIR, BAG, BASKET, LAMP, CHARGER, 2 WRENCHES, 3 ALLEN WRENCHES, SET OF SCREWS AND WASHERS (2 LARGE SILVER, 2 SMALL SILVER, 4 BLACK) INSTRUCTION MANUAL

NOTE: THE SET CONSISTS OF TWO PACKAGES!

TECHNICAL SPECIFICATION

Frame material: Carbon steel

Unfolded dimensions: Length 130 cm, Width 55 cm, Height 81-100 cm

Motor: 350W brushless

Folded dimensions: Length 54 cm, Width 55 cm, Height 24 cm

Armrest height: 21 cm

Backrest height: 30 cm

Seat depth: 37 cm

Seat width: 40 cm

Angle of inclination: 6°

Basket dimensions: Length 50 cm, Width 30 cm, Height 27 cm Battery:

36V10.4A×1 lithium-ion

Seat: Leather

Controller: Brushless controller

Speed: 0–6 km/h

Brake: ABS electromagnetic

Range: 20 km

Front wheel: 10" inflatable

Maximum load: 120 kg

Rear wheel: 8" solid

Net weight: 36.3 kg

This mark indicates the user's maximum weight!

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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Antar Medizin GmbH