AT52334

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Thank you for purchasing our product. Please read this user manual carefully. Do not use the product before thoroughly reading and understanding this manual. If you do not understand the warnings, notices, or recommendations, contact a healthcare professional or the seller to avoid product damage or personal injury.

WARNING: Inspect all parts for damage caused during transportation. If such damage is detected, DO NOT use the product. For more information, contact the seller.

TARGET GROUP OF PATIENTS

Individuals dealing with diseases, dysfunctions, or injuries for which this product is designed for treatment/rehabilitation or compensation purposes (see the section regarding the purpose of the product in this user manual). The product can be purchased independently by the user or upon recommendation by a physician, therapist, or other specialist. Whether purchased independently or recommended by a physician/therapist/other specialist, consideration should be given to the available sizes/necessary functions and product variants, indications, contraindications for use, and the information provided by the manufacturer.

APPLICATION

The product is intended for the movement of individuals with disabilities, limited mobility, or a total inability to walk. The wheelchair serves as walking support for individuals with limited mobility, mainly indoors (controlled by a joystick or caregiver), but also outdoors (controlled by a caregiver or joystick, with limitations described in this manual). The wheelchair is suitable for walking, riding, and pushing on hard, flat surfaces. In walker mode, the user can rest on the seat.

The wheelchair can be converted into a walker. The user can sit on the seat and use the electric assistance system, either being pushed with assistance or moving independently using the joystick.

INDICATIONS

- Significant mobility limitations: Individuals with severe muscle strength limitations, for example, due to neurological or muscular diseases or pain syndromes, which prevent or significantly hinder independent movement.
- Neurodegenerative diseases: Individuals suffering from conditions such as multiple sclerosis, Parkinson's disease, muscular dystrophies, or ALS, leading to progressive loss of muscle control and strength.
- Fatigue: Individuals for whom prolonged movement or manual wheelchair operation is too taxing due to illness or health condition, causing rapid fatigue and exhaustion.
- Age-related limitations: Older individuals who, due to age, have reduced mobility and strength may benefit from an electric wheelchair to maintain independence and mobility.
- Rehabilitation and recovery periods: Individuals undergoing rehabilitation after injuries or surgeries who temporarily cannot move independently may use the electric wheelchair as part of their treatment and recovery process.

CONTRAINDICATIONS

- Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product.
- Individuals with difficulties judging distance or space may have trouble safely maneuvering an electric wheelchair.
- Electric wheelchairs require some manual dexterity and coordination to operate the joystick or other control devices. Individuals who cannot effectively operate these devices may be unable to use an electric wheelchair safely.
- While many individuals with cognitive impairments can safely use electric wheelchairs, severe impairments in some may prevent safe operation.
- Individuals who cannot maintain a stable sitting position may require additional support systems to safely use an electric wheelchair.
- Epilepsy or other conditions that can cause sudden loss of consciousness or reaction may constitute a contraindication.
- If a person has difficulty with motor coordination or cannot independently operate wheelchair controls, this may be a contraindication.
- Individuals with awareness disorders that may affect their ability to safely operate a wheelchair may also be unsuitable for its use.
- In some cases, severe heart problems or blood pressure issues may limit the safe use of an electric wheelchair, especially when emotions or stress could trigger adverse reactions.
- Conditions such as advanced multiple sclerosis or other diseases affecting muscle control may require specialized adaptive solutions in wheelchairs.
- Problems with depth perception, spatial awareness, or general orientation may significantly limit the safe use of an electric wheelchair.

- General poor health or recent surgeries, particularly on the spine or lower limbs, may necessitate postponing the use of a wheelchair until health improves.

IMPORTANT INFORMATION

- Use the product according to the instructions to ensure your safety and the safety of those around you.
- This manual refers to the electric assistance system, which includes a joystick for control and an assistance button that controls the wheel motors.
- Use only the provided charger to charge the wheelchair.
- Use only original accessories; using unauthorized accessories may be dangerous and will void the warranty.
- Use only original parts approved by an authorized dealer for repairs and replacements. Other parts may be hazardous and will void the warranty.
- Damage caused by overloading the product or improper folding and unfolding will not only void the warranty but may also lead to dangerous situations.
- Flammable materials: avoid contact with open flames or sparks. Protect the battery from flammable materials.
- The wheelchair is designed to assist/transport one person at a time.
- Always use the handles in the proper position.
- Regularly check and/or adjust the brakes to prevent loss of braking power.
- Ensure that all power sources are connected and that electrical components, including the battery, are undamaged before first use. Damaged or improperly connected parts may pose risks or create hazardous situations.
- Joystick operation performs well on smooth surfaces but may not suffice for effective maneuvering in more demanding terrain conditions.
- While sitting in the wheelchair and using the joystick for electric assistance, always position the handles in the wheelchair mode, with handles above the frame, to access manual brakes if necessary.
- Electric assistance controlled via the assistance button may be used outdoors on surfaces deemed safe in this manual.
- While sitting in the wheelchair and using electric assistance with the joystick, move only on slopes with a maximum angle of 6°.
- While pushing the wheelchair, the slope limits are as follows: 15° for descending, 7° for ascending, and 3.5° for lateral inclines.
- Regularly check the battery charge level by looking at the joystick display or assistance indicator to know when it needs recharging.
- Keep tools and other metal objects away from the battery – contact with terminals may cause short circuits or electric shocks.
- Ensure you are familiar with the wheelchair's features before using it.
- Make sure the wheelchair is turned off when not in use.
- If the wheelchair is not used for an extended period, ensure the battery is charged before reuse.

WARNINGS

- Using the joystick outdoors is limited to a maximum safe slope of 6°, a maximum obstacle height of 15 mm, and a maximum ground unevenness of 10 mm. Exceeding these limits may lead to dangerous situations.
- The user must be physically able to perform all operations described in this manual independently. Otherwise, assistance should always be used.
- Do not use the wheelchair if any part is damaged, cracked, broken, missing, or if error codes are displayed.
- Ensure all components are securely fastened before use.
- Keep the wheelchair away from open flames and other heat sources, such as cigarettes, and electric or gas heaters.
- The user must not remain in the wheelchair during transportation in a taxi, bus, car, or other vehicles.
- The width of the wheelchair may limit access when using evacuation routes.
- The maximum range of travel using the joystick or assistance button is 10 kilometers. The range will decrease on slopes, uneven surfaces, and when overcoming curbs. Range is also affected by the user's weight, surface type, battery condition, and driving style.
- The wheelchair is not intended for use on the beach.
- We recommend using a safety belt while sitting in the wheelchair.
- Electrical devices, such as wheelchairs, may be susceptible to electromagnetic interference (EMI) from radio transmitters, TV stations, amateur radio stations, two-way radios, and mobile phones. These sources may cause interference with wheelchair components or other devices. This interference can result in unexpected wheelchair movements, such as releasing brakes. If the wheelchair begins to move due to electromagnetic interference, immediately turn it off until it is safe to resume use.

UNSUITABLE USAGE CIRCUMSTANCES

- Before boarding an airplane, the wheelchair battery must be removed and secured against short circuits by insulating the terminals (e.g., with tape). The battery should be packed separately in a protective container. Contact the airline in advance for details.

- The wheelchair is not designed for climbing stairs or for use on uneven terrains such as gravel, grass, sand, mud, or forest floors.
- Do not use escalators while seated in the wheelchair.
- Do not use or park the wheelchair in rain, snow, or other wet conditions.
- Do not use the wheelchair in the shower.
- Do not use the wheelchair on roads treated with salt.
- Do not bring the wheelchair too close to water (e.g., ditches, ponds, rivers, lakes, or the sea).
- Do not use the wheelchair on surfaces covered with snow, ice, water, or oil.
- Exercise caution on slippery surfaces, and if one of the wheels loses traction, stop the wheelchair immediately.
- Do not expose the wheelchair to temperatures below -5°C or above 45°C.

ELECTROMAGNETIC INTERFERENCE

Electronic devices may cause situations where the wheelchair does not respond normally. Devices such as transmission masts, radio stations, TV stations, and mobile phones can create a magnetic field that may permanently damage the electrical system or brakes.

- Do not use wireless devices while operating the electric assistance system.
- Stay away from radio sources.
- Immediately turn off the wheelchair if you notice uncontrolled movements.

TECHNICAL SPECIFICATIONS

- **Weight:** 19 kg
- **Dimensions of the unfolded wheelchair (maximum height, length, width):** 96 x 88 x 67 cm
- **Minimum height:** 79 cm
- **Dimensions of the folded wheelchair (height, length, width):** 79 x 66 x 32 cm
- **Seat dimensions (height, depth, width):** 52 x 30 x 42 cm
- **Front and rear wheels:** 8 inches, solid
- **Range:** ≤ 10 km
- **Maximum speed:** 5 km/h
- **Maximum user weight:** 100 kg
- **Distance between handles:** 54 cm
- **Motor power:** 160W * 2
- **Battery:** 24V 12A Lithium
- **Charger and maximum output current:** AC100-240V 50-60Hz 2A
- **Sound level:** <65 dB
- **Braking distance:** <100 cm
- **Brakes:** Electric and manual
- **Frame:** Aluminum
- **Rear wheel:** Aluminum alloy rim with rubber
- **Front wheel:** TPR + PP
- **Under-seat basket:** PU
- **Cushion:** PU foam + 600D Oxford fabric
- **Armrest:** PU
- **Backrest strap:** PE + foam + elastic fabric
- **Footrest:** Steel + plastic footplate


This symbol indicates the maximum user weight.

Dimensions provided may vary by ±2 cm.

The driving range may decrease if the wheelchair is frequently used on inclines, rough terrain, or when overcoming curbs. Braking distances on slopes may be significantly longer than on flat surfaces.

LIST OF COMPONENTS

Electric wheelchair, Armrest, Seat cushion, Under-seat basket, Motor controller, Backrest, Caregiver control, Joystick control, Lithium battery pack, Wheel motors (x2), Adjustable footrests (x2), User manual.

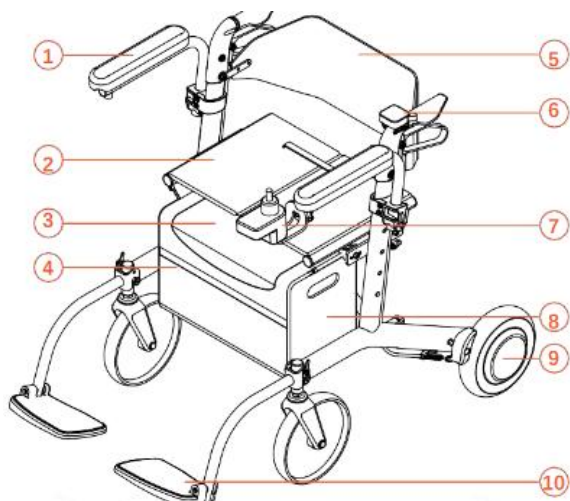LIST OF COMPONENTS

1. Armrest
2. Seat cushion
3. Under-seat basket
4. Motor controller
5. Backrest
6. Caregiver control
7. Joystick control
8. Lithium battery pack
9. Wheel motor (x2)
10. Adjustable footrests (x2)

USING THE JOYSTICK

While seated in the wheelchair, you can activate the electric assistance system by pressing the Power button on the front control panel.

1. Power On/Off button
2. Battery level indicator
3. Horn button
4. Speed increase button
5. Speed level indicator
6. Speed decrease button
7. Joystick

The front joystick controller displays the battery level on the left side, from level 1 (less than 20% / 1 light) to level 5 (over 80% / 5 lights). The speed level is displayed on the right side, from level 1 (slowest / 1 light) to level 5 (fastest / 5 lights).

REAR JOYSTICK

1. Forward and backward lever
2. Speed level indicator
3. Power On/Off button
4. Speed level indicator
5. Battery level indicator

Moving the lever forward means forward movement, and moving it backward means reverse. The rear joystick controller displays speed levels from level 1 (slowest) to level 5 (fastest).

USING THE WHEELCHAIR

Start by removing the wheelchair from the box along with spare parts and the battery. Remove the protective film, then position the wheelchair upright.

Install the joystick controller on the armrest. Adjust to the desired distance and then tighten it.

REAR CONTROLLER INSTALLATION

Insert the left bracket – Insert the left bracket into the inner handle groove. Ensure it fits and sits securely.

Rotate the controller – Gently rotate the controller in the opposite direction to allow the right bracket to fit into the outer groove.

Press and check the mounting – Once both brackets are in their grooves, verify that the controller is securely installed.

The rear controller can only be used when the joystick is switched on and the caregiver is pushing the patient. When the joystick is off, it operates in rollator mode – the person can push the chair, and the electric brake will no longer work.

FOOTREST INSTALLATION

To unfold the footrests, open the lock and pull the footrest upward. To fold the footrest, push it downward and close the lock.

UNFOLDING THE WHEELCHAIR

Lift the safety release handle, and the wheelchair will release the safety lock located under the seat, allowing it to unfold. Once the release handle is lifted, the wheelchair should begin to open slightly. Release the handle to fully open the wheelchair by pressing on the sides of the seat frame.

To ensure the safety lock is engaged, lift the wheelchair; if it doesn't fold, the lock is properly secured.

The backrest can be rotated to face forward or backward.

FOLDING THE WHEELCHAIR

To fold the wheelchair, grab the safety release handle on the seat and lift it. This will release the safety lock located under the seat. Once released, the wheelchair's arms can be brought closer together into the folded position. When the wheelchair is properly and fully folded, you will hear the safety lock click into place.

ENSURE THE WHEELCHAIR IS TURNED OFF WHEN UNFOLDING OR FOLDING.

ELECTRONICS

To connect the joystick to the battery, locate the ports on the joystick and secure them in place by tightening.

To connect the left and right motors to the drive system, find the ports on the rear controller and the cable under the seat. Connect the cable labeled "rear connection" (yellow label) through the corresponding male and female ports and secure it.

Ensure that the connections for the wheel motors (left and right) are securely fastened and that the arrows on the connectors are correctly aligned.

Take care not to bend the connector pins, as this may affect the wheelchair's functionality.


Connect the battery using the 2-pin connector. The battery is located in a pocket on the left side inside the storage bag, while the "brain" of the wheelchair is on the right side of this bag. The battery is housed in a bag under the seat on the left side. At the rear left side of the bag, there is an opening through which the cable connects to the battery.

Store cables in the Velcro-secured pocket on the outer rear part of the bag.

RAISING THE ARMREST

To raise the armrest, lift it and rotate it backward. To lower it, rotate the armrest forward and press it down.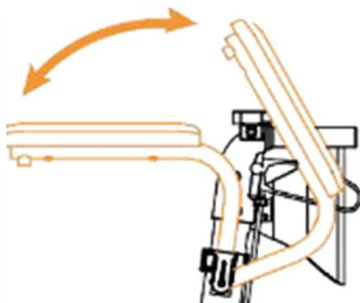

ADJUSTING HANDLES AND BACKREST

The release button for the handle is located on both sides of the wheelchair. Press the release button to raise the handle on either side individually. The handles/backrest can be adjusted in 4 increments, moved up or down, and locked into place.


MANUAL BRAKE

The wheelchair is equipped with manual brakes that can be used when operating the wheelchair as a walker or as a manual wheelchair. The manual brakes are located just below the handles on the left and right sides and can help maintain balance while walking. The brakes can also be locked by pressing the brake lever until it locks into place. This is recommended when transferring into or out of the wheelchair.

BATTERY CHARGING

- Ensure the battery is placed in the storage bag and connected to the joystick's power cable.
- Check that the connectors are properly aligned and tightened if necessary.
- Improper battery connection or misaligned connections may affect wheelchair operation and trigger an error alarm.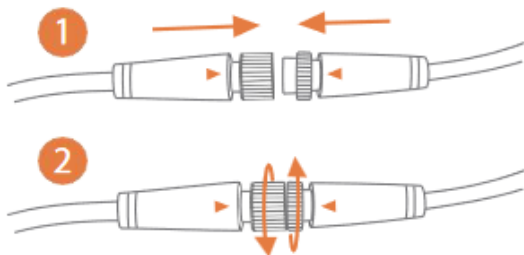

You can charge the battery by connecting the 3-pin charger to the joystick.

Note: Do not force male and female connectors together. Bent pins may result in malfunction or permanent damage to the wheelchair.

The wheelchair uses a 24V6Ah lithium battery.

- The battery should allow travel up to 10 km under normal usage.
- Charging time is 4-6 hours or until the green light on the charger turns on.
- After the green light illuminates, continue charging for an additional hour to fully charge the battery.
- Once fully charged, disconnect the charging cable.
- If the wheelchair has not been used for two months, charge the battery before use to prevent it from depleting.

- Keep the charger and joystick away from moisture and check the battery connectors regularly to ensure they are not corroded.

TEMPERATURE WARNINGS:

- Do not allow the battery to freeze, as this shortens its lifespan. Never charge a frozen battery.
- Storage should be in a clean room with humidity below 80% and a temperature below 40°C.
- Extreme temperatures can damage the battery.
- Remove the battery from the wheelchair and carry it as hand luggage when traveling. Inform the airline in advance about the battery.

OVERLOAD PROTECTION:

The wheelchair is equipped with an overload protection system that automatically cuts off power from the battery to protect the motor and battery from damage.

The lithium battery lifespan averages 400 charge cycles.

LONG-TERM BATTERY STORAGE:

- Fully charge the battery before storage.
- Remove the battery from the wheelchair.
- Store the battery at room temperature in a dry place.
- Avoid extreme temperatures, and do not allow the battery or charger to freeze.
- If the battery has frozen due to extreme weather, allow it to thaw for several hours before charging.

USAGE

The AT52334 model has both an electric wheelchair function and a rollator function. When the user sits on the chair and switches on the joystick, the electric wheelchair mode is activated. The electric wheelchair has an automatic braking function. When switched on, it brakes when the user tries to push it; it also brakes on a slope when the user releases the joystick – this is a safety feature. If the user wants to use the rollator mode, they just need to switch off the joystick to push it like a normal rollator.

PRECAUTIONS

- Always set the slowest speed when descending slopes.
- Ensure the battery is fully charged before starting a journey.
- Charge the electric wheelchair in well-ventilated areas to prevent risks.
- Avoid extreme temperatures (both cold and hot).
- Always ensure there are no obstacles behind you when reversing.
- Maintain the slowest speed when reversing.
- Avoid sudden movements and sharp turns.
- Operate the wheelchair safely and predictably.
- Keep your hands on the armrests and feet on the footrest.
- Fold the footrest away when standing to avoid accidentally stepping on it, which could tip the wheelchair.
- Leaning out beyond the armrest to reach objects may cause the wheelchair to tip over—always exit the wheelchair before reaching for items.

ERROR CODES AND SOLUTIONS

This wheelchair is equipped with an automatic fault warning system. If a malfunction occurs, an error code will appear on the control panel, accompanied by an audible alert. The following table outlines possible faults and their solutions. If the issue persists, contact the service provider.

LEFT MOTOR FAULT: Check the connector or swap the left and right motor plugs. If the error appears on the right motor, the issue is with the left motor. If no changes occur, the problem lies with the controller.

LEFT BRAKE FAULT: Check the connector connections.

RIGHT MOTOR FAULT: Check the connector or swap the right and left motor plugs. If the error appears on the left motor, the issue is with the right motor. If no changes occur, the problem lies with the controller.

RIGHT BRAKE FAULT: Check the connector connections.

JOYSTICK MISALIGNMENT: Move the joystick to ensure it operates smoothly, then restart the device.

JOYSTICK DISCONNECTION: Contact the supplier to resolve the issue.

ABNORMAL CURRENT MEASUREMENT: Contact the supplier to resolve the issue.

DRIVE SHAFT DAMAGE: Contact the supplier to resolve the issue.

CONTROL BOARD SHORT CIRCUIT: Contact the supplier to resolve the issue.

COMMUNICATION FAILURE: Contact the supplier to resolve the issue.

ELECTROMAGNETIC DRIVE FAULT: Contact the supplier to resolve the issue.

LOW BATTERY LEVEL: Charge the battery.

HIGH BATTERY VOLTAGE: Contact the supplier or recharge after discharging.

TROUBLESHOOTING

If the wheelchair is not functioning properly, check the following:

- Ensure the battery is charged.
- Turn the wheelchair off and back on.
- Verify that all battery plugs are securely connected.
- Ensure the lever is in the DRIVE position.
- Confirm that the controller is not locked.

NOTICE:

In the event of a "serious incident" related to the device that directly or indirectly:

- a) caused, may have caused, or could cause death of the patient, user, or another person, or
- b) led to a temporary or permanent deterioration of the health of the patient, user, or another person, or
- c) poses a serious public health risk,

the "serious incident" must be reported to the manufacturer and the competent authority of the member state where the user or patient resides.

For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

NOTICE:

If you experience pain, allergic reactions, or other concerning or unclear symptoms related to the use of this medical device, consult a healthcare professional.

REUSE

The wheelchair is suitable for reuse.

To ensure safe reuse, provide the new user with the full technical documentation.

If you rent the wheelchair, ensure that the user reads the manual before starting to use the product.

Before transferring the wheelchair to a new user, it should be cleaned, disinfected, and inspected by the seller for any damage to ensure safe use.

CLEANING AND MAINTENANCE

Regular maintenance is important for the longevity of the wheelchair and to maintain the warranty. Even minimal cleaning helps keep the device in good condition.

We recommend inspecting the wheelchair two or three times a year:

- **Frame:** Tighten any loose screws if necessary.
- **Wheels:** Ensure they rotate freely; resistance may indicate a need to replace bearings.
- **Armrest:** Check for excessive wear or looseness.
- Replacement parts and major repairs should be carried out by a professional.

Maintenance of the electric wheelchair includes cleaning the wheelchair, checking tires and the battery, and charging the battery. For advanced maintenance, contact a service center.

Recommended maintenance schedule: Inspect the electric wheelchair every six months and conduct a full annual service.

CLEANING THE ELECTRIC WHEELCHAIR

Regular cleaning is recommended. Components frequently touched by the user (e.g., seat cushion, armrests, controller) should be wiped with a clean, slightly damp cloth. Avoid using organic solvents for cleaning.

- If the wheelchair is used by a patient, clean it weekly. For a wheelchair used by someone with an infectious disease, disinfect it with appropriate disinfectants.
- For dirty surfaces, wipe with clean water or a mild detergent and dry with a cloth.
- Do not use alcohol, gasoline, kerosene, or other corrosive, volatile chemical solvents for cleaning, as this may severely damage the wheelchair's appearance and internal structure.
- Never use a high-pressure washing gun to clean the wheelchair to avoid damaging internal electronic components and wiring.

TIRES

Regularly check tire pressure and wear. If the tread depth decreases to 1 mm, the tire should be replaced.

BATTERY

Ensure the battery is frequently fully charged. To extend its lifespan, avoid charging it until it is completely discharged.

STORAGE

- Store the product in a dry place and avoid direct sunlight. Colors of wheelchair parts may fade with prolonged sun exposure.
- Do not expose the product to temperatures below -5°C or above 45°C, as this may damage some components and affect their functionality.
- The wheelchair contains electronic components—avoid getting it wet, and do not use it outdoors during heavy rain.
- Keep the product away from acids, bases, and other corrosive substances.

DISPOSAL

This product contains electronic components. When it is no longer in use, do not dispose of it in regular trash. For proper disposal, take it to an appropriate electronic and electrical equipment collection point. Proper disposal prevents potential harm to the environment and human health, in compliance with legal regulations on the recycling of electrical and electronic equipment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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