AT53059

**ORTEZA KRĘGOSŁUPA INSTRUKCJA OBSŁUGI
SPINAL ORTHOSIS USER MANUAL
BENUTZERHANDBUCH FÜR WIRBELSÄULENORTHESEN
UŽIVATELSKÁ PŘÍRUČKA K PÁTEŘNÍ ORTÉZE
POUŽÍVATEĽSKÁ PRÍRUČKA SPINÁLNEJ ORTÉZY
MANUEL D'UTILISATION DE L'ORTHÈSE RACHIDIENNE
SPINALE ORTHESE GEBRUIKSAANWIJZING
MANUAL DE USUARIO DE LA ÓRTESIS ESPINAL
MANUALE D'USO DELL'ORTESI SPINALE
BRUKSANVISNING FÖR RYGGMÄRGSORTOSER**

PL

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ENG

GENERAL WARNING

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product until you have read and understood these instructions. If you do not understand the warnings, cautions and recommendations, please contact your healthcare professional or dealer to avoid damage to the product.

NOTE: Check all parts for damage caused during transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

APPLICATION

The back brace has been designed to provide pressure relief and correction of the lumbar and thoracic spine in the sagittal plane, where the spine moves during bending and straightening movements, in order to strengthen the core muscle system.

This orthosis is particularly useful for people with fractures of the thoracic and lumbar vertebrae, as well as those with juvenile Scheuermann's disease, a spinal growth disorder that can cause pain due to postural problems. It can also benefit those with hyperkyphosis and chronic back pain, providing relief and correction.

INDICATIONS

- Conservative/post-operative treatment.
- Osteoporotic bone collapse in the thoracic and lumbar spine.
- Juvenile Scheuermann's disease.
- Hyperkyphosis with chronic back pain.

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling.

FEATURES

The lightweight orthosis can be easily adapted to each patient thanks to the aluminium back support, which can be easily shaped to the person's back. The orthosis is comfortable to wear under clothing.

SIZE (to find the right size, measure your waist circumference)

S: 75-85 cm

M: 85-95 cm

L: 95-105 cm

XL: 105-115 cm

XXL : 115-125 cm

COMPOSITION

65% polyamide

15% elastane

10% polyester

10% sponge

SCOPE OF SUPPLY

Spinal orthosis, instruction manual

NOTES:

In the event of a device-related "serious incident" that directly or indirectly led to, could have led to, or is likely to lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above 'serious incident' must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

NOTE: It is forbidden to use the product other than in accordance with its intended use.

ATTENTION: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from non-compliance with the recommendations contained in these instructions for use.

TARGET PATIENT GROUP

The healthcare professional shall indicate, on his own responsibility, the use of the product for adults and children, taking into account the available variants/sizes/unnecessary functions/sizes and indications, bearing in mind the information provided by the manufacturer.

USE

The orthosis has adjustable strap length. Try the product on and cut the straps to the right length, then fix them with Velcro straps. After adjusting the appropriate length you can put the orthosis on.

1. Make sure the brace is in the correct position. Place your hands in the pockets of the belt. Fasten the brace with the Velcro fasteners on the main strap and tighten the abdominal strap firmly.
2. Then fasten the two lower secondary straps by gripping both ends at the same time.
3. Fasten the top two additional straps in the same way to increase the stability of the support.
4. Finally, make sure once again that the product fits correctly on the spine, is worn at the correct height and that the straps are squeezed tightly enough to provide stability.

CLEANING AND MAINTENANCE

The product is recommended to be washed by hand with a mild detergent, at a temperature of up to 40°C. Do not wash in a washing machine. Do not use strong detergents, bleach or rough objects for cleaning. Dry flat laid out in the shade, do not expose to direct sunlight. Do not tumble dry. Do not dry clean. Do not iron.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

HOW TO DISPOSE OF THE PRODUCT

Once the product has been taken out of service, the medical device can be disposed of as normal household waste.

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S: 75-85 cm
M: 85-95 cm
L: 95-105 cm
XL: 105-115 cm
XXL: 115-125 cm

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65% Polyamid
15% Elasthan
10% Polyester
10% Schwamm ????

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślńska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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