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INSTRUCTIONS FOR USE

**AT53501 ORTHOPEDIC LEATHER INSOLES AGAINST TRANSVERSE AND
LONGITUDINAL FLATFOOT ON A RIGID SUBASSEMBLY**

**AT53502 ORTHOPEDIC INSOLES MADE OF LEATHER AGAINST TRANSVERSE
AND LONGITUDINAL FLATFOOT - CUSHIONING**

**AT53503 ORTHOPEDIC INSOLES MADE OF LEATHER AGAINST LONGITUDINAL
FLATFOOT - CUSHIONING**

**AT53504 ORTHOPEDIC LEATHER INSOLES AGAINST TRANSVERSE FLATFOOT
 $\frac{3}{4}$ -AMORTIZING**

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid damage to the product.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

AT53501

INDICATIONS:

Recommended for ankle joint discomfort and for valgus foot position, transverse and longitudinal flat feet.

FEATURES:

The insoles are made of foamed polypropylene of appropriate hardness and elasticity covered with natural leather of appropriate thickness. The insoles are certified "Healthy Foot". Connected to the insole is a contoured stiffening element supporting the transverse and longitudinal arches of the feet, having an opening in the back with latex filling, to relieve pressure on the heel bone. In the front part of the insoles, a perforation is made to allow calm air circulation inside the shoes, which also causes a gentle massage of the feet in the region of the heads of the toes. The insoles have a stabilizing character, keeping the foot in the right position.

Sizes (footwear): 35 - 45

Package contains two pieces of insoles, instruction manual

AT53502

INDICATIONS:

Recommended for longitudinal and transverse flat feet, as well as for valgus position of the heels and knees. They cushion shocks, relieve discomfort and pain in the soles of the feet insoles do not take up much space in shoes and improve walking comfort.

FEATURES:

The bottom of the insoles is made of foamed latex with the right hardness and resilience, without permanent deformation. The insoles have a contoured supinating wedge proportional to the size of the insole and a metatarsal pellet, the size of which depends on the length of the foot (from 1 to 5). The insoles gently support the arches of the foot - longitudinal and transverse. In the heel part they have a hollow that performs the function of stabilizing the foot. In the front part of the insoles, a perforation is made to allow free circulation of air inside, which also causes a gentle massage of the feet in the area of the heads of the toes. The top of the insoles is covered with natural leather of the appropriate thickness having a "Healthy Foot" certificate.

Sizes (footwear): 35 - 45

Package contains two pieces of insoles, instruction manual

AT53503

INDICATIONS:

They are recommended for longitudinal flat feet and valgus alignment of the heels and knees. They cushion shocks, relieve discomfort and pain in the soles of the feet. The insoles do not take up much space in shoes and improve walking comfort.

FEATURES:

The underside of the insoles is made of foamed latex with sufficient hardness and elasticity, without permanent deformation. The insoles have a contoured supinating wedge proportional to the size of the insole. The insoles gently support the longitudinal arch of the feet. In the heel part they have a hollow that fulfills the function of stabilizing the feet. In the front part of the insoles, a perforation is made to allow free air circulation inside the shoes, which also causes a gentle massage of the feet in the area of the toe heads. The top of the insoles is covered with natural leather of the appropriate thickness with a "Healthy Foot" certificate.

Sizes (footwear): 35 - 45

Package contains two pieces of insoles, instruction manual

AT53504

INDICATIONS:

They are recommended for transverse flat feet. They alleviate pain in metatarsals and hammer toes, prevent redness of the heads of the fingers especially in the middle part. They are recommended for lowering the transverse arch of the foot, especially for deformities of the big toe(hallux). They cushion the shock, relieve discomfort and pain in the sole side of the feet. The insoles do not take up much space in shoes and improve walking comfort.

FEATURES:

The underside of the insoles is made of foamed latex with sufficient hardness and elasticity, without permanent deformation. The insoles have a metatarsal pelota, proportional to the size of the foot. They support the transverse arch of the foot. The top of the insoles is covered with natural leather of appropriate thickness having a "Healthy Foot" certificate. The insoles have a suitable shape in the front part to fit any type of footwear.

Sizes (footwear): 35 - 45

Package contains two pieces of insoles, instructions for use.

CONTRAINDICATIONS

Physical or mental limitations (such as visual impairment) that prevent safe handling.

CAUTION:

In the event of a device-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a health care professional.

NOTE: It is forbidden to use the product other than in accordance with its intended use.

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or as a result of failure to follow the recommendations in this manual.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this product is intended (see the section on the purpose of the product of this user manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

CLEANING AND CARE

Remove the insoles from the shoes and air them out. The product can be hand washed. The insoles can be wiped with a damp cloth with soap or baking soda to neutralize the odor. Finally, the product should be left to dry. Do not use any harsh detergents, do not bleach.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

HOW TO DISPOSE OF THE PRODUCT

After the product is taken out of service, the medical device can be disposed of as ordinary municipal waste.

INFORMATION

The products AT53501, AT53502, AT53503, AT53504, AT53505, AT53506, AT53507, AT53508, AT53509, AT53510, AT53511, AT53512, AT53513, AT53514 and other products with the symbol AT535XX (where X stands for any number) - insoles and heels under the ANTAR brand - contain tissues of animal origin (animal skin).

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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