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En

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid damage to the product.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

AT53505/AT53507

ORTHOPEDIC HEELS MADE OF LEATHER ON LATEX SUB-ASSEMBLY MALE/FEMALE

APPLICATION/ INDICATIONS

Heel pads indicated for ankle injuries, spinal curvatures and lower limb alignment.

FEATURES:

The top of the insoles is covered with natural leather of the appropriate thickness with a "Healthy Foot" certificate. They have cushioning properties. They prevent chafing of the foot. The insole has a double-sided adhesive mounting tape.

Sizes: universal

AT53509/AT53511

ORTHOPEDIC INSOLES MADE OF LEATHER "FOR SPURS" ON LATEX SUB-ASSEMBLY FOR MEN/WOMEN.

APPLICATION/INDICATIONS:

The insoles prevent pain associated with inflammation and degeneration of the heel bone. The insoles relieve pressure on the heel bone bump. Indicated for heel spurs and after injuries to this bone.

FEATURES:

The top of the insoles is covered with natural leather of appropriate thickness certified by "Healthy Foot". In the middle part they have a cut-out hole that allows the heel bone to sink freely. The insole has a double-sided adhesive mounting tape.

Sizes: universal

AT53506/AT53508

ORTHOPEDIC LEATHER INSOLES ON CORK FOR MEN/WOMEN

APPLICATION/INDICATIONS:

The product cushions the load well. The heel pads perfectly adjust the height of the heel, prevent rubbing.

FEATURES:

The top of the insoles is covered with natural leather of the appropriate thickness having a "Healthy Foot" certificate. The bottom part of the insole is natural cork. The insole has a double-sided adhesive mounting tape

Sizes: universal

AT53510/AT53512

ORTHOPEDIC LEATHER HEEL PADS "FOR SPURS" FOR MEN/WOMEN

APPLICATION/INDICATIONS:

The heel pads are indicated for heel spurs and after injuries to this bone.

FEATURES:

The top of the insoles is covered with natural leather of the appropriate thickness having a "Healthy Foot" certificate. The bottom part of the insole is natural cork. In the middle part, they have a cut-out hole that allows the heel bone to sink freely. The insoles prevent pain associated with inflammation and degeneration of the heel bone. The insoles relieve pressure on the heel bone bump. The insole has a double-sided adhesive fastening tape

Sizes: universal

CONTRAINDICATIONS

Physical or mental limitations (such as visual impairment) that prevent safe handling.

SCOPE OF DELIVERY:

2 X orthopedic support, Instructions for use.

NOTES:

In the event of a device-related "serious incident" that directly or indirectly led to, could have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a health care professional.

NOTE: It is forbidden to use the product other than in accordance with its intended use.

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this produkt is intended (see the section on the intended use of the device of this manual). The product may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the product on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

USE

The insoles should be fixed in the footwear using the fixing double-sided adhesive tape, which is included on the product. Make sure that the insoles do not cause discomfort during walking and general use.

CLEANING AND CARE

Remove the insoles from the shoes and air them out. The product can be hand washed. The insoles can be wiped with a damp cloth with soap or baking soda to neutralize the odor. Finally, the product should be left to dry. Do not use any harsh detergents, do not bleach.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

HOW TO DISPOSE OF THE PRODUCT

After the product is taken out of service, the medical device can be disposed of as ordinary municipal waste.

INFORMATION

The products AT53501, AT53502, AT53503, AT53504, AT53505, AT53506, AT53507, AT53508, AT53509, AT53510, AT53511, AT53512, AT53513, AT53514 and other products with the symbol AT535XX (where X stands for any number) - insoles and heels under the ANTAR brand - contain tissues of animal origin (animal skin).

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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