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EN

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid damage to the product.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

APPLICATION

The chair is designed to provide comfort for users to sit, relax, and is also designed to support and enable safe standing for people who find this activity difficult. It supports people with injuries, damage, the elderly, etc. in standing. The chair is designed to provide comfort to users.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

-Balance and stability disorders: People who have balance or stability problems may have difficulty using the lift chair safely. Sudden movements of the chair can lead to loss of balance.

-Dermal diseases: People with active skin infections or open wounds, especially in areas of the body that come into contact with the seat, may experience worsening of the condition due to constant friction or pressure.

-Sensory disorders: people with sensory disorders, especially in the legs, may not feel comfort or pain adequately, which can lead to injuries caused by prolonged pressure going unnoticed.

-Severe circulation problems: People with severe circulation problems in their legs may experience discomfort or aggravation when using a lift chair, which increases pressure on the lower limbs during lifting and lowering.

-Safety concerns: Lift chairs are often equipped with electric mechanisms that may not be suitable for people with impaired perception or confusing behavior (e.g., in some cases of dementia), as they may accidentally operate the mechanism, leading to falls or other accidents.

AT56250 SPECIFICATION

Maximum user weight: 170 kg

Seat width: 54cm

Seat height: 51cm

Seat depth: 50cm

Width of armrests: 81cm

Overall depth: 87cm

Height of armrests: 63cm

Backrest width: 66cm

Backrest height: 71cm

Total height: 109cm

Backrest with microfiber filling density 30KGS/M3

Seat and armrest with full 30KGS/M3 density foam

Main motor: 6000N,

Backrest motor: 6000N,

Motor speed: 6mm/sec.

Input: 100-240V, 50/60Hz, 1.3A

Output: 29.0V-1.8A 52.2W

100% polyester.

Weight: 54 kg

Reclining angle: 180°

AT56251 SPECIFICATION

Maximum user weight: 170 kg

Seat width: 57cm

Seat height: 50cm

Seat depth: 56cm

Total width: 86cm

Total height: 107cm

Total depth: 92cm

Main motor: 6000N,

Backrest motor: 6000N,

Motor speed: 6mm/sec.

Input: 100-240V, 50/60Hz, 1.3A

Output: 29.0V-1.8A 52.2W

100% polyester.

Weight: 54 kg

Reclining angle: 180°

WARNING!

Place the chair on a flat floor.

DO NOT use the chair in a damp or dusty environment, as this may cause electric shock or problems with the device. The chair must be in a dry room.

DO NOT use the chair at a room temperature higher than 40 °C.

DO NOT expose the chair to heaters, stoves or direct sunlight.

DO NOT use the chair outdoors. For indoor use only. Avoid exposure to rain, snow, ice,

Salt or standing water. Preserve and store in a clean and dry place.

When the chair is NOT in use, disconnect the power cord from the main power source to avoid overheating.

DO NOT lean or place any heavy objects on the armrest, footrest or seat back.

DO NOT jump or play on the chair.

DO NOT sit or stand on the footrest.

DO NOT sleep on the chair while working.

Children must be supervised by an adult when using the chair.

Keep pets away when the chair is in motion.

Check that the voltage is in accordance with the specifications given for the chair.

The electrical connection must be made in accordance with local regulations.

DO NOT connect or disconnect the plug from the electrical outlet with wet hands. This may cause electric shock or equipment failure.

When disconnecting the plug, pull it out by holding the plug, not the cord.

DO NOT place the AC/DC adapter cable under the seat, as this may cause damage to the AC/DC adapter and cause fire or electric shock.

DO NOT touch the surface of the plug immediately after use, as it may be very hot.

DO NOT operate the remote control with wet hands, sharp or heavy objects.

DO NOT put your fingers or hands between the seat and footrest when the seat is reclined or when the chair returns from a reclined position to a sitting position.

NOTE: When using the chair in a cold room, do not increase the temperature abruptly. It is recommended to gradually increase the temperature to a normal level. If the chair is stored in a cold place and moved to a warm place, it is recommended to use it an hour later, as its performance may not be at the optimal level, as there may be dew on the mechanical parts. Problems with the machine can occur when a cold machine is suddenly used in a warm room.

NOTE: In the following cases, immediately turn off the power and unplug the power cord from the electrical outlet and contact a professional for service.

- If the seat is accidentally flooded with water. This may cause an electric shock.
- When the upholstery is torn and the internal mechanism is exposed.
- When any main function or other abnormal condition is detected during operation.
- Power failure may result in injury when power is unexpectedly restored.
- In case of heavy lightning


NOTE: it is forbidden to use the product other than for its intended purpose.

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual


SUBMITTING

STEP 1: PUT THE BACKREST CUSHION ON THE PINS.

STEP 2: ATTACH THE FABRIC FLAP WITH VELCRO TO THE SEAT BASE.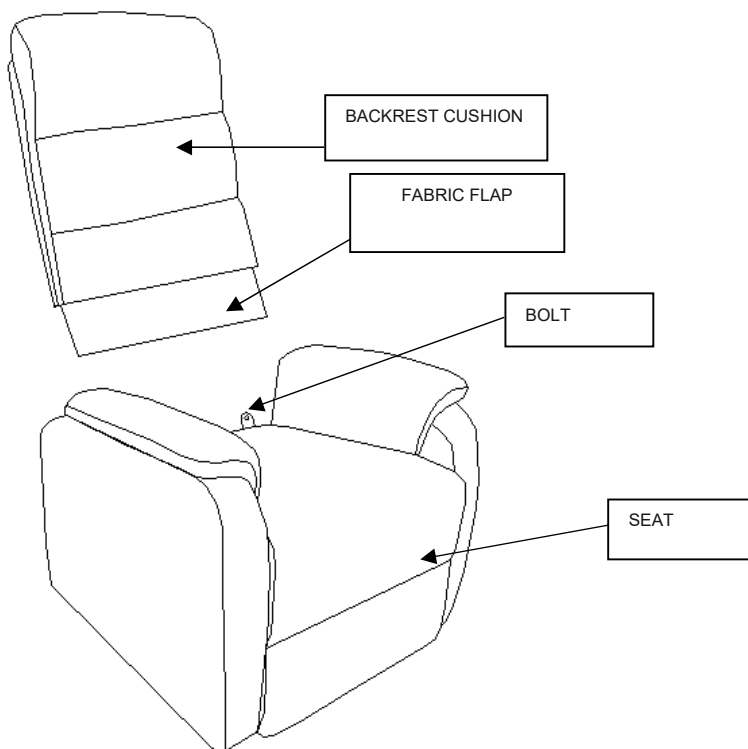

- (1) Motor and wiring connector.
- (2) Wiring and adapter connector.
- (3) Adapter wiring connector and plug.
- (4) Motor and manual control connector.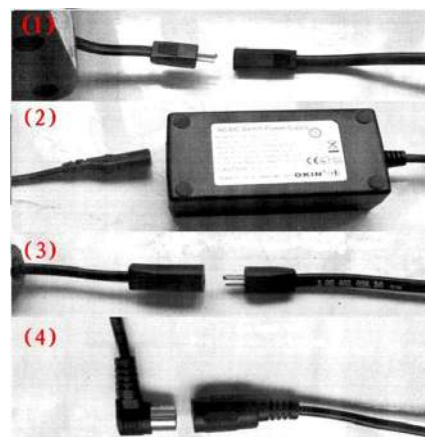


DISCLAIMER

Repeat the above steps in reverse order.

USE OF REMOTE CONTROL

The manual control activates motors in the seat to adjust and position the seat.

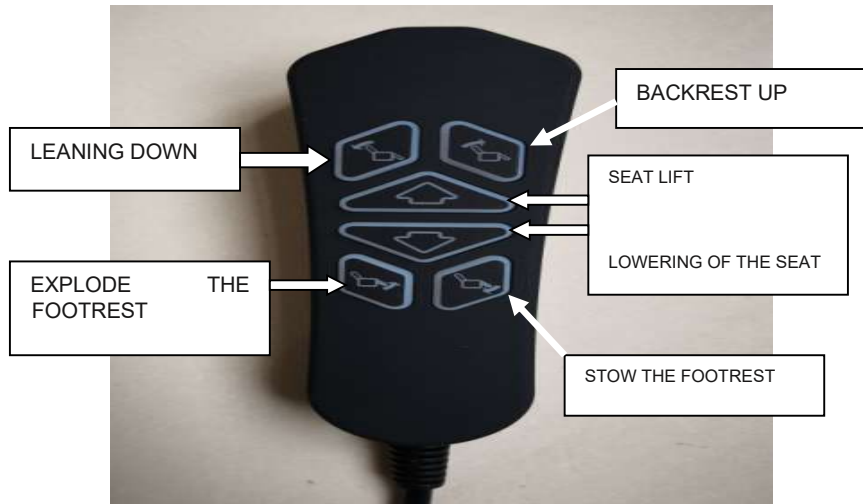MANUAL CONTROL MASSAGE (AT56251)

OPIS FUNCTIONS OPERATIONAL PRESSURES

- 1) **POWER**: Product power button
- 2) **HEAT**: A button that activates the heating function. The heating temperature is within the range of 50-60 °C.
- 3) **INTENSITY**: Button to select the intensity of the massage. This function has two intensities: strong and weak
- 4) **MODE**: Massage mode selection button. Five vibration modes are available for this function
- 5) **BACK**: Button to control the back massage.
- 6) **LUMBAR**: Button to control the waist massage.
- 7) **THIGH**: Button for thigh massage control
- 8) **LEG**: Button to control calf massage
- 9) **TIMER**: Massage time selection button. This function has three time options: 15 minutes, 30 minutes and 60 minutes. The default massage time is 30 minutes

Product manual

- 1) Connect all product connectors and turn on the power supply
- 2) Activate the power button. The product is ready for use. The default massage intensity is strong, the vibration mode is automatic, and the massage time is 30 minutes. The intensity button, mode button and time button can be customized.

CLEANING AND CARE

Wipe off dust and dirt regularly. Keep the chair clean and away from high humidity areas.

If the chair is not used for a long time, cover it with a piece of clean cloth/fabric to protect it from dust.

NOTE: Before cleaning, make sure the main switch is turned off and the AC plug is removed from the electrical outlet.

A synthetic and very mild soap can be used to clean the plastic, pipes and upholstery of the seat. Carefully wipe off any traces of soap with a dry cloth.

To clean a dry stain, dip a piece of soft cloth in warm soapy water, wring the cloth dry and apply it to the stain.

CAUTION: Do not use cleaning agents such as benzene or thinner, or any other cleaning agents that may fade the color of the fabric and scratch the upholstery.

Do not remove the upholstery for washing. Ironing of upholstery is strictly prohibited.

Fabric should not come into direct contact with any heat source, such as a heater. This can cause injury or fire.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

Do not store the chair in direct sunlight or in places with high temperatures. This may cause the color of the fabric to fade.

METHOD OF DISPOSAL OF THE PRODUCT

This product contains electronic parts. After its use is over, do not dispose of it in the regular trash garbage can. In order to dispose of the product properly, please take it to an appropriate collection point for electrical and electronic equipment. Disposal in a proper manner will prevent potential harm to the environment and human health, in accordance with current laws on recycling of electrical and electronic equipment.

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Seal of the point of sale

and legible signature of Vendor:

Importer – Antar MEDICAL Sp. z o.o. hereby guarantees that the product sold is free from defects. In the event of a complaint, Antar will choose between the option of repair or replacement of the defective part, provided that the following criteria are met.

The product is guaranteed for 2 years and covers the following components of the chair:

Remote control, upholstery, mechanical components, wooden components, electrical components.

This warranty applies only under normal use conditions.

The warranty does not apply in cases:

- in case of wear and tear of the parts, especially fabrics and covers
- if the warranted part has been misused or inadequately maintained / improperly handled, damaged during transport or use, modified or repaired by unauthorized persons.
- if the electrical components have been subjected to impact, shock and other mechanical damage
- if the cables/plugs are in poor condition (cable torn or cut when moving the chair, e.g. to wash the floor, uncleaned (dust accumulation) or uncovered cable (e.g. bitten by a pet).
- if electrical components have been exposed to liquid or surge (lightning strike).
- if the proper distance from the wall is not maintained.
- if the user falls/leaves himself/herself to fall on the chair.
- if the chair is used beyond the weight tolerance limits.
- to cover the cost of unnecessary service visits to the repair center (e.g., wrong cable/plug connection; other situations mentioned above).

This warranty does not cover claims for conditions or damage caused by misuse, accidents, negligence or lack of care in cleaning and care of the fabric. This warranty also does not cover claims for scratches, tears, excessive or abrasive wear, and does not cover claims for damage caused by repairs, tampering or modification of the product by unauthorized persons.

Read and strictly follow the instruction manual before operation. Operation of the product not in accordance with the intended use and recommendations will cause irreversible loss of warranty.

THIS WARRANTY DOCUMENT IS VALID ONLY WITH PROOF OF PURCHASE (RECEIPT, BILL, INVOICE). PLEASE INCLUDE PROOF OF PURCHASE WITH EACH COMPLAINT.

Complaints submitted without proof of purchase and a warranty card with the product serial number written on it will not be accepted.

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ANHUI LRE FURNITURE CO.,LTD