DOMICILE - HOMECARE

CLIPUP

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EN - Instructions for installation and use

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Copyright 2025 par HERDEGEN SAS– ALL RIGHTS RESERVED

In the interests of continuous improvement, HERDEGEN SAS reserves the right to change any detail or specification in this document without prior notice.

The information contained in this document is confidential. It may only be shared with a third party with the prior written consent of HERDEGEN SAS. This document may not be reproduced or communicated, in full or in part, in any way or by any means, whether technical or electronic, by photocopying, recording, or using any information or data extraction systems.

You can obtain a new copy of this document from HERDEGEN SAS by contacting your regional sales representative or directly contacting HERDEGEN SAS.

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FOREWORD

You have just purchased a HACARE product and we are pleased to have you as one of our clients. We also thank you for placing your trust in our company and our products.

Your trust assures us that the HACARE company continues to meet your specific requirements and needs. That's why we always strive to innovate, providing you with quality solutions and products that improve the comfort and quality of patient care while making the caregiver's daily tasks easier.

HACARE products have gained the confidence of many professionals and institutions due to their functional and aesthetic qualities, and are now considered to be a solid benchmark. Simplicity, reliability and safety are the hallmarks of our electrical healthcare cradles.

You've chosen HACARE, and we're committed to satisfying your needs.

However, you are responsible for the proper use of the cradles. That's why we invite you to familiarize yourself with the technical specifications, and the instructions for use and maintenance of the equipment by reading through this manual.

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EN - Please read these safety instructions carefully before use and keep them for future question.

1. SAFETY INDICATION, ABNORMAL USES, DANGERS

- ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- Be careful not to get your fingers caught when tightening the bracket and installing the device.
- Do not lean out of the raised seat to avoid falling out of the device.
- Do not seat or step on the lid.
- Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature).
- Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- Don't use outside but only inside and on standard toilet bowls.
- Check carefully all parts of this item (shell and antislip pads) and if all parts are tightened.
- Use only as a raised toilet seat and in proper way : don't knee or step on the product, don't seat on the lid.
- Proceed slowly when getting up or sitting down, do not rotate while sitting to avoid any risk of the device coming off the toilet.
- Caution, taking certain medications can cause skin fragility.
- Be careful, in case of dermatoporosis, ask your doctor for advice.
- ALWAYS observe the weight limit on the labelling of your device.
- No modification must be made on this device. Use only original

parts to replace the broken or defect one and let this be done by the technical staff.

- After assembling the product, make sure that it is securely locked and fixed before using.
- In case you didn't use your raised seat for a long time, let it checked by the technical staff, before using.
- Always check stability before using.
- Beware In very few cases (such as rectal prolapse), the product can entrap genitalia or rectal prolapse, leading to pain and serious injury such as cuts, lacerations and testicular bruising. If in doubt, please contact your physician or supplier to determine a safe use of the device.

- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (qualite@hacare.com) and the competent authority of the member state in which the user and/or the patient is established.

- The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. INTENDED USE, INDICATIONS AND CONTRAINDICATIONS

Intended use:

A raised toilet seat is an attachment to an existing toilet which adds extra height for the user. It is designed for those with limited mobility who have difficulty sitting down or standing up from the toilet.

Indication

- Decreased joints (knee or hip) and spinal mobility
- Difficulty in standing up and seating making impossible the use of standard toilet.

Contraindication

- Severe cognitive disorder, severe loss of balance, sitting impairment.
- Patient weight is greater than the product's supported weight.
- Other contraindications to be determined by the prescribing physician based on the patient's condition.

3. INSTALLATION

A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

This item is equipped with a closing mechanism enabling a tight fixation on the toilet bowl.

If the 6 antislip pads are missing, please do not use this device and contact your dealer.

Installation, see steps [1-2]

The Clipup is preset to the maximum opening.

- 1-Place the 6 antislip pads in the designated positions [1].
- 2-Lift the toilet seat and lid.
- 3-Place the toilet raiser onto the toilet bowl and make sure that it is properly fitted on the rim of the toilet [2].
- 4-Push the bracket tight. The minimum setting [6.2] must be reached, otherwise do not use
- 5-Check the stability before using the product.

Removal, see steps [3-4-5-6-7]

- 1-Simply remove the raised seat by lifting up as described in steps [3].
- Restart from step 4 to 7 to install it again.

2-Check the stability before using.

Rental:

This product can be rented to different patients. A thorough maintenance, cleaning and disinfection must be performed before the product is given to a new patient in accordance with the following steps:

- 1- Visual inspection: ensure that the product has not been damaged and check for any defect.
- 2- Functional inspection: ensure that the product functions properly and repair it if needed.
- 3- Cleaning and disinfection: this product can be cleaned and disinfected with mild, chlorine-free detergents such as Survanios (Anios laboratories).

The user's manual is an integral part of the product and must be handed over to every new user.

4. CLEANING

The raised seat has to be cleaned regularly with a mild cleaning agent and a soft cloth. It may be washed with soap and water or disinfected with non-chlorinated products.

Do NOT use any abrasive cleaners or high-pressure cleaners that could damage the product. Use only appropriate tools to perform the maintenance, cleaning and disinfection.

5. DISPOSAL

Dispose of this product in accordance with your regional waste disposal regulations.

6. TECHNICAL SPECIFICATIONS

See table page 6. This product is biocompatible. The warranty

duration of the product and its various elements is specified in the commercial conditions.

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