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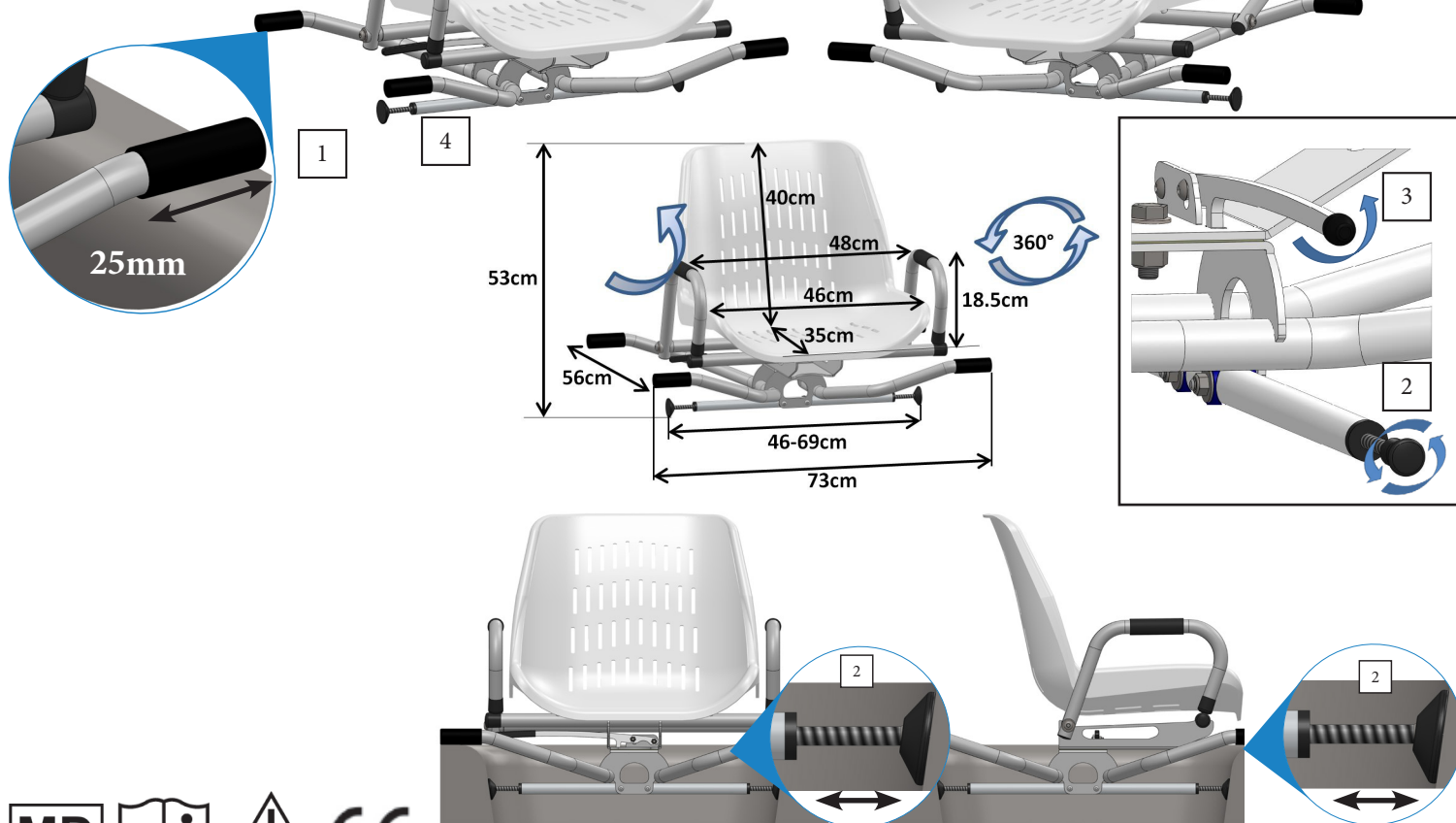
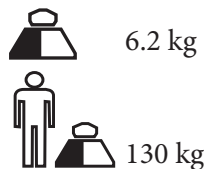
EN- Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, shellseat...) and that all screws are tightened. Check that the bath seat swivels easily, smoothly and locks into place. If the locking mechanism does not feel positive or there is excessive movement in the device, remove from use and contact your dealer.
- ♥ Check for cracks and damage, check in particular the plastic seat and tubing.
- ♥ Use only as a bath seat and in proper way : don't stand up on the seat.
- ♥ ALWAYS observe the weight limit on the labeling of your product.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After installing the seat, make sure that it is securely locked and stable.
- ♥ Make sure that seat is locked into position and no longer rotates freely before starting washing. Make sure that after a rotation, the seat is locked again and the lever correctly engaged into the hole.
- ♥ Do not stand up or sit down without having checked the stability of the bath seat and that the seat no longer rotates freely. Proceed slowly when you sit down or stand up.
- ♥ You need someone to assist you when using the bath seat.
- ♥ Beware when exiting the seat, the floor might be wet and slippery.
- ♥ Do not lean out of the bath seat to avoid losing balance.
- ♥ In case you didn't use your bath seat for a long time, let it checked by the technical staff, before use.
- ♥ Do not store outdoor, the product must be used at home.
- ♥ Check the bathseat before use, in case of doubt ask your physician or provider.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

2. Intended use, indications and contraindications, installation

Intended use:

The purpose of this product is to provide an aid to get into and out of a bath. It is designed to sit on top of the bathtub and to swivel round at 360°, locking in place with an easy-to-use lever. The swivel bath seat will fit most standard

baths having a dimension of ca 70cm.

Indications

Inability to get in/out of the bath independently and to sit or stand in the bath independently.

Functional impairment or lack of functional capability of the lower extremities
Severe limited trunk control with enough residual functions of the upper extremities.

Contraindications

Don't use in case of:

- Cognitive disorder - Loss of balance - Sitting impairment - Inadequate hip and knee flexion and inability to transfer safely on / off bath seat independently.

Beware ! The plastic shell is equipped with holes to drain water away from the seat. In very few cases (such as rectal prolapse), those drainage holes can entrap genitalia or rectal prolapse, leading to pain and serious injury such as cuts, lacerations and testicular bruising. If in doubt, please contact your physician or supplier to determine a safe use of the device. You can also place material over the bathseat to prevent flesh or anatomy from passing through.

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Installation:

Products are delivered mounted and ready to use.

1. Place the bath seat over the bath. At least 25mm of each blue non-slip arm must rest on the flat sides of the bath. Check stability, the seat shall not be unstable or shaky [1].

2. Secure the frame by turning the two fixation screws [2] until secure against walls of bath on either side. Make sure that the frame is secure, but do not over tighten the screws.

3. Lift the locking lever (located under the seat) to release and rotate the seat [3].

Check stability before use and make sure that the locking lever works properly and in each locking holes.

4. Both armrests are swinging [4].

Using the swivel bath seat:

1. Lift the lever and turn the seat to face the user, then lock the seat into position by lowering the lever and keep swiveling the seat until it locks in position.

2. Transfer the use carefully into seat, beware of wet and slippery surfaces.

3. Unlock the seat by lifting the lever.

4. Lift the user's legs over the edge of the bath one at a time or both together while the bath seat is swiveled slowly.

5. When the user is in proper bathing position lock the seat and ensure that it is locked into position and no longer rotates freely.

6. To exit the bath, unlock the seat (by lifting the lever) and turn the seat slowly, while lifting the user's legs over the edge of the bath one at a time or both together.

7. Lock the bath seat when the user faces the open side of the bath. The user can now stand up carefully and slowly.

Rental: this bath seat can be rented under the condition that it was correctly cleaned. This user's manual belongs to that product and must be delivered to each new user.

3. Cleaning

The bath seat has to be cleaned regularly with a mild cleaning agent and a soft cloth.

4. Materials

The product is made up of steel (doesn't rust) tubing treated against rust and covered by epoxy paint. The plastic shell seat is made of Polypropylene and various plastic components.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

Bath width: 56 to 68cm (inside) / 73cm (outside).

This product has a 2 year warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible. Duration of use: 2 years.

HR- MOLIMO Pažljivo zapitajte ove sigurnosne upute prije uporabe, zadržite ih za buduće pitanje.

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