DOMICILE - HOMECARE

SIMPLY NEO

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EN - Instructions for installation and use

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Copyright 2025 par HERDEGEN SAS– ALL RIGHTS RESERVED

In the interests of continuous improvement, HERDEGEN SAS reserves the right to change any detail or specification in this document without prior notice.

The information contained in this document is confidential. It may only be shared with a third party with the prior written consent of HERDEGEN SAS. This document may not be reproduced or communicated, in full or in part, in any way or by any means, whether technical or electronic, by photocopying, recording, or using any information or data extraction systems.

You can obtain a new copy of this document from HERDEGEN SAS by contacting your regional sales representative or directly contacting HERDEGEN SAS.

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FOREWORD

You have just purchased a HACARE product and we are pleased to have you as one of our clients. We also thank you for placing your trust in our company and our products.

Your trust assures us that the HACARE company continues to meet your specific requirements and needs. That's why we always strive to innovate, providing you with quality solutions and products that improve the comfort and quality of patient care while making the caregiver's daily tasks easier.

HACARE products have gained the confidence of many professionals and institutions due to their functional and aesthetic qualities, and are now considered to be a solid benchmark. Simplicity, reliability and safety are the hallmarks of our electrical healthcare cradles.

You've chosen HACARE, and we're committed to satisfying your needs.

However, you are responsible for the proper use of the cradles. That's why we invite you to familiarize yourself with the technical specifications, and the instructions for use and maintenance of the equipment by reading through this manual.

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4. NETTOYAGE, DESINFECTION, MAINTENANCE

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EN - Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place. If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. SAFETY INDICATION, ABNORMAL USES, DANGERS

- Always consult with your physician or supplier to determine the proper adjustment and use of the device.
- Don't store outside and don't leave in the sun (Beware : don't let the product in a car in case of high outside temperature).
- Don't use in extreme temperatures (above 38°C or 100°F or below 0°C or 32°F).
- Check all the parts of this item carefully (screws, tubes, tips, grips..).
- Regularly check if all screws are tightened.
- Hold the handle firmly and completely when using this product.
- In case of falling: let the product fall down on the side, so that you don't fall on it.

- Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
- Don't use on wet or icy surfaces (snow, ice, water..)
- Don't use in stairs, severe risk of injuries
- Use only as a walking aid and in the intended way: don't use as a lever or to move furniture.
- Always observe the weight limit on the labelling of your rollator.
- After unfolding or assembling the walking aid, make sure that it is securely locked in OPEN position and level to the ground before use.
- Do not hang anything on the Walking aid.
- Always test to see that the walking aid and attachments are properly and securely locked in place before using.
- No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- In case you didn't use your walking aid for a long time, let it be checked by technical staff first before using it.
- Be careful when walking: beware holes and deformations in the floor.
- Be careful: risk of crushing and squeezing fingers when folding/unfolding.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (qualite@hacare.com) and the competent authority of the Member State in which the user and/or the patient is established.

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. INTENDED USE, INDICATIONS AND CONTRAINDICATIONS

Walking aids are used for two purposes: as part of a rehabilitation programme when the user is recovering from an injury or operation and as a long-term aid to mobility when the user has a permanent difficulty with walking. These devices help in maintaining balance, aid in the prevention of injuries (particularly falls), and allow the person to

attend to his/her errands independently. Mobility aids can be used indoor or outdoor, depending on the model.

Mobility aids are used to modify the weight redistribution: some of the weight carried through the legs when walking is transferred through the arms of the frame or stick as it is leant on for support. Products equipped with a seat allow short rest to the user.

Indications:

- Loss of autonomy due to walking disorders
- Balance and/or posture disorders, walking problems
- Rehabilitation programme after an injury or an operation

Contraindications:

- Severe cognitive disorder, severe loss of balance, severe motor or perception disorders, weakness in the upper limbs
- Patient weight is greater than the product's supported weight.
- Others contraindications: to be determined by the prescribing physician based on the patient's condition

3. INSTALLATION

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Check the contents of the box, see picture [1].

Push the seat forward to unfold the rollator. Make sure that the seat is properly fitted and locked in place, see picture [2].

Installing and adjusting the anatomical handles (see pictures [3 and 3abcd]):

1-Insert the handles into the tubes and ensure that the anatomical support are placed inwards: right handle on the right side, and vice versa (see picture [3]).

2-Position the anatomical handle so that it aligns with the user's wrist when they let their arm hang naturally by their side.

3-Insert the screw through both the frame and the handle's tube, then screw the tightening knob clockwise. Repeat these steps for the other handle and set both sides to the same height.

Folding the rollator (see picture [4]): To fold the rollator, pull the seat up using the handle.

4. CLEANING AND DISINFECTION

This product must be cleaned regularly with a mild cleaning agent and a soft cloth. Never use oily substances (slip hazard). Ask your retailer or the technical staff to check the walking aid regularly.

The rubber tips must be checked and changed regularly.

5. DISPOSAL

Dispose of this product in accordance with your regional waste disposal regulations.

6. TECHNICAL SPECIFICATIONS

See the table above. This product is biocompatible. The warranty duration of the product and its various elements is specified in the commercial conditions.

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