AT51139
INSTRUKCJA UŻYTKOWNIA, CHODZIK
INSTRUCTION MANUAL, ROLLATOR
GEBRAUCHSANWEISUNG, ROLLATOR
NÁVOD K POUŽITÍ, CHODÍTKO
NÁVOD NA POUŽITIE, CHODÍTKO
MANUEL D'INSTRUCTION, DÉAMBULATEUR
GEBRUIKSAANWIJZING, ROLLATOR
MANUAL DE INSTRUCCIONES, ANDADOR
MANUALE DI ISTRUZIONI, DEAMBULATORE
BRUKSANVISNING, RULLATOR

PL

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EN

Thank you for purchasing our product. Please read this instruction manual carefully before using the product. Do not use the device before reading and fully understanding these instructions. If you do not understand the warnings, notes, or recommendations, please contact a healthcare professional or the seller to avoid product damage or personal injury. NOTE: Check all parts for any damage that may have occurred during transport. If such damage is found, DO NOT USE the product. For further information, contact the seller.

INTENDED USER GROUP

The product is intended for individuals suffering from illnesses, dysfunctions, or injuries for whose treatment, rehabilitation, or compensation this device is designed (see the Purpose section of this manual). The product may be purchased independently by the user or on the recommendation of a physician, therapist, or another specialist. In both cases, it is important to consider the available sizes, necessary functions, and product variants, as well as indications and contraindications for use and information provided by the manufacturer.

PURPOSE: The product is designed to support mobility for people with limited walking ability. The rehabilitation rollator is intended for use by a single person.

INDICATIONS

The product can be used for:

Balance training: The rehabilitation rollator enables patients to practice maintaining balance, which is crucial in the rehabilitation process after injuries or neurological disorders.

Posture correction: Patients can use the rehabilitation rollator to train proper body posture, which is especially important for individuals with postural defects or stability problems.

Improving motor coordination: Exercises performed with the rehabilitation rollator can help improve motor coordination and movement control, which is essential for people with neurological impairments.

Supporting post-injury rehabilitation: The rehabilitation rollator may be used in the recovery process after lower limb injuries, helping patients gradually regain mobility.

Increasing independence: For some patients, the rehabilitation rollator can serve as a tool to enhance independence in daily life, allowing for self-supported movement.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product.

The use of a rehabilitation rollator may be inappropriate for individuals with severe spinal instability or spinal injuries, as it may increase the risk of further harm.

In persons with uncontrolled seizures or epilepsy, use of the rollator may be dangerous for both the patient and others nearby.

In persons with very limited muscle strength, using the rollator may be difficult or ineffective; other rehabilitation methods such as manual therapy or muscle-strengthening exercises may be required.

For patients with serious balance disorders, use of the rollator may be unsafe as it could increase the risk of falls or additional injuries.

LIST OF COMPONENTS: Rollator, bag, backrest, instruction manual

PRODUCT FEATURES

Lightweight and easy to store or transport

Excellent maneuverability and stability

Adjustable handle height for user comfort

Equipped with a seat for resting

Includes a shopping bag for carrying small items

TECHNICAL SPECIFICATION

Frame: material and finish – aluminium; powder-coated finish

Wheel fork: PA + TPE

Shopping bag: canvas bag with flap and zipper

Seat: 85% PA + 15% GF; 1 cm thick PU leather cushion

Backrest: 100% Oxford fabric

Handles: PA + TPE, smooth, non-slip surface with ergonomic design

Maximum load capacity: 135 kg

Maximum bag load: 5 kg

Total weight: 8.55 kg

Length: 61 cm

Total width: 62 cm

Total height: 82–99 cm

Folded height (minimum): 82 cm

Width unfolded: 62 cm

Width folded: 30 cm

Folded dimensions (L × W × H): 62 × 30 × 88 cm

Seat width: 41 cm

Seat depth: 26 cm

Seat height: 57 cm

Backrest height: 13 cm

Distance between handles: 52 cm (outer) / 41 cm (inner)

Adjustable handle height (min–max): 82–99 cm

Brakes: 2 maneuvering brakes with parking function

Front wheels: 200 × 36 mm – black, PA + 15% GF + TPR, soft surface

Rear wheels: 200 × 36 mm – black, PA + 15% GF + TPR, soft surface

NOTE: The individual product dimensions may vary within ± 2 cm.

This symbol indicates the maximum user weight.

UNFOLDING THE ROLLATOR

Hold the left handle with your left hand, then gently push the backrest of the seat forward with your right hand until the seat stabilizes. Press the seat downward until you hear a “click,” indicating that the frame locking mechanism is engaged.

After unfolding or assembling the rollator, always make sure that the seat support is securely locked and the frame does not move.

Always check that both handle tubes are securely locked into the main support tube using the red height adjustment locking button to prevent accidental slipping of the handles.

Always ensure that all accessories are correctly and securely attached before use.

Failure to follow these warnings may result in personal injury.


FOLDING THE ROLLATOR

Open the red height adjustment locking button, then gently lower the handles.

Lock the height adjustment button again.

Pull out the locking pins of the folding mechanism.

Gently lift the seat and fold the rollator.

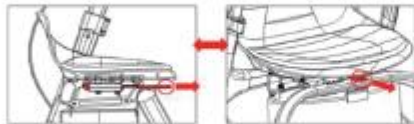ADJUSTING THE HANDLE HEIGHT

Release the handle lock button.

- Set the handles to the desired height.
- Press the locking button until you hear a “click,” indicating the frame mechanism is locked.
- Both handles must be set at the same height.
- Properly adjusted handles should be at the user’s hip level.
- Never leave the rollator without the brakes engaged.

ASSEMBLING THE ROLLATOR

Step 1: Installing the seat

Place the seat above the rollator frame in the designated mounting position. Align the holes in the seat with the mounting bracket holes on the frame. Insert the mounting screws from above and tighten them with the Allen key. Ensure that all screws are firmly tightened and the seat is stable.

Step 2: Installing the seat cushion


Position the soft seat part so that it lies securely and does not slide. Make sure the protrusions fit correctly into the designated holes.

Step 3: Installing the handles


Slide the part with the adjustment latch downward. Then insert the handles into the frame tubes on both sides of the seat. Position them vertically and push until you hear a characteristic locking click.

Step 4: Adjusting the handle height

Adjust the handles to the user’s height. To do this, release the adjustment latches located near the handle tubes, set the desired height, and then lock the latches. Make sure both handles are at the same height.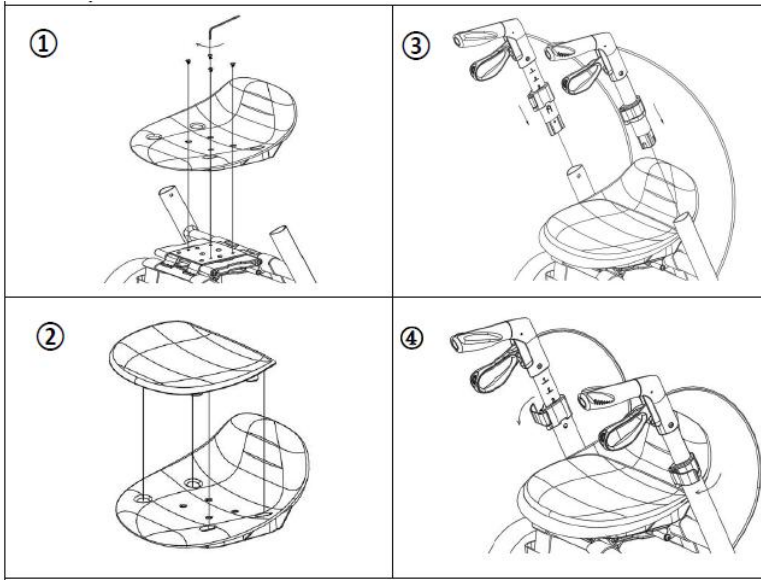

INSTALLING THE BACKREST: Attach the backrest by fastening the straps to the side tubes of the rollator.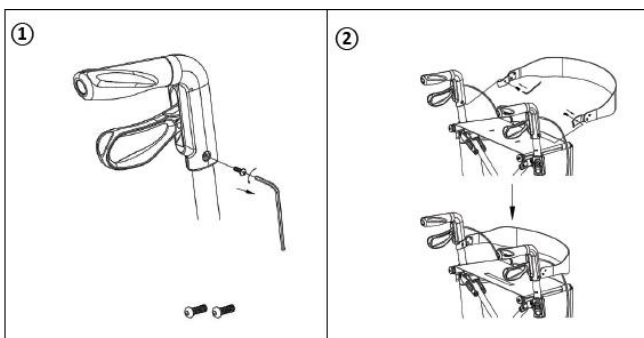

USING THE BRAKES

To stop the rollator, pull the brake levers upward toward the handles.

To keep the rollator stationary, press the brake levers down into the locked position.

To start moving, release the brake levers – they will automatically return to the released position.

Always engage the brakes when you do not intend to move the device.

Never leave the device without the brakes engaged while someone is sitting on it!

Never leave the device on an incline without the brakes engaged!


When sitting on the rollator, always sit upright, with your forearms or hands resting lightly on the handles.

Never lean backward while sitting on the rollator.

Locking/Unlocking the hand brakes

To lock the brakes, press the brake levers downward until you hear a “click.”

To unlock the brakes, pull the brake levers upward until you feel them release.

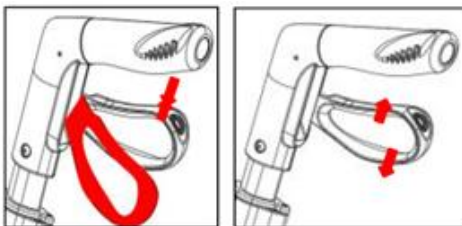WARNING

In the event of a “serious incident” related to the device that has led, might have led, or could lead to:

- a) the death of a patient, user, or another person; or
- b) a temporary or permanent deterioration in the health of a patient, user, or another person; or
- c) a serious public health threat;

this “serious incident” must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. CAUTION: If pain, allergic reactions, or other concerning or unclear symptoms occur during use of the medical device, contact a healthcare professional. CAUTION: It is forbidden to use the product in any way other than its intended purpose.

USAGE LIMITATIONS

The seat must not bear a weight exceeding 135 kg.

The push handle must not bear a weight exceeding 120 kg.

The bag has a maximum load limit of 5 kg.

The device is intended for use on flat surfaces, both indoors and outdoors.

The device is not intended for self-propulsion while seated.

The brake **MUST** be locked before using the seat.

ACCIDENT PREVENTION

Use the device only for its intended purpose.

Use the device only when in proper technical condition.

Do not make structural modifications to the device without the manufacturer's written consent.

When adjusting any components, exercise caution to avoid pinching fingers or other body parts between moving elements.

Use the device only on a stable surface.

All wheels **MUST** remain in contact with the ground at all times during use to ensure proper balance and prevent accidents.

When using the rollator in a stationary position, the hand brakes **MUST** be locked.

CLEANING

Clean the device regularly and keep it tidy.

Light dirt and dust can be removed with a damp cloth.

Heavier dirt can be cleaned with a mild, non-corrosive detergent and warm water.

Do not use strong abrasives or chlorine-based cleaners as they may cause damage or discoloration.

Dry parts with a soft cloth.

Wheels: Clean wheels with warm water and mild detergent. Dry with a clean cloth. Lubricate periodically with multi-purpose oil or when wheels start to squeak.

MAINTENANCE

To ensure safe and proper use, inspect the device regularly according to the schedule below:

Weekly checks:

Check that all screws are tight.

Inspect brake cables for looseness or damage.

Tighten screws and nuts if necessary.

Monthly checks:

Check handle stability and condition.

Inspect brake levers for proper function.

Inspect bases (legs) for stability and looseness.

Quarterly checks (every 4 months):

Inspect wheels to ensure they rotate freely and are undamaged.

Check the bag for wear or damage.

Inspect the frame and supports for cracks or deformations.

TRANSPORT

The rollator folds easily, making it convenient to transport. For transportation, fold it as described above. Always transport the rollator securely to prevent sudden, unsafe movements.

REUSE

The rollator is suitable for reuse. Before passing it to another user, it must be inspected by a qualified dealer. Check the following:

Visual condition of the frame.

Secure fit of screws and nuts.

Condition of plastic parts.

Condition of wheels and bearings.

Function and condition of brakes.

If any part is damaged or broken, it must be replaced. Wheels should be replaced whenever significant wear is noticed. All loosened screws and nuts must be tightened. The product must be thoroughly cleaned and disinfected before reuse.

WARNING: Do not reuse the rollator if the frame is bent or broken.

The user manual must always accompany the rollator when changing users.

STORAGE

Store the product in a dry place. Do not expose it to direct sunlight.

DISPOSAL

When taken out of service, dispose of the product as regular household waste. Follow local waste separation regulations. Protect the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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