
Rica 820066


Rica 820076

Opta 820075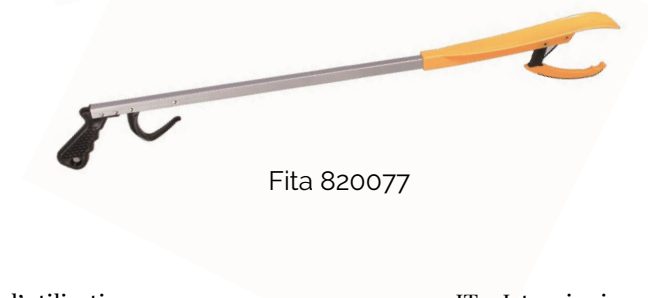

Fita 820077

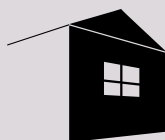DOMICILE - HOMECARE

Pinces de préhension Reachers Greifzangen

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EN - Instructions for installation and use

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Copyright 2025 par HERDEGEN SAS– ALL RIGHTS RESERVED

In the interests of continuous improvement, HERDEGEN SAS reserves the right to change any detail or specification in this document without prior notice.

The information contained in this document is confidential. It may only be shared with a third party with the prior written consent of HERDEGEN SAS. This document may not be reproduced or communicated, in full or in part, in any way or by any means, whether technical or electronic, by photocopying, recording, or using any information or data extraction systems.

You can obtain a new copy of this document from HERDEGEN SAS by contacting your regional sales representative or directly contacting HERDEGEN SAS.

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FOREWORD

You have just purchased a HACARE product and we are pleased to have you as one of our clients. We also thank you for placing your trust in our company and our products.

Your trust assures us that the HACARE company continues to meet your specific requirements and needs. That's why we always strive to innovate, providing you with quality solutions and products that improve the comfort and quality of patient care while making the caregiver's daily tasks easier.

HACARE products have gained the confidence of many professionals and institutions due to their functional and aesthetic qualities, and are now considered to be a solid benchmark. Simplicity, reliability and safety are the hallmarks of our electrical healthcare cradles.

You've chosen HACARE, and we're committed to satisfying your needs.

However, you are responsible for the proper use of the cradles. That's why we invite you to familiarize yourself with the technical specifications, and the instructions for use and maintenance of the equipment by reading through this manual.

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EN - Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place. If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. SAFETY INDICATION

- ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- Do not store outside and do not leave under the sun (Beware: do not leave the product in a car in case of high outside temperature).
- Use only as a reacher and in the intended way.
- No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- Always assess the suitability of an item before picking it up; do not attempt to carry heavy or fragile items with your reacher.
- Do not under any circumstances attempt to use the reacher to lift or move hot objects, this could lead to serious injury.

- The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear

2. INSTALLATION

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

The reachers are designed to assist in reaching and lifting items from the floor, table or shelf.

Using:

Using the reacher is simple, place the grip over the object you wish to lift and squeeze the trigger. Ensure the moving finger of the grip and the main frame of the reacher are properly secured around the object and lift.

Place the object you have lifted safely onto a flat and level surface and release the trigger.

«Opta» reacher is equipped with a non slip rubber for a better grip.

References 820066 and 820076 are equipped with a magnet allowing to grab small metallic objects.

Reference 820077 is equipped with a shoehorn and can be used as a long shoehorn for many people with flexibility limitations. Use in a seated position.

Maximum load: 2 kg.

3. CLEANING AND DISINFECTION

The product has to be cleaned regularly and after each use with a mild cleaning agent and a soft cloth. It may be washed with soap and water or disinfected with non-chlorinated products. The product must be dried after each cleaning.

4. DISPOSAL

Dispose of this product in accordance with your regional waste disposal regulations.

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