**INSTRUKCJA OBSŁUGI INHALATOR KOMPRESOROWY**  
**INSTRUCTION MANUAL COMPRESSOR INHALER**  
**GEBRAUCHSANWEISUNG KOMPRESSOR-INHALATOR**  
**NÁVOD K POUŽITÍ KOMPRESOROVÝ INHALÁTOR**  
**NÁVOD NA POUŽITIE KOMPRESOROVÝ INHALÁTOR**  
**MANUEL D'INSTRUCTIONS INHALATEUR À COMPRESSEUR**  
**GEbruiksaanwijzing COMPRESSOR INHALATOR**  
**MANUAL DE INSTRUCCIONES INHALADOR COMPRESOR**  
**MANUALE DI ISTRUZIONI INALATORE A COMPRESSORE**  
**BRUKSANVISNING INHALATOR MED KOMPRESSOR**

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid damage to the product.

**NOTE:** Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

### USE/INDICATIONS

The inhaler is designed for inhalation during the treatment of various respiratory conditions, including asthma, or bronchial and lung diseases. The inhaler is a quiet and easy-to-use nebulizer for the whole family, designed to support treatment of the lower respiratory tract. The small size of the particles produced makes it easy for the medicine to penetrate the lungs and settle in their alveoli, thus guaranteeing high effectiveness.

### CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

**COMPONENTS:** compressor housing, cover, air vents, on/off switch, power cord, storage compartment (for accessories), air outlet, handle

### SPECIFICATIONS:

Type: pneumatic

Supply voltage: 220 V / 50 Hz 120 V / 60 Hz

Power consumption: 180 VA

Nebulizer volume: 6 ml

Particle size: inhalable fraction 0.5 to 5  $\mu$ m 80%

MMAD: 2.0  $\mu$ m

Average inhalation rate: more than 0.2 ml / min.

Volume: < 60 DbA

Compressor pressure: 30-45 psi

Operating pressure: 12-45.5 psi (0.85-1.1 bar)

Flow rate: 8-10 l / min

**DELIVERY RANGE:** inhaler, mouthpiece, tubing, baby mask, adult mask, air filter (5 pieces).

### NOTES:

- Do not use the device in the presence of any anesthetics, flammable mixtures or oxygen.
- Improper operation of the device may affect electromagnetic interference, exceeding the limits indicated by the applicable European standards. If the device interferes with the operation of another electrical device, unplug it and plug it into another electrical outlet.
- In the event of a malfunction or failure, do not open the compressor housing, as it is not a self-service product.

The use of non-original spare parts, such as a filter, may cause the device to malfunction.

When using any electrical device, you should always follow certain safety rules, such as:

- use only original parts and components
- do not immerse the device in water, it is not waterproof
- do not touch the device with wet/humid hands
- do not leave the device outside

- during operation, the device must stand stably, on a flat surface
  - make sure that the vent holes are not clogged
  - do not allow unaccompanied children to use the device
  - do not turn off the device by disconnecting the plug from the electrical outlet
- Before connecting the device, make sure that the electrical voltage shown on the nameplate on the bottom of the device matches the voltage at the electrical outlet
  - Do not use an extension cord or splitter
  - Do not leave the appliance plug in the outlet when the appliance is not in use. Unplug it from the outlet when not in use.
  - Installation of the device should be carried out in accordance with the recommendations. The manufacturer is not responsible for incorrect installation.
  - The user is not allowed to replace the power cord./ If the cord is damaged, please contact the customer service/service department.
  - The power cord should always be fully extended due to the possibility of dangerous overheating.
  - Before cleaning or replacing the filter, disconnect the device from the main power supply by unplugging the plug or only the main power switch.
  - Some parts of the device are small and can be swallowed by children. Unsupervised children and people with disabilities should be prevented from accessing the device.
  - If the user decides not to use the device, it is recommended to dispose of it in accordance with local regulations.
- It is important to note that the device can only be used with medications prescribed by a doctor, without selecting medications yourself.
- Free to use only those recommended by the doctor, necessary for the therapy.

#### NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person or
- (b) temporary or permanent deterioration of the health of the patient, user or other person or
- (c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

#### NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

**NOTE: it is forbidden to use the product other than for its intended purpose .**

**NOTE:** The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.

#### TARGET PATIENT GROUP

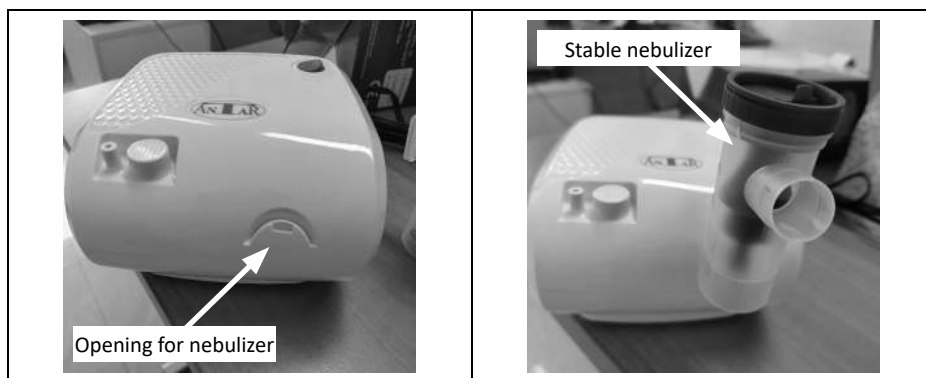
Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

### USE

1. Before using it for the first time, check the device for any damage caused during transportation. During therapy, sit upright and relax.
  - (2) Before using the device, clean it as described in the "CLEANING AND MAINTENANCE" section.
  3. Open the nebulizer by turning the top counterclockwise.
  4. Make sure the baffle is set in the correct position inside the nebulizer chamber.
  5. Place the assigned amount of drug in the drug container.
  6. Close the nebulizer by holding the device by the bottom and rotating the top in a clockwise motion. Make sure the device is firmly closed.
  7. Connect one end of the air tube to the nebulizer and its other end to the air outlet of the device.
  8. connect the angle mouthpiece to the nebulizer.
  9. If a mask is used, disconnect the angle mouthpiece and connect the mask directly to the nebulizer.
  10. connect the compressor to an electrical outlet.
  11. to start therapy, turn on the device by setting the button to the "I" position
- NOTE: DEVICE CYCLE: 30MIN ON/30MIN OFF
12. When the therapy is finished, turn off the device by changing the position of the button to "0" and unplug the device from the electrical outlet.

### ADDITIONAL FEATURE

The nebulizer can be mounted to the main unit in a stable way by plugging it into the hole

### CLEANING AND CARE

Cleaning the compressor: the exterior of the device should be cleaned with a dry cloth and mild detergents.

**NOTE:** Before cleaning, make sure the power cord is unplugged, and during cleaning, be careful that the internal parts do not come into contact with liquids.

#### Clean the nebulizer after each therapy:

The nebulizer can be cleaned with liquid soap and water and then disinfected with boiling water ( 10 min max). It is recommended to disinfect the mask and angle mouthpiece with cold disinfectant fluids. Dry the nebulizer with a clean cloth.

**CAUTION:** Do not clean the angle mouthpiece, air tube or mask in boiling water or autoclave.

**NOTE:** It is important to follow the cleaning instructions to ensure optimal operation.

#### Replacing the nebulizer:

Nebulizers should be replaced after a long, period of use or if it is clogged by dried medication. With normal use, the nebulizer should remain functional for at least 6 months.

**Filter replacement:**

The filter should be replaced after about 30h of use or when it looks dirty.  
Open the filter holder, then remove the old filter and insert the new one.

Do not use the device without a filter.

**PROBLEM SOLVING;**

1. the device does not turn on:

- make sure the plug is properly connected to the power supply
- make sure that the device operates within operating limits (30 min off/30 min on)

2 The device does not atomize or atomization is weak:

- Make sure the ends of the air tube are firmly connected to the main unit and to the nebulizer
- check that the nebulizer is not empty or overfilled with drug (max. 6 ml)
- check that the nebulizer nozzle is not clogged
- check that the baffle is placed correctly

**MAINTENANCE AND REPAIR**

Absolutely do not open the device. The device does not contain serviceable parts. The compressor does not require maintenance.

**STORAGE:** Store the product in a dry cool room, away from moisture and direct sunlight.

**METHOD OF DISPOSAL OF THE PRODUCT**

The product is an electrical waste - please proceed in a manner appropriate for the disposal of electrical and electronic equipment.

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#### WARRANTY CARD

Model: .....  
Serial number: .....

Point of sale stamp  
and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawisłńska 43 Tel. 22 518 36 00,  
Fax 22 518 36 30 www.antar.net

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