AT04501, AT04502, AT04503, AT04505, AT04506
ORTEZA KRĘGOSŁUPA INSTRUKCJA UŻYTKOWANIA
SPINAL ORTHOSIS USER MANUAL BENUTZERHANDBUCH
FÜR WIRBELSÄULENORTHESEN NÁVOD K POUŽITÍ K
ORTEŽÁM ZAD A BEDERNÍM PÁSŮM NÁVOD NA
POUŽITIE ORTÉZA CHRBTICE, BEDROVÝ PÁS MANUEL
D'UTILISATION DE L'ORTHÈSE RACHIDIENNE SPINALE
ORTHESE GEBRUIKERSHANDLEIDING
MANUAL DE USUARIO DE LA ÓRTESIS ESPINAL
MANUALE D'USO DELL'ORTESI SPINALE
ANVÄNDARHANDBOK FÖR SPINALORTOS

PL

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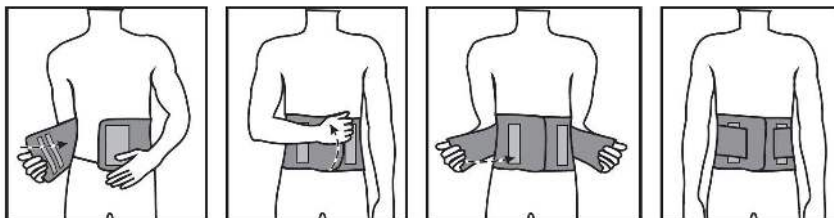
Schemat zakładania ortez AT04502, AT04503, AT04506

ENG

Thank you for purchasing our product. Do not use the product until you have read and understood these instructions. If you do not understand the warnings, cautions or instructions, contact your healthcare professional or dealer to avoid damage to the product. The products have a wide range of adjustment. The orthoses can be quickly and easily adjusted to the patient's body shape.

Velcro fastenings guarantee effective immobilisation.

NOTES:

The orthoses should be used as recommended by a doctor or physiotherapist. The products should not be used in the case of open wounds, abrasions of the epidermis or after prior application of warming gel or ointment.

CAUTION:

In the event of a device-related "serious incident" that has directly or indirectly led to, could have led to or is likely to lead to any of the following:

- (a) death of a patient, user or other person, or
 - (b) temporary or permanent deterioration of the health of a patient, user or other person, or
 - (c) a serious risk to public health
- the above 'serious incident' must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

ATTENTION: it is prohibited to use the product other than for its intended purpose.

NOTES:

- Before each use, ensure that there is no abrasion of seams and individual components on the product.
- Use of the product contrary to the instructions may result in serious complications.
- The brace should be used as prescribed by a doctor or physiotherapist.
- The product should not be used in case of open wounds, skin abrasions or after the application of warming gel or ointment.
- The night-time use of the orthosis is only possible if recommended by the attending physician.
- The orthosis should not be worn for more than 4 hours a day unless otherwise prescribed by the doctor.
- The orthosis should be used as prescribed by the doctor or physiotherapist.

ATTENTION: The manufacturer is not responsible for damage caused by negligent cleaning or failure to follow the instructions in this manual.

CONTRAINDICATIONS

physical or mental limitations (e.g. visual impairment) which prevent safe handling of the product.

WASHING INSTRUCTIONS

- The brace may only be washed by hand in lukewarm water (up to 40°C) with a mild detergent (e.g. grey soap).
- Do not iron.
- Do not dry clean.
- Do not use bleach.
- Protect from moisture.
- Do not expose to high temperatures.
- Do not tumble dry.
- Dry unfolded, squeeze gently before drying.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

HOW TO DISPOSE OF THE PRODUCT

The medical device can be disposed of as normal municipal waste.

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/size/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

AT04501

Spinal orthosis AT04501 is used to strengthen the spine in the sacral, lumbar and thoracic sections. It is made of a unique composition of two meshes, the outer one - strong and load-bearing and the inner one - soft and comfortable in contact with the body. This makes the brace extremely breathable, ensuring excellent drainage of excess heat and sweat, as well as "breathing" of the body under the brace. The brace has tightening straps around the shoulders with a special soft zone to prevent excessive pressure on the shoulders and armpits. The sewn-in reinforcements, lateral tighteners, as well as the circumferential shoulder straps provide excellent bracing. The product is also equipped with a system of tapes and cut-outs that facilitate its application for people with arm paresis.

APPLICATION

The brace is designed for various spinal disorders, e.g. multilevel discopathy, osteoporosis, degenerative changes, spinal instability, lumbar spondylolisthesis, osteoporotic compression fracture of the spine, sciatica, lumbosacral pain syndrome and post-surgery pain syndrome.

Sizes (circumference around the waist): S (85-95 cm), M (95-105 cm), L (105-115 cm), XL (115-125 cm),

XXL (125-136 cm), XXXL (136-145 cm).

AT04505 Torso brace, rigid, high, with braces - DESCRIPTION AND APPLICATION

The AT04505 spinal brace is designed to stabilise and strengthen the spine in the sacral, lumbar and thoracic sections. It is made of elastic orthopaedic tapes, thanks to which the orthosis is extremely breathable, ensuring excellent drainage of excess heat, as well as "breathing" of the skin under the orthosis. The product consists of two parts: a high lumbosacral brace and a Velcro-fastened dorsal part, which allows the appropriate size adjustment to the patient's height. The product is additionally equipped with a system of straps allowing the adjustment of pressure to individual patient needs.

APPLICATION

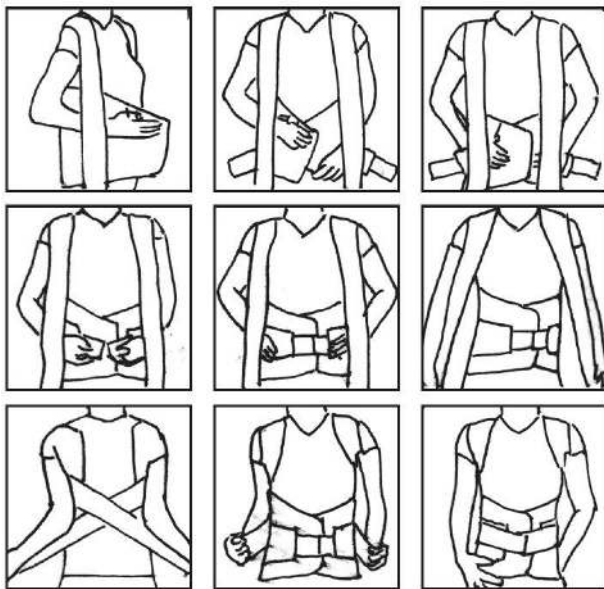
The brace is designed for various spinal conditions, e.g. osteoporosis of the spine, increased muscle tension due to lumbar discopathy, lumbosacral pain syndromes and spine instability.

Height of orthosis: 54cm (length of metal supports 44cm)

Sizes (circumference around the waist): S (80 - 95cm), M (90-100cm), L (95-105cm), XL (105-115cm), XXL (115 - 125

cm), XXXL (125-135cm)

Orthosis application diagram AT04501 and AT04505

AT04502

Spinal support AT04502 is designed to strengthen the spine in the sacral and lumbar sections. It is made of a unique composition of two meshes, the outer one - strong and load-bearing and the inner one - soft and comfortable in contact with the body. This makes the brace extremely breathable, ensuring excellent drainage of excess heat and sweat, as well as "breathing" of the body under the brace. The brace has a system of 6 reinforcements (4 with a length of 28 cm and 2 with a length of 24 cm) and lateral tightening straps, providing excellent strengthening of the lumbosacral spine. The product is also equipped with a system of straps and cut-outs to facilitate its application for people with arm paresis.

APPLICATION

The support is intended for the treatment of various spine disorders, e.g. multilevel discopathy, osteoporosis, degenerative changes, spinal instability, lumbar spondylolisthesis, osteoporotic compression fracture of the spine, sciatica, lumbosacral pain syndrome and post-surgical pain syndromes.

Height of the orthosis - 30 cm.

Sizes (circumference around the waist): S (80 - 90 cm), M (90 - 100 cm), L (100 - 110 cm), XL (110 - 120 cm),

XXL (120 - 130 cm), XXXL (130 - 140 cm), XXXXL (140 - 150 cm).

AT04503

The lumbosacral belt is used to strengthen the spine in the lumbar region. The belt has a system of 4 reinforcements with a length of 20 cm and side tightening straps to ensure good strengthening of the lumbar spine. The product is also equipped with a system of tapes and cut-outs to facilitate its application for people with arm paresis.

APPLICATION

The belt is applied in various spine disorders, e.g. lumbar discopathy; degenerative spine disease; root syndromes (sciatica).

Belt height - 23 cm.

Sizes (circumference around the waist): XS (70 - 80 cm), S (80 - 95 cm), M (90 - 100 cm), L (95 - 105 cm),

XL (105 - 115 cm), XXL (115 - 125 cm), XXXL (125 - 135 cm).

AT04506

The lumbosacral orthosis is used to strengthen the lumbar spine. It is made of elastic orthopaedic tapes that are friendly to the skin, thanks to which the orthosis is extremely breathable, ensuring excellent drainage of excess heat, as well as "breathing" of the skin under the orthosis. The product has a system of 4 reinforcements in the form of 20 cm long stays and lateral tightening straps, providing excellent strengthening of the lumbar spine. The product is equipped with a system of tapes and cut-outs to facilitate its application for people with arm paresis. The orthosis fastens in an asymmetrical manner which allows for a better fit to the patient's body within the range of one size.

APPLICATION

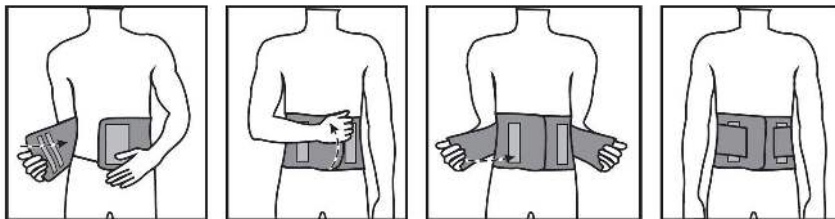
The product is applicable to various spinal disorders: thoracolumbar degeneration, thoracolumbar discopathy, degenerative spine disease, thoracolumbar spondylolisthesis, thoracolumbar overload conditions.

Orthosis height - 24 cm.

Sizes (circumference around the waist): S (60 - 75 cm), M (76 - 90 cm), L (91 - 100 cm), XL (101 - 115 cm),

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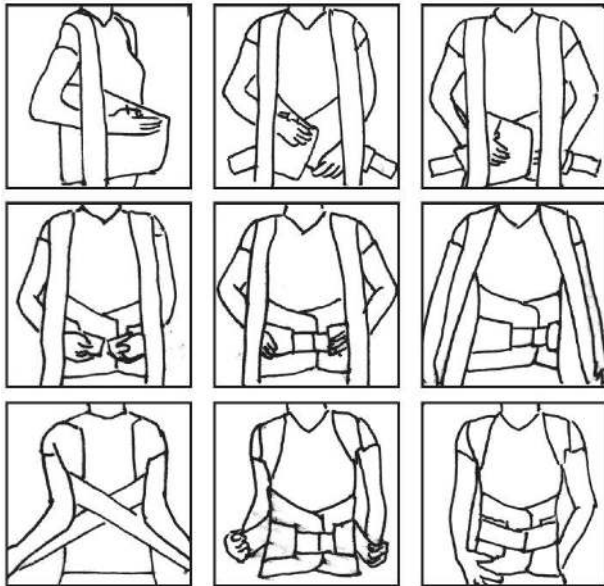
Storlekar (omkrets runt midjan): S (80 - 95cm), M (90-100cm), L (95-105cm), XL (105-115cm), XXL (115 -125cm), XXXL (125-135cm)

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Storlekar (omkrets runt midjan): S (80 - 90 cm), M (90 - 100 cm), L (100 - 110 cm), XL (110 - 120 cm), XXL (120 - 130 cm), XXXL (130 - 140 cm), XXXXL (140 - 150 cm).

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Storlekar (omkrets runt midjan): XS (70 - 80 cm), S (80 - 95 cm), M (90 - 100 cm), L (95 - 105 cm), XL (105 - 115 cm), XXL (115 - 125 cm), XXXL (125 - 135 cm).

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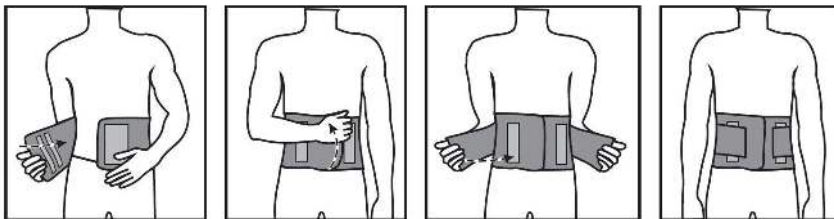
Storlekar (omkrets runt midjan): S (60 - 75 cm), M (76 - 90 cm), L (91 - 100 cm), XL (101 - 115 cm), XXL (116 - 125 cm), XXXL (126 - 140 cm).

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Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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