AT52309

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After unpacking, please ensure that all components are free from visible mechanical damage, otherwise contact the manufacturer.

SAFETY RULES

- 1) Only get on and off the product with the brakes adjusted and applied;
 - 2) Do not use the brakes while driving (only when stationary)!
 - 3) When leaning forward, take care that your body weight does not shift too far forward. To increase the stability of the wheelchair, make sure that the front wheels are facing forwards before leaning forward!
 - 4) Do not lean forward too much and do not lean sideways out of the wheelchair!
 - 5) Only use the wheelchair on surfaces where all four wheels of the wheelchair are in contact with the ground and this contact allows the wheels to drive.
 - 6) Carry the wheelchair by grasping the frame structure. Never carry by other parts of the product.
 - 7) Do not use if there is any visible or perceptible damage, or if the degree of wear on any part of the wheelchair indicates a potential risk of accident (wear of wheels, damage, damage to upholstery).
 - 8) Special care should be taken when folding/unfolding the product, lack of attention can lead to the risk of fingers being trapped by moving parts of the wheelchair.
 - 9) When getting in and out of the wheelchair, do not stand on the footrests as this may lead to the wheelchair tipping over;
 - 10) When driving over thresholds or other obstacles, extreme care should be taken when transferring all weight to the rear wheels. If there is any concern as to whether this manoeuvre can be carried out safely, it is imperative that a second person is requested to assist.
- ATTENTION:** Incorrect or careless shifting of the load to the rear wheels can lead to the wheelchair tipping over backwards, which is a very dangerous event and can cause serious injuries! If you are in any doubt as to whether it will be possible to safely pass an obstacle, it is imperative that you ask for help from another person!
- 11) When moving the wheelchair, take care that your fingers do not get between the spokes. This could lead to hand injury.
 - 12) When driving on inclines, take extra care and do not allow the wheelchair to accelerate. If you find it difficult to safely pass an incline, it is essential that you ask another person for assistance.
 - 13) Avoid contact between the wheelchair and open flames. If textile, plastic and rubber parts come into direct contact with fire, they may catch fire.

CLEANING/MAINTENANCE/STORAGE

- Periodically check the condition of the wheelchair and make maintenance by:

- a) checking the condition of the tyres - if they show heavy wear, they should be replaced at the same time;
- b) checking the tightness of the bolts of the connecting parts;
- c) cleaning the wheelchair of dirt with a damp cloth;
- d) cleaning of upholstery with a damp cloth, possibly with a mild cleaning agent;

e) to maintain the performance of the product it is extremely important to keep all parts of the wheelchair clean and to store the wheelchair properly. Storing the wheelchair in rooms with very high humidity (laundry rooms, etc.) or in the open air will cause a fast loss of its useful properties. Store in a dry and cool area away from direct sunlight.

f) do not expose the pram to extreme weather conditions (rain, sun, snow).

g) do not use solvents, chlorine-containing agents, sharp brushes and other cleaning objects or corrosive cleaning agents.

ASSEMBLING/ HANDLING

1. after unpacking, remove the wheelchair and check the product for damage.
2. unfold the backrest until a characteristic clicking sound is heard.
3. using two hands, unfold the frame of the pushchair sideways, press the seat down to fully unfold the pushchair.
4. hook the footrests into the holes provided and lock in place by twisting.
5. fit the footrest plate and screw in the nut/knob.
6. the rear wheel can be freely removed if required using the quick release.
7. push the brake lever forward to brake.

DISASSEMBLING

All the above-mentioned steps must be performed in reverse order.

DRIVING THE WHEELCHAIR/ACCESSING OBSTACLES:

- When selecting a wheelchair, in order to purchase the best model, we advise you to seek medical advice.
- If for any reason you are unable to pass an obstacle with your wheelchair, you should seek assistance from another person. Attempts to pass an obstacle on your own may result in the wheelchair tipping over and injury.
- Descending a small threshold - position yourself forward, perpendicular to the axis of the threshold, ensure that the footrests do not overlap the ground and drive off the threshold. Avoid leaning forward;
- Always ask for help when passing obstacles or stairs;

TECHNICAL SPECIFICATIONS

- seat width (mm): 450 / 430
- seat height (mm): 490
- seat depth (mm): 410
- total length (mm): 1070
- backrest height (mm): 370
- overall width (mm): 660 / 630
- width after folding (mm): 335
- front wheel diameter ("): 8
- rear wheel diameter (lane wheel) ("): 24
- maximum load (kg): 136
- packaging size (mm): 820*355*780
- net weight (kg): 13 / 12.6

THIS SYMBOL INDICATES THE
MAXIMUM WEIGHT OF THE USER

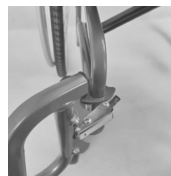
1.

2.

3.

4.

5.

NOTE: It is forbidden to stand on the footrests as this can lead to a dangerous fall and injury.

ATTENTION

- the wheelchair has a folding frame - it is folded by pulling the central part of the seat upwards and unfolded by pressing the sides of the seat downwards. Make sure that the sides of the seat rest accurately in the plastic hooks. Failure to unfold the seat accurately can result in the catches breaking and damaging the wheelchair.

CAUTION:

In the event of a device-related „serious incident“ that has directly or indirectly led to, could have led to or may lead to any of the following:

- (a) death of a patient, user or other person, or
 - (b) temporary or permanent deterioration of the health of a patient, user or other person, or
 - (c) a serious risk to public health
- this „serious incident“ must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTE:

In case of pain, allergic reactions or other disturbing, unclear to the user symptoms related to the use of the medical device, consult a healthcare professional.

ATTENTION: The manufacturer cannot be held responsible for damage caused by negligent maintenance, inadequate servicing or as a result of not following the instructions in this manual.

ATTENTION: It is forbidden to use the product other than in accordance with its intended use!

ATTENTION: During use and handling of the product and when assembling and adjusting the mechanisms, there may be a danger of entrapment and/or compression of parts of the user's/companion's body in openings/gaps between components. Perform these operations with particular care.

NOTE: There may be a risk of tipping over if the product is used incorrectly. Please follow the recommendations for getting on/out/moving.

Once the adjustment has been completed, stabilise the positions by tightening the nuts/bolts carefully.

PURPOSE

The wheelchair is intended for the mobility of a disabled person or a person with mobility difficulties.

6.

7.

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling.

DELIVERY SCOPE: wheelchair, footrests x2, adjustment key, calf restraint strap, instruction manual

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

HOW TO DISPOSE OF THE PRODUCT AT END-OF-LIFE

Once the device has been withdrawn from service, the medical device can be disposed of as normal municipal waste.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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