## **AT52316**

**INSTRUKCJA OBSŁUGI, WÓZEK INWALIDZKI  
USER MANUAL, WHEELCHAIR  
GEBRAUCHSANWEISUNG, ROLLSTUHL  
NÁVOD K POUŽITÍ - INVALIDNÍ TRANSPORTNÍ VOZÍK  
NÁVOD NA POUŽITIE VOZÍK INVALIDNÝ  
MANUEL D'UTILISATION FAUTEUIL ROULANT  
GEBRUIKSAANWIJZING ROLSTOELWAGEN  
MANUAL DE INSTRUCCIONES CARRO PARA SILLA DE RUEDAS  
MANUALE ISTRUZIONI CARRELLO CARROZZINA  
BRUKSANVISNING RULLSTOLSVAGN**

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En

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

**NOTE:** Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

#### **APPLICATION**

The wheelchair is designed For transporting people with disabilities or who have difficulty moving independently. It serves as a support during movement. Also recommended for the elderly and severely limited mobility.

#### **CONTRAINDICATIONS**

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

- People who have difficulty maintaining a stable sitting position due to muscle weakness or other neurological problems may need a specialized wheelchair with additional support.
- Blood pressure problems - a change in position from lying to sitting can cause blood pressure problems, such as orthostatic lowering of blood pressure, which can be dangerous.
- The use of a wheelchair may not be appropriate for people experiencing acute pain, which can be exacerbated by sitting in one position for long periods of time.
- Surgical restrictions - after certain surgeries, such as spinal surgery or major abdominal surgeries, it may be advisable to avoid sitting in a wheelchair for a certain period of time.
- People who are at particular risk of developing bedsores may need special wheelchairs equipped with anti-bedsores mats or other solutions to reduce their risk.

#### **FEATURES**

- lightweight aluminum frame
- reclining armrests
- adjustable footrest length
- locking device to prevent folding of the frame
- parking brakes for the caregiver

#### **COMPONENTS**

Wheelchair, manual

#### **NOTES:**

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person or
- (b) temporary or permanent deterioration of the health of the patient, user or other person or
- (c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

#### **NOTES:**

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

**NOTE: It is forbidden to use the product other than for its intended purpose.**

**NOTE:** The manufacturer is not responsible for damage caused by negligent maintenance of the product, use of the product in a manner inconsistent with the recommendations, or resulting from failure to follow the notes contained in this manual.

**CAUTION:** If the product is not used properly, there may be a risk of tipping over. Please use extreme caution when getting on/off and moving the wheelchair.

**NOTE:** Before each use of the product, check that all components are in the correct position and in good working condition.

**CAUTION:** Do not lean over the side of the wheelchair, as it may lead to loss of balance and falling.

**CAUTION: Standing on the footrests is prohibited. Doing so may cause the wheelchair and the user to tip over and cause serious injury.**

#### **GENERAL WARNINGS**

- When getting in or out of a wheelchair, make sure both brakes are on.
- Steep and uneven surfaces should be avoided, as this can cause the product to tip over.
- Avoid soft surfaces, as the wheels may collapse.
- Move the wheelchair smoothly in all conditions.
- Always observe the surroundings to identify hazards, such as furniture, doors, holes, hills, curbs, entrances, etc.
- Keep both feet on the footrests at all times, and ensure that clothing is properly tucked and does not block the wheels.
- Always drive carefully on hills.
- Always keep both hands on the push handles for proper control of the wheelchair (applies to the caregiver).
- Always remember to maintain balance.
- Keep the wheelchair away from fire sources.
- Under no circumstances should the maximum load of the product be exceeded.

- Do not hang any heavy objects on the back of the wheelchair, as this may affect the stability of the product especially when climbing hills.
- Only one person can use the wheelchair at a time.
- Do not use a wheelchair when the patient is unconscious.
- Do not use other means of transportation to push the wheelchair.
- Do not lift the front wheels while driving, as you may risk falling and seriously injuring yourself.
- Do not get into a wheelchair while moving.
- Do not drive the wheelchair too fast and do not make sudden turns.
- Do not drive a wheelchair across a hill (in either direction) with a slope of more than 7 degrees.
- Do not stand on the footrests when getting on or off the product.
- Do not climb stairs using a wheelchair.
- Do not attempt to lift the wheelchair with moving (removable) parts. Lifting with moving parts of the product may lead to injury to the user and/or caregiver or damage to the product.
- Do not get the product wet.

#### TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device independently or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

#### SPECIFICATION

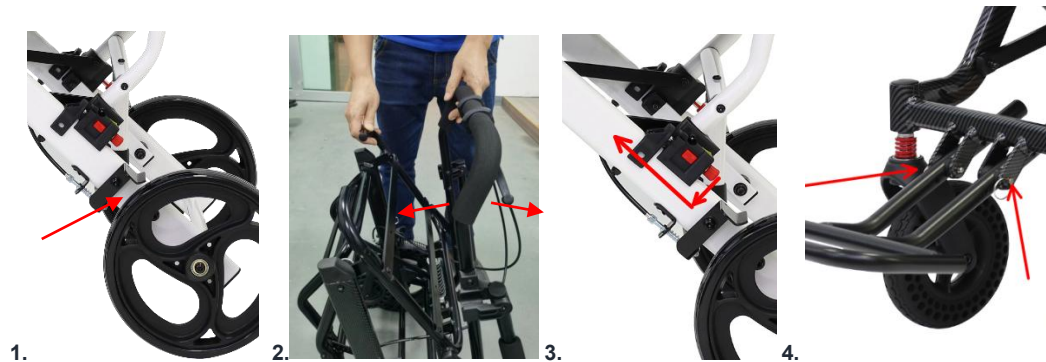
Dimensions when folded (w.L.H.): 51x36x75 cm  
 Length: 96 cm  
 Width: 51 cm  
 Height: 90 cm  
 Seat width: 42 cm  
 Seat depth: 40 cm  
 Height from the ground to the seat: 48 cm  
 Backrest height: 37 cm  
 Footrest: 4 degrees of length adjustment (in 2.5 cm increments)  
 Front wheel: 6"  
 Rear wheel: 8"  
 Type of wheels: solid  
 Shock absorber: front  
 Maximum load: 100 kg  
 Net weight: 7.5 kg  
 Package Size: 54\*29\*78 cm  
 Frame: aluminum

#### SYMBOLS

|                                                                                     |                                                                                                            |
|-------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------|
|  | Manufacturer                                                                                               |
|  | Certification of assessment of conformity of the product with the essential requirements of EU regulations |
|  | Medical device                                                                                             |
|  | Lot number                                                                                                 |
|  | Production date                                                                                            |
|  | Read the instruction manual before use                                                                     |
|  | Warning                                                                                                    |
|  | This symbol indicates the maximum weight of the user                                                       |

#### DISCLAIMER


1. Before unfolding the frame, release the safety lock by pressing the red button (fig.1)
2. Then unfold the frame by pulling the seat and push handle outward until it is fully unfolded. Press down on the seat to make sure the wheelchair is properly unfolded.
3. Please unfold the footrests and adjust their length using the length lock (fig.4 )
4. Finally, unfold the armrests

#### SUBMITTING

To assemble the product, all the above-mentioned steps must be performed in reverse order.
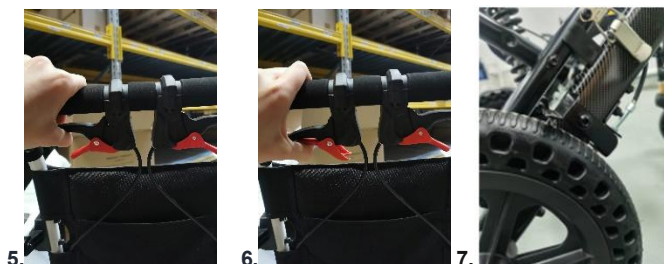
#### USE OF THE BRAKES

- To brake, press the black handle.
- To release the brake lock, press the red handle.

#### CLEANING

In order to prolong the life of the wheelchair, it is recommended to clean it regularly (preferably daily) especially if it is used in rainy or snowy weather. We recommend that you do not use the product during rain or snow.

Clean the product with a damp cloth and mild detergent.

**CAUTION:** Do not use rough objects, strong detergents or solvents for cleaning.

#### CONSERVATION

All parts of the wheelchair should be checked periodically. Inspect the frame and fixed components before each use. The condition of the wheels should be checked once a week.

To ensure the long-term and safe operation of the product, it is recommended that all screws and mounting components be inspected regularly. Before each use of the product, check that all screws are properly tightened and there are no signs of loosening.

Regularly check that the product is not missing any components and that all components are in good working condition. If you notice wear, mechanical damage or missing parts, immediately stop using the product and contact the manufacturer's service department to avoid the risk of an accident or further damage.

Pay attention to any unusual sounds or vibrations that may indicate loose parts or other technical problems.

Remember to use only original parts recommended by the manufacturer when replacing screws or components.

#### STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

#### METHOD OF DISPOSAL OF THE PRODUCT

For its proper disposal, please contact your local authority or the nearest waste collection point to find out how to properly dispose of the product. By acting in accordance with local regulations, you are helping to protect the environment.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawiślańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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