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### **AT53044 ORTEZA NADGARSTKA**

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### **WRIST ORTHOSIS**

**General information**

The wrist orthosis is made of high-quality fabric, providing excellent strength, flexibility and comfort, as well as a proper fit. The built-in silicone pad provides a gentle pressure massage to the soft tissues, which increases blood flow and accelerates the reduction of swelling. It provides firm and effective support and prevents injury. The product is designed to provide maximum support while maintaining a comfortable fit. The product is suitable for both everyday use and sports use. It is important to fit the correct size to ensure maximum benefit and to avoid excessive compression. If the orthosis is used after an injury, seek medical advice if the condition does not improve within 72 hours. If the injury is severe or if you have any doubts, seek medical advice immediately. Adult supervision is required for use by children. Keep out of the reach of small children.

**NOTES:**

In the event of a product-related 'serious incident' that has directly or indirectly led to, could have led to or is likely to lead to any of the following:

- a) the death of a patient, user or other person, or
- b) a temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

this 'serious incident' should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

**NOTES:**

If you experience pain, allergic reactions or other distressing, unclear symptoms related to the use of the medical device, consult a healthcare professional.

**WARNING:**

If you have diabetes or poor circulation, consult your doctor before using the product. If at any time you experience pain, rash or an increase in pain, discontinue use and consult your doctor.

**WARNING:** do not use the product if warming liquids or ointments have been applied, do not use in case of skin injuries!

**Indications:**

- wrist injuries,
- wrist conditions,
- rehabilitation after various wrist disorders,
- a protective measure in case of wrist weakness,
- post-traumatic changes,
- during the rehabilitation period after surgery

**Contraindications:**

- if soreness or rash occurs, discontinue use immediately and seek medical advice

**Features:**

- The orthosis ensures function within the optimal anatomical range of mobility.

**Product cleaning instructions:**

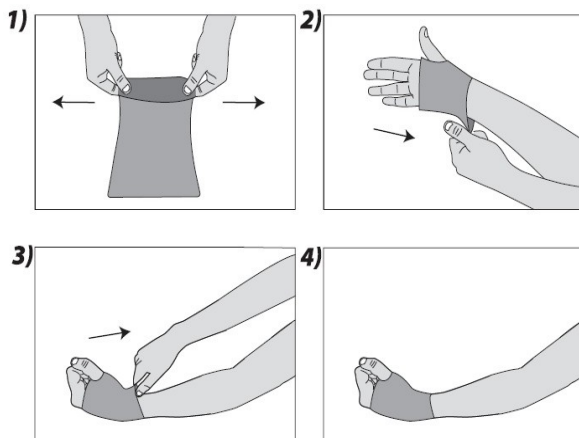
- The brace may only be washed by hand in lukewarm water (up to 40°C) with a mild detergent (e.g. grey soap).
- Do not iron.
- Do not dry-clean.
- Do not use bleach.
- Protect from moisture.
- Do not expose to high temperatures.
- Do not tumble dry.
- Dry unfolded, squeeze gently before drying.

**Storage:**

Store in a dry area.

### How to dispose of the product at end-of-life

Once the device is decommissioned, the medical device can be disposed of as normal municipal waste. The orthosis is worn by pulling it onto the hand.

To find the right size, measure the circumference of the hand (Figure 5)

Fig. 5

**Available sizes:** M (17-19 cm), L (19-21 cm), XL (21-23 cm), XXL (23-25 cm)

**The brace is available for the right and left hand.**

**Scope of delivery:** Orthosis

**Composition:** 60% nylon, 30% latex, 10% spandex

**Warnings:** do not use the product if warming fluids or ointments have been applied, do not use in case of skin damage.

### Notes:

- Read the contents of this manual before putting on the orthosis.
- Before each use, make sure there is no abrasion of the seams on the orthosis.
- Use of the product contrary to the instructions may result in serious complications.
- The orthosis should be used as recommended by your doctor or physiotherapist.
- The product should not be used in the case of open wounds, abrasions of the epidermis, or after prior application of a warming gel or ointment.
- Wearing the orthosis at night is only possible if the attending physician so recommends.
- The duration of wearing the orthosis should be consulted with your doctor.

## WARRANTY CARD

Model: .....

Serial number: .....

Seal of the point of sale

and legible signature of the Seller: .....

1) ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty for the product, starting from the date of purchase of the product by the Purchaser. For products the purchase of which is reimbursed by the National Health Fund, the warranty period is extended to cover half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.

2) During the warranty period, ANTAR undertakes to carry out the repairs necessary to restore the proper functioning of the product, within a period of 14 days from the date of receipt of the product by the service, together with the proof of purchase and the warranty card.

3) ANTAR reserves the right to replace the product if it is considered that the repair costs are not cost-effective. Repair or replacement of defective products does not extend the warranty period.

4) If the product is found to be malfunctioning, it must be sent immediately to the ANTAR address (given in point 1) or contact the point of sale.

5) Read the instructions carefully before use and follow the instructions. Use of the product contrary to its intended use and recommendations will result in irrevocable loss of warranty.

6) The warranty does not cover damage caused by external mechanical force, loading in excess of the user's maximum permissible weight as described in the instructions, or contact of the product with alcohol, oily substances or petrol. The guarantee does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.

7) A product submitted for service in connection with a complaint must not cause an outbreak.

8) THIS GUARANTEE IS ONLY VALID IF ACCOMPANIED BY PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN THE EVENT OF A CLAIM, THE PROOF OF PURCHASE MUST BE ENCLOSED. IN ORDER TO RECOGNISE THE EXTENDED WARRANTY PERIOD FOR PRODUCTS WHOSE PURCHASE WAS REIMBURSED BY THE NATIONAL HEALTH FUND, A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES THAT ARE ORTHOPAEDIC AND AUXILIARY EQUIPMENT MUST BE ATTACHED.

9) Complaints submitted without proof of purchase and the warranty card with the product serial number will not be accepted.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawisłańska 43  
Poland  
Tel. 22 518 36 00, Fax 22 518 36 30  
[www.antar.net](http://www.antar.net) e-mail: [antar@antar.net](mailto:antar@antar.net)

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