
THUASNE®

Genu PROMASTER

- de **Gebrauchsanweisung - Patellabandage mit verstärktem propriozeptiven Effekt**
en **Instructions for use - Patellar knee brace with reinforced proprioceptive effect**
fr **Consignes d'utilisation - Genouillère à effet proprioceptif renforcé**
it **Istruzioni per l'uso - Ginocchiera rotulea con effetto proprioceptivo rinforzato**
es **Instrucciones de utilización - Rodillera rotuliana con efecto proprioceptivo reforzado**
pt **Instruções de uso - Joelheira rotuliana com efeito proprioceptivo reforçado**
nl **Gebruiksaanwijzing - Knieverband voor de knieschijf met versterkte proprioceptieve werking**
da **Brugsanvisning - Knæbind med forstærket proprioceptiv virkning**
sv **Bruksanvisning - Knäskålsförband med förstärkt proprioceptiv verkan**
fi **Käyttöohje - Polvilumpion suojuksella varustettu polvituki, jolla on tehokas asento- ja liikeaistiin ohdistuva vaikutus**
el **Οδηγίες χρήσης - Συνδεσμική επιγονατίδα με ενισχυμένη ιδιοδεκτική δράση**
cs **Návod k použití - KČEŠSKOVÁ kolenní ortéza se zesíleným proprioceptivním účinkem**
pl **Sposób użytkowania - Orteza stawu kolanowego ze wzmocnionym efektem , proprioceptywnym**
et **Kasutusjuhend - Põlvekedrabandaaž tugevdatud propriotseptiivse efektiga**
lv **Lietošanas instrukcija - Ceļa kaula bandāža ar uzlabotu proprioceptīvo efektu**
lt **Naudojimo instrukcija - Sutvirtintas kelio girtelės įtvirtas su proprioceptiniu poveikiu**
sk **Návod na použitie - Kolenná ortéza so zosilneným proprioceptívnym účinkom**
sl **Navodila za uporabo - Kolenski opornica za pogačico z okrepljenim proprioceptivnim učinkom**
hu **Használati utasítás - Patella védő térdrögztítő megerősített proprioceptív hatással**
ro **Instrucțiuni de utilizare - Genunchieră rotuliană cu efect proprioceptiv pronunțat**
bg **Инструкция за употреба - Пателарна колянна ортеза с подсилено проприоцептивно действие**
hr **Upute za uporabu - Bandaža za patelu (čaišicu koljena) s pojačanim proprioceptivnim učinkom**

Größe ¹	Oberschenkelumfang ²		Wadenumfang ³	
	cm	inch	cm	inch
XXS	35 - 38	13 ¾ - 15	25 - 28	9 ¾ - 11
XS	38 - 41	15 - 16	28 - 31	11 - 12 ¼
S	41 - 44	16 - 17 ¼	31 - 34	12 ¼ - 13 ½
M	44 - 47	17 ¼ - 18 ½	34 - 37	13 ¼ - 14 ⅔
M+	47 - 50	18 ½ - 19 ¾	34 - 37	13 ¼ - 14 ⅔
L	47 - 50	18 ½ - 19 ¾	37 - 40	14 ⅓ - 15 ¾
L+	50 - 53	19 ¾ - 21	37 - 40	14 ⅓ - 15 ¾
XL	50 - 53	19 ¾ - 21	40 - 43	15 ¾ - 17
XL+	53 - 56	21 - 22	40 - 43	15 ¾ - 17
XXL	53 - 56	21 - 22	43 - 46	17 - 18
XXL+	56 - 59	22 - 23 ¼	43 - 46	17 - 18

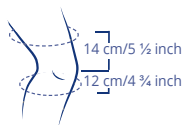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Patellar knee brace with reinforced proprioceptive effect**Purpose:**

Genu PROMASTER is a compression bandage with patella pad for treating knee irritation. The bandage may only be used for knee treatment.

Properties:

- Brace made of elastic knit
- 3D knit at the patella
- Highly elastic knit for comfort of the popliteal fossa
- Patellar pad
- Integrated lateral reinforcements
- Latex-free

Indications:

Post-traumatic, postoperative and chronic irritation; Fibromyalgia and sprains of the lateral ligaments; Osteo- and rheumatoid arthritis; Mild joint instabilities; Patellar deficiencies

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Make sure that the kneecap is properly centered in the central patellar ring. The brace should be worn directly against the skin.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

**Composition:**

78 % polyamide, 14 % elastane, 5 % cotton, 2 % polyurethane, 1 % polyester

Explanation of the reinforced proprioceptive effect:

Orthoses in the PROMASTER range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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Thuasne Cervitex Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
Thuasne Deutschland GmbH